

# AliMed® Vinyl Covered Positioning Wedges

## Instructions for Use and Care

MD

### Intended Use

The AliMed Vinyl Covered Positioning Wedges are intended to provide support for patients undergoing surgical or diagnostic procedures.

### Instructions for Use

*Reusable.*

1. Orient the wedge around the patient to ensure that the affected area is properly positioned for the procedure. The patient wedges are applicable for a wide range of procedures.

### Contraindications/Warnings



- Monitor the patient to ensure safety and proper positioning during the procedure.
- Consult your institution's guidelines for assessing the risk of each individual patient.
- Replace positioner if covering is torn or otherwise damaged, exposing the foam core.
- Report any serious incident to AliMed and the appropriate regulatory body.

### Care and Cleaning

Positioners can be wiped clean with the following active ingredients: Hydrogen peroxide, isopropyl alcohol, quaternary ammonium, and ammonium chloride.

Clean positioners after each procedure. Failure to do so increases risk of cross contamination between patients and users.

Using improper cleaning solutions or methods can result in damage to the product or irritation to the patient.

Do not soak positioners.

Do not place positioners in autoclave.

### Materials



Covering: Vinyl

Foam Core: Polyurethane foam

Not made with natural rubber latex.

### Storage/Disposal

Store in a clean, dry environment protected from direct sunlight. Storage at ambient room temperature is advised. Dispose with regular trash and in accordance with existing federal, state, and local environmental control laws.



**AliMed® Inc.**

297 High St., Dedham, MA 02026 USA

PHONE 800.225.2610 • FAX 800.437.2966

AliMed.com

©2024 AliMed® Inc. All rights reserved.

IFU-0099R01 ECO-1664

8183REV0624

# Symbol Glossary



Medical Device



Caution



Not made with  
Natural Rubber Latex



Batch Code



Manufacturer's  
catalog number



Date of  
Manufacture



Manufacturer

## Item Numbers

<b>REF</b>	<b>2970010644</b>	<b>2970011413</b>	<b>2970011606</b>	<b>2970011726</b>	<b>2970011727</b>	<b>2970011728</b>	<b>2970013014</b>
(Former Item #)	(81604)	(9-060)	(91-269)	(920405)	(920407)	(920409)	(927795)
<b>REF</b>	<b>2970013182</b>	<b>2970013374</b>	<b>2970013376</b>	<b>2970013377</b>	<b>2970013378</b>	<b>2970013383</b>	<b>2970013386</b>
(Former Item #)	(930795)	(931711)	(931713)	(931714)	(931716)	(931723)	(931726)
<b>REF</b>	<b>2970017445</b>	<b>2970017477</b>	<b>2970018232</b>	<b>2970018233</b>	<b>2970011403</b>		
(Former Item #)	(95-029)	(95-152)	(9-673)	(9-674)	(9-007)		

## Warranty

ALIMED warrants to Purchaser that the Product(s) sold hereunder, is at the time of shipment to Purchaser from ALIMED, free from defects in material and workmanship. As Purchaser's sole exclusive remedy under this warranty ALIMED agrees either to repair or replace, at ALIMED's sole option and free of charge to Purchaser, any part or parts of such Product(s) which under proper and normal conditions of use prove to be defective within six months from the date of shipment. Visit [AliMed.com/terms](https://www.AliMed.com/terms) for full warranty info.

California Proposition 65 Statement:

**65** **WARNING:** This product can expose you to chemicals including Di(2-Ethylhexyl) Phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to <https://www.p65warnings.ca.gov>.



**AliMed® Inc.**

297 High St., Dedham, MA 02026 USA  
PHONE 800.225.2610 • FAX 800.437.2966

AliMed.com

©2024 AliMed® Inc. All rights reserved.  
IFU-0099R01 ECO-1664  
8183REV0624