

Comfort Foot Single (Clamp-on or Bolt-on)

permobil

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

Instructions for Use

⚠ WARNINGS:



- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- Check daily to make sure that all hardware is fully tightened. DO NOT use the product if any part cannot be tightened or is broken.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

Clamp-on

The assembly includes one clamp and one screw ($\frac{1}{4}$ -20 x 1" or $\frac{1}{4}$ -20 x $1\frac{1}{4}$ ").

- 1 Slightly loosen the screw and then slide the product forward until the wheelchair footplate is between the clamp and the base of the product. (Fig. 1)
- 2 Using the base straps, adjust the angle of the product to the desired position. (Fig. 2)
- 3 Securely tighten the screw.

Fig. 1 and 2

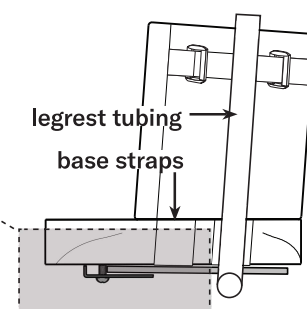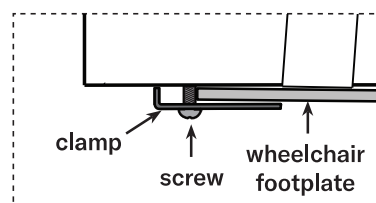

- 4 Release the hook/loop fastener strips on the side strap. Securely tighten the side strap around the legrest tubing (C). (Fig. 4, below)

Bolt-on

The assembly includes two screws ($\frac{1}{4}$ -20 x 1" or $\frac{1}{4}$ -20 x $1\frac{1}{4}$ ") and two Fender washers ($\frac{1}{4}$ x $1\frac{1}{4}$ " OD).

- 1 Place the template* on the wheelchair footplate in the desired position. Drill two $\frac{5}{16}$ " diameter holes through the wheelchair footplate, as marked on the template.

**See the other side of this page.*

- 2 Adjust the angle of the product, as desired, using the base straps.
- 3 From the bottom of the wheelchair footplate, install the screws** with the washers into the base of the product. (Fig. 3)
- 4 Release the hook/loop fastener strips on the side strap. Securely tighten the side strap around the legrest tubing (C). (Fig. 4)

Fig. 3


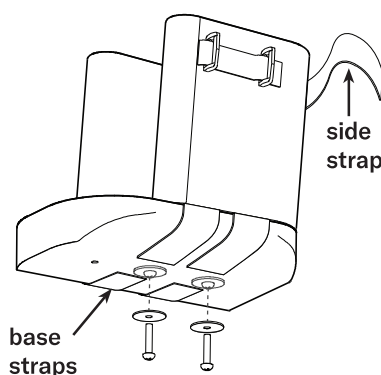



Fig. 4***

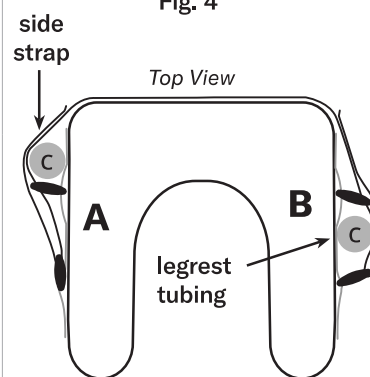

***Note about Fig. 4: Depending on the location of the legrest tubing, tighten the side strap in the (A) or the (B) configuration.

***The thickness of the wheelchair footplate determines the correct length of screws. Screws are either $\frac{1}{4}$ -20 x 1" or $\frac{1}{4}$ -20 x $1\frac{1}{4}$ ".*

For Comfort Foot Single with Gel, use $\frac{1}{4}$ -20 x 1" screws.

 The Comfort Companies, LLC
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.
Contact Information: 800.736.0925
Customer Support: orders.comfort@permobil.com
permobil.com

The Comfort Companies, LLC is a part of Permobil.
Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.
© 2013, 2021 Permobil

Comfort Foot Single
BOLT-ON
Template

