


Panacea® SoftTurn™ Limited Lateral Rotation Mattress

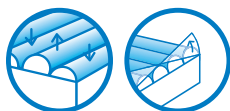

The Panacea® SoftTurn™ Mattress provides either rotational or alternating-pressure redistribution. The support surface fits most standard healthcare beds and consists of four air cells that run longitudinally underneath the entire body and are contained in a foam cavity. The air and foam flotation system aids in pressure redistribution by maintaining low interface pressures throughout the surface.

The Panacea SoftTurn Mattress uses a quiet, energy-efficient pump, which provides either alternating-pressure or rotational therapy in a 10-minute cycle. The inflation levels are designed to maintain low interface pressures throughout the mattress, to redistribute peak interface pressures during the cycle and to further relieve peak pressure points as the cycle alternates.

	SIZE		CONFIGURATION	
	Width	Length	Flat	Raised Perimeters
PANACEA® SOFTTURN	35"	76"	#C4577	#C4580
		80"	#C4578	#C4581
		84"	#C4579	#C4582

See DirectSupply.net for bariatric sizes.

Specifications	
Standard Dimensions	35"W; 7"H; 76", 80" or 84"L
Weight Capacity	350 lbs.
Therapy Style	Alternating Pressure, Lateral Rotation
Firm Perimeters	Yes
Raised Perimeters	Yes, Optional
Heel Slope	No
CPR Release	Yes
Safety Base†	Yes
Air Cell Height	5.5"
Inflation Time	Less than 45 minutes
Cover Type	4-way stretch
Limited Warranty	Mattress – 2 Years; Pump – 18 Months

†Prevention against bottoming out during loss of power or air cell puncture

Lateral Rotation Mattresses

With an ability to laterally rotate a resident up to 40 degrees, these mattresses may be appropriate for residents with pulmonary or circulatory ailments. Others can be used as an adjunct to turning – each manufacturer may have a different indication for use. Lateral rotation is often combined with low air loss or alternating pressure.



Panacea® SoftTurn™ Limited Lateral Rotation Mattress

An air mattress featuring lateral rotation with a foam topper above air cells



Foam Cavity

A foam cavity is used to contain the air cells and stabilize the entire surface, assist with ingress/egress and minimize resident falls. The mattress is also available with optional 3" raised side perimeters to further reduce resident falls.

Air Cells

Four polyurethane air cells run longitudinally from head to toe.

Side Perimeters

3"W firm perimeter provides support when entering/exiting the bed. The Panacea SoftTurn Plus Mattress has 3" raised side perimeters to help minimize falls.

Cover

Top cover – the 4-way stretch bacteriostatic top fabric is fire resistant, fluid resistant, tear resistant, cleanable and replaceable.

Bottom cover – PVC vinyl bottom for durability and easy cleaning.

Height

The Panacea SoftTurn Plus Mattress provides a 7" thick pressure management surface, and the total mattress height is 10" with raised perimeters.

Friction/Shear

The cover reduces both friction and shear.

Mattress

The mattress should be placed directly on an existing healthcare bed frame of an appropriate size.

Warranty

The mattress warranty is 2 years against manufacturing defects. The control unit warranty is 18 months.

CONTROL UNIT

Power On/Off Switch

Powers the control unit on and off.

Firmness Dial

Adjusts the overall firmness of the mattress.
Adjusts the pressure within the system.

Low-Pressure Alarm

Indicates loss of air pressure.

OPERATING MODES

Panacea SoftTurn Mattress Rotational Configuration

Rotational cycle – provides 10-minute gentle turn cycles

Panacea SoftTurn Mattress Alternation Configuration

Alternation cycle – provides 10-minute loading and unloading cycles



DirectSupply.com

APTURA / EQUIPMENT & FURNISHINGS / DSSI / TELS