

ICC - Bed IH820DLX, IH820-3MDLX, SC900DLX



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 05-019. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

**For Customer Service and Technical Support, Call:
1-800-668-2337**

**For Literature Requests, Call:
1-800-668-2337 or go to our website
www.invacare.com**

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments
NOTE: = General information pertaining to that specific page
NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length
Washers Inside Diameter X Outside Diameter X Thickness
Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2018 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.


TABLE of CONTENTS

Description	Page Number
Bed Rails - Assist	30
Bed Rails - IH5748A	16
Bed Rails - IH6801A	18
Bed Rails - IH6803A	26
Bed Rails - IH6808A	20
Bed Rails - IH6815A	22
Bed Rails - IH6816A	24
Bed Rails - IHRAILTS-DLX ThinkSoft® Positioning Device	28
Carriage Assembly - 820DLX & 820-3MDLX	8
Carriage Assembly - SC900DLX	10
Frame - Pan Deck - 820DLX, 820-3MDLX & SC900DLX	6
Frame - Wire (Grid) Deck - 820DLX, 820-3MDLX & SC900DLX	4
Manual Pull Tube & Crank Handle - 820DLX	12
Miscellaneous Accessories	32
Motors and Components, Full Electric - 820-3MDLX & SC900DLX	14
Packaging, Literature and Decals	3

Packaging, Literature and Decals

Part Number	Description
1125016	Sleeve, Carton
1125021	Carton, Cap
1125017	Insert, Carton
1125339	Carton (11 x 11 x 14")
1124031	Kit, Mattress Keeper Includes: 4 ea. Mattress Keepers, 8 ea. Nuts, 8 ea. Screws, and 1 ea. Poly Bag
1142004	Packaging, Carton Assembly 820DLX Includes: 1 ea. Carton Sleeve, 2 ea. Carton Caps, 2 ea. Carton Inserts, 1 ea. Owner's Manual, 1 ea. Side Rail Sheet, 1 ea. Carton, 1 ea. Corner Bracket Kit, and 1 ea. Mattress Keeper Kit
1130183	Label, Entrapment Warning
1142032	Paint, Touch-Up Spray Can, Taupe
1161900	Paint, Touch Up, Spray Can, White
1134871	Owner's Manual, ICCG Bed 820DLX & SC900DLX


Frame, Wire (Grid) Deck 820DLX, 820-3MDLX & SC900DLX

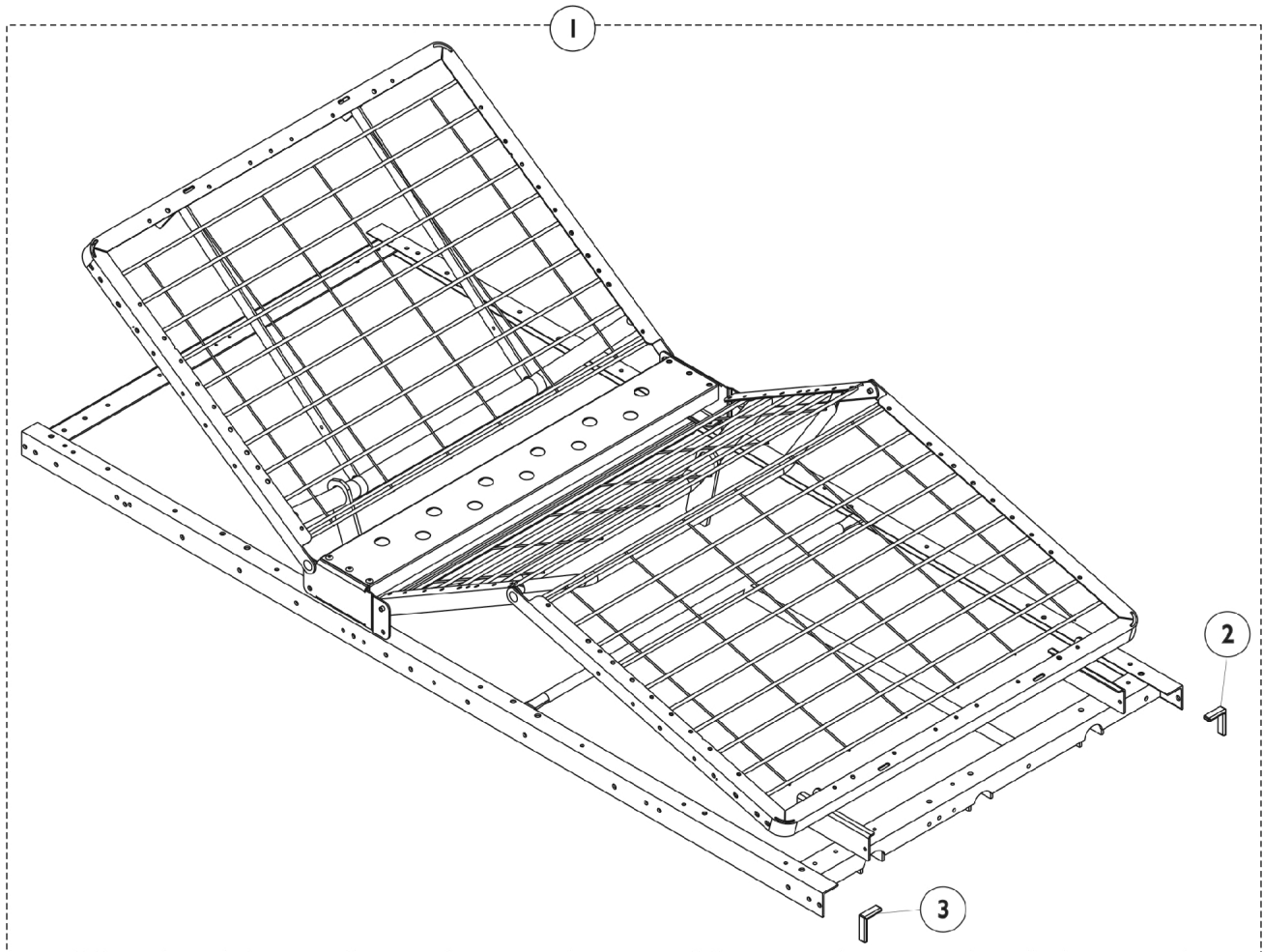

Frame, Wire (Grid) Deck 820DLX, 820-3MDLX & SC900DLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1137371	Frame and Wire Deck w/ Seat Pan	1	1
2		1125015	Cap, Angle - Right (2 x 1-1/2 x 1/8")	1	1
3		1125002	Cap, Angle - Left (2 x 1-1/2 x 1/8")	1	1
NOTE: A - Includes: Items 2-3					


Frame, Pan Deck - 820DLX, 820-3MDLX & SC900DLX

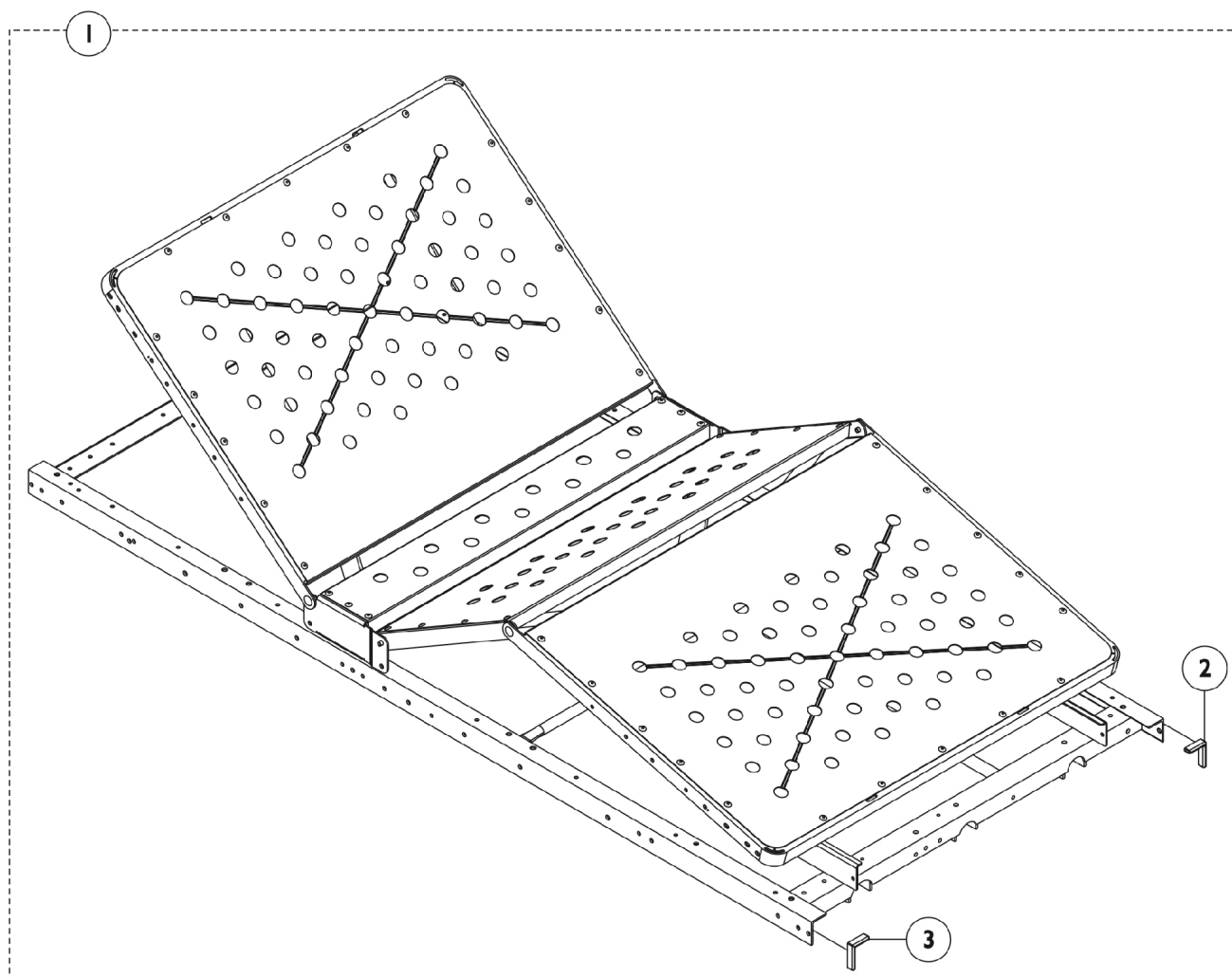

Frame, Pan Deck - 820DLX, 820-3MDLX & SC900DLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1140805	Frame and Pan Deck Assembly	1	1
2		1125015	Cap, Angle - Right (2 x 1-1/2 x 1/8")	1	1
3		1125002	Cap, Angle - Left (2 x 1-1/2 x 1/8")	1	1
NOTE: A - Includes: Items 2-3.					


Carriage Assembly - 820DLX & 820-3MDLX

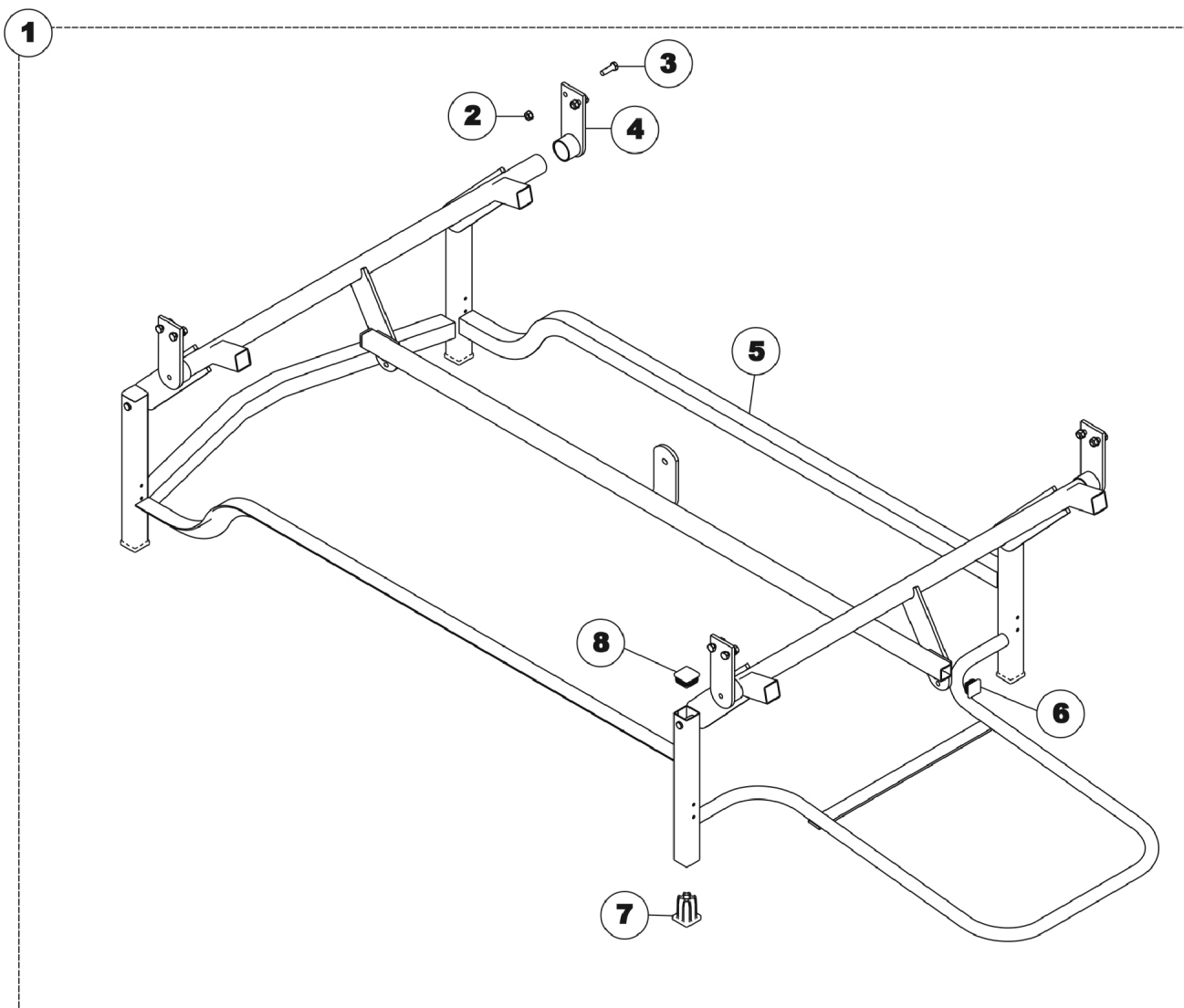

Carriage Assembly - 820DLX & 820-3MDLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1140821	Carriage, Floor Lock Reverse Assembly	1	1
2		000037-NLA	NLA - Nut, Keps (5/16-18)	1	8
3		001758	Screw, Hex Head (5/16-18 x 1")	1	8
4		025187	Support, Lift Crank	1	1
5		1140820	Frame, Carriage Lock Floor Reverse	1	1
6		020717	Glide, Cap	1	2
7		1131077	Foot, Rubber	1	4
8		1012756	Plug Button, Black (1-1/4")	1	4
	B,C	009077	Kit, Caster	1	1
	B,D	51000M083	Socket, Caster	1	4
	B,D	1123551	Caster Assembly, Swivel w/ Brake (3")	1	2
	B,D	1123549	Caster Assembly, Swivel w/o Brake (3")	1	2
NOTE: A - Includes: Items 1-8. B - Not Shown C - Includes: 4 Sockets, 2 Locking Casters and 2 Non-Locking Casters. D - Included in part # 009077.					


Carriage Assembly - SC900DLX

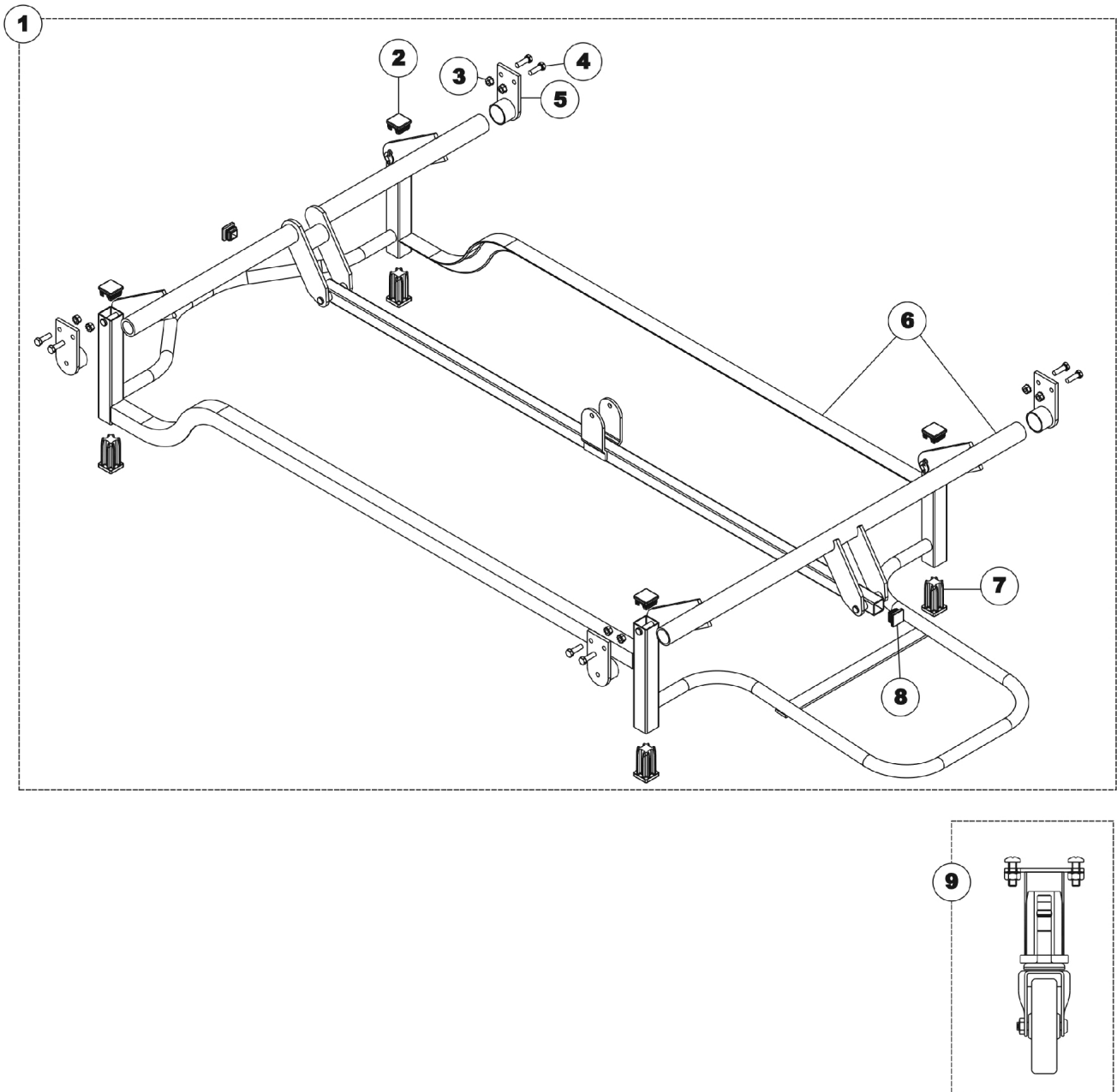

Carriage Assembly - SC900DLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1137378	Carriage Assembly - SC900DLX	1	1
2		027047	Plug Button with Hole (1-1/4" Square)	1	4
3		000037-NLA	NLA - Nut, Keps (5/16-18)	1	8
4		001758	Screw, Hex Head (5/16-18 x 1")	1	8
5		1133050	Support, Lift Crank	1	2
6		1137377	Frame, Carriage - SC900DLX	1	1
7		1131077	Foot, Rubber	1	4
8		020717	Glide, Cap	1	2
9	B	027027	Kit, Caster 3" Swivel w/ Bracket	1	1
	C	000050	Nut, Keps (1/4-20)	1	8
	C	51000M789	Screw, Phillips Pan Head (1/4-20 x 3/4")	1	8
	C	027025	Bracket, Caster	1	4
	C	51000M083	Socket, Caster	1	4
	C	1123551	Caster Assembly, Swivel w/ Brake (3")	1	2
	C	1123549	Caster Assembly, Swivel w/o Brake (3")	1	2
NOTE: A - Includes: Items 2-8. B - Includes: 4 Brackets, 4 Sockets, 8 Screws, 8 Nuts, 2 Locking Casters and 2 Non-Locking Casters. C - Included in part number 027027 - Caster Kit					


Manual Pull Tube & Crank Handle - 820DLX

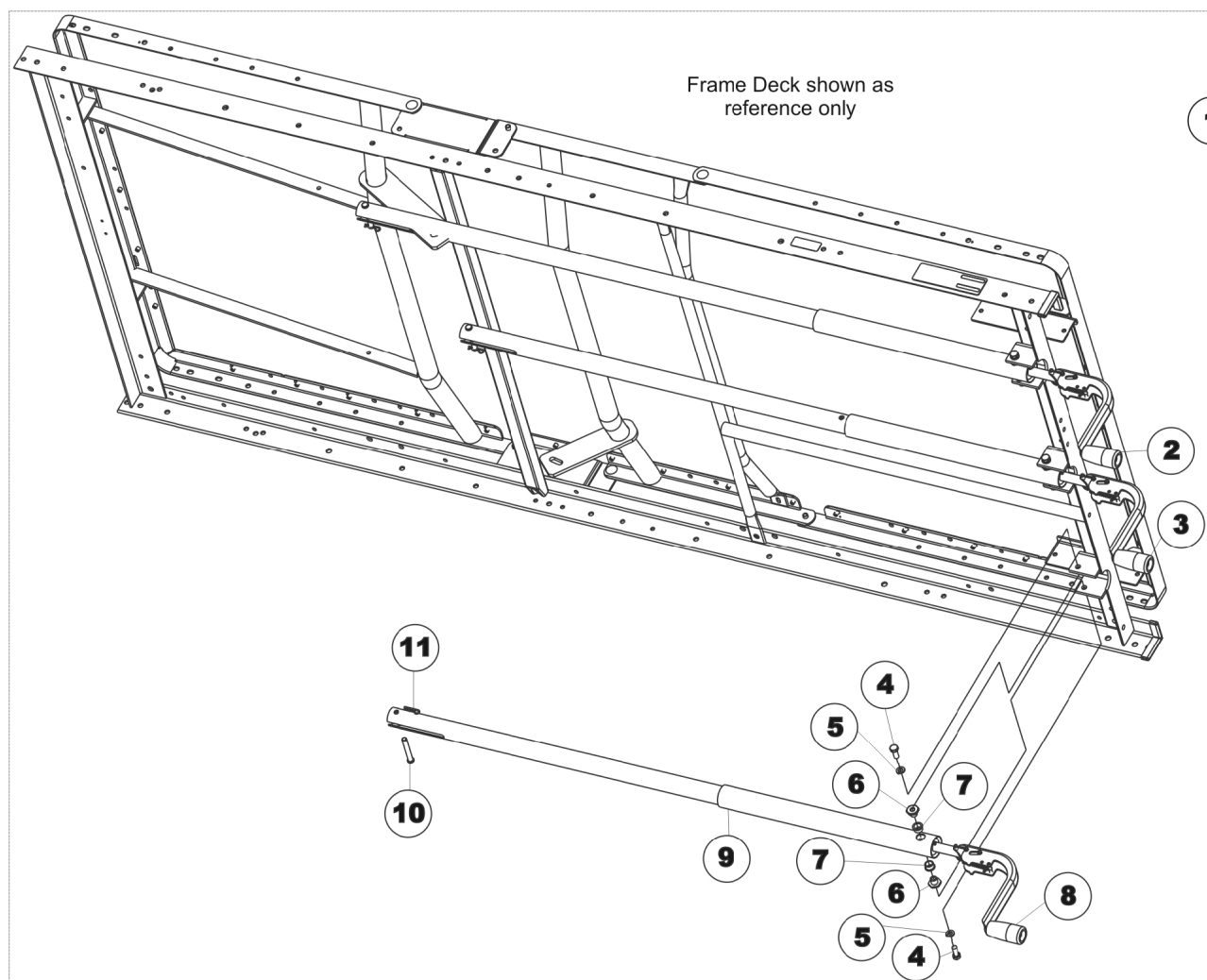

Manual Pull Tube & Crank Handle - 820DLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1140811	Kit, Crank and Pull Tubes (Includes: Foot, Head, and Hi-Lo)	1	1
2		1141407	Manual, Head Crank and Pull Tube Assembly	1	1
3		1161917	Manual, Hi-Lo Crank and Pull Tube Assembly	1	1
4		001741	Screw, Hex Head (5/16-18 x 3/4")	1	6
5		001008	Washer, Split Lock (5/16")	1	6
6		001443	Bushing, Shoulder - Pivot	1	6
7		001405	Bushing, Flanged, Nylon	1	6
8		1141406	Manual, Foot Crank and Pull Tube Assembly	1	1
9		51000M191	Cover, Pull Tube (Head/Foot)	1	3
10		005952	Pin, Clevis (5/16 x 2-1/8")	1	3
11		51000M743	Pin, Hitch (3/8 x 1-3/16")	1	3
NOTE: A - Includes: Items 1-11.					


Motors and Components, Full Electric - 820-3MDLX & SC900DLX

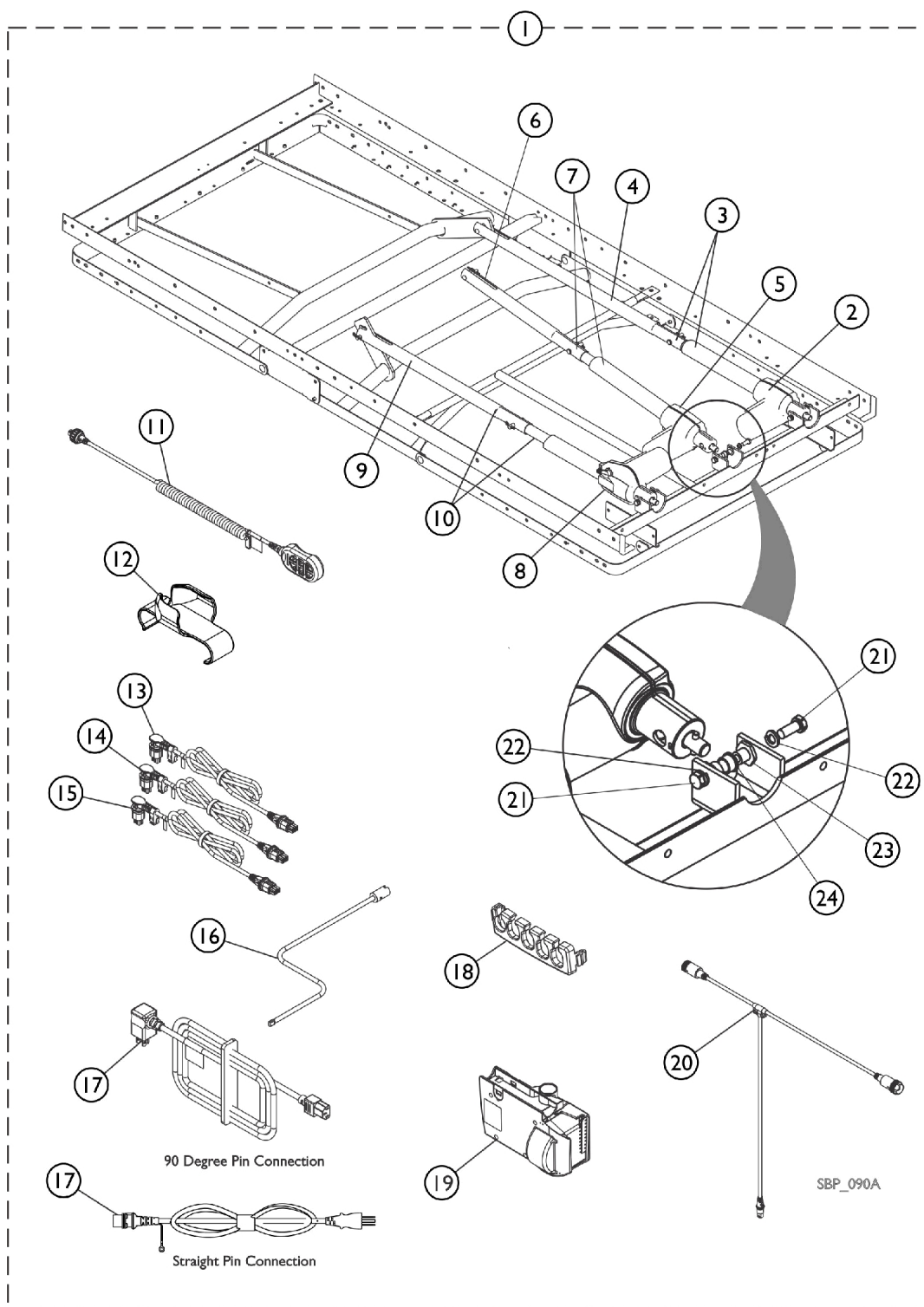

Motors and Components, Full Electric - 820-3MDLX & SC900DLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1140814	Motor & Component Assembly - 820-3MDLX	1	1
1	B	1140803	Motor & Component Assembly - SC900DLX	1	1
2		1115291	Head Actuator - (Linak #LA27-0006-04)	1	1
3		1137384	Motor/Gear Assembly, Head	1	1
4		1137343	Head Actuator Pull Tube	1	1
5		1137338	HiLo Actuator - 820-3MDLX (Linak #LA27-U026)	1	1
5		1137337	HiLo Actuator - SC900DLX (Linak #LA27-U025-00)	1	1
6		1137341	HiLo Actuator Pull Tube - 820-3MDLX	1	1
6		1137381	HiLo Actuator Pull Tube - SC900DLX	1	1
7		1140806	Motor/Gear Assembly, Hi-Lo - 820-3MDLX	1	1
7		1137383	Motor/Gear Assembly, Hi-Lo - SC900DLX	1	1
8		1115292	Foot Actuator - (Linak #LA27-0007-03)	1	1
9		1137342	Foot Actuator Pull Tube	1	1
10		1137385	Motor/Gear Assembly, Foot	1	1
11		1180040	Pendant, 3 Function	1	1
12		1133841	Holster, Pendant	1	1
13		1116640	Cable, Hi-Lo Motor w/ Red O-Ring (Head End)	1	1
14		1116641	Cable, Head Actuator w/ Blue O-Ring	1	1
15		1116642	Cable, Foot Actuator w/ Yellow O-Ring	1	1
16		1133062	Crank, Emergency	1	1
17		1132002	Power Cord, 10-1/2' (Hospital Grade) with Angled Pin Connectors	1	1
17		1144786	Power Cord, 17' with Straight Pin Connectors	1	1
18		1116655	Lock, Cable	1	1
19		1117178	Controller/Junction Box, 3 Function	1	1
20		1140812	Cable, "T"	1	1
21		001741	Screw, Hex Head (5/16-18 x 3/4")	1	2
22		001008	Washer, Split Lock (5/16")	1	2
23		001443	Bushing, Shoulder - Pivot	1	1
24		001405	Bushing, Flanged, Nylon	1	1
NOTE: A - Includes items 2-5, 6, 7, 8-24 B - Includes items 2-4, 5, 6, 7, 8-24					


Model IH5748A - 8 Bar Nylasar

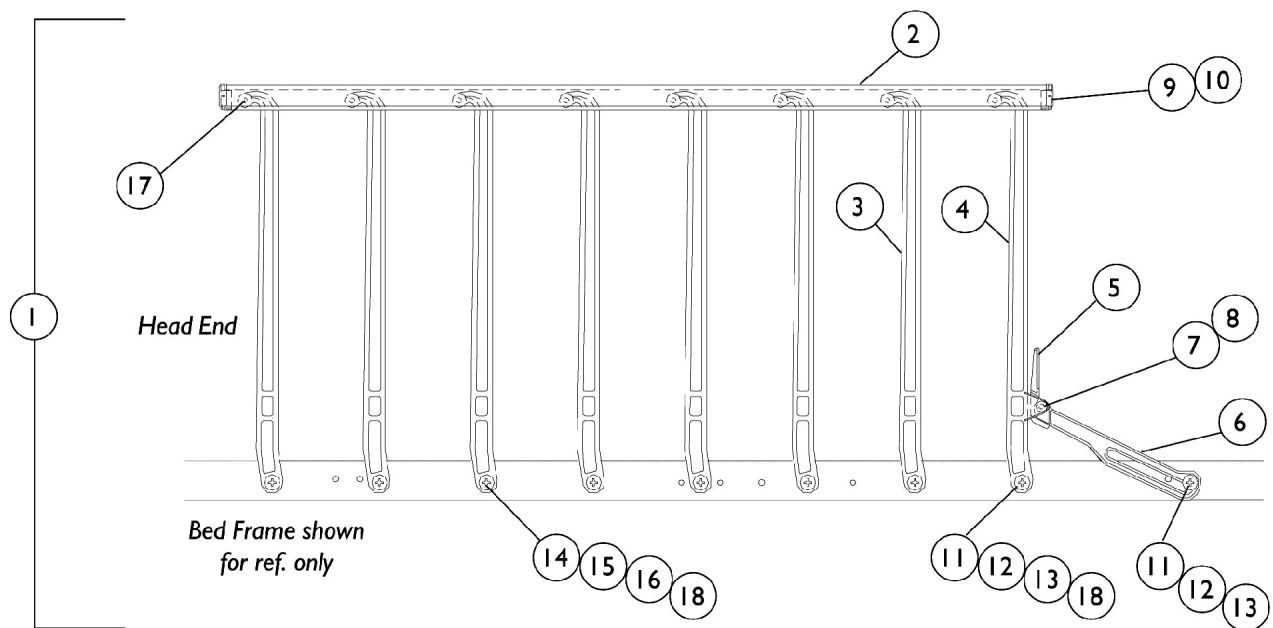

Model IH5748A - 8 Bar Nylasar

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1132013	Safety Side Rail Assembly - Right	1	1
1	A	1132014	Assembly, Safety Side Rail - Left	1	1
2		1142008	Rail, Cover and Guard - 8 Rail	1	1
3		000390	Arm, Pivot	1	7
4		000389	Arm, Latch	1	1
5		000388	Latch, Restraining Side Actuator	1	1
6		000494	Arm, Restraining Side Latch	1	1
7		000391	Pin, Latch	1	1
8		000173	Ring, Retaining (1/4")	1	1
9	C	000386	Cap, Guard Rail	1	2
10	C	009289	Adhesive	1	1
11		000397	Screw, Shoulder Hex Socket Head (3/8 x 1-1/2", 5/16-18)	1	9
12		53000M434	Locknut (5/16-18)	1	9
13		000387	Spacer (3/8 x 3/4 x 1/64")	1	9
14		000394	Bearing, Nylon (3/8")	1	16
15		000395	Washer, Steel (7/16 x 13/16 x 1/16")	1	8
16		000393	Washer, Rubber (7/16 x 27/32 x 1/8")	1	8
17		033466	Pin, Spring (1/4 x 13/16")	1	8
18		031195	Spacer (5/16 x 3/4 x 5/16")	1	8
	B	026626	Hardware, Tamper Resistant - Includes: (1) Bit, (18) Nuts, and (18) Spacers.	1	1
	B	1143214	Instruction Sheet, Bed Rail and ThinkSoft® Positioning Device	1	1
	B	1130183	Label, Entrapment Warning	1	1
NOTE: A - Includes: Items 2-18. B - Not Shown. C - The Adhesive is No Longer Available as a service part. Please use your own adhesive when installing the Guard Rail Cap.					


Model IH6801A - Full Length Epoxy

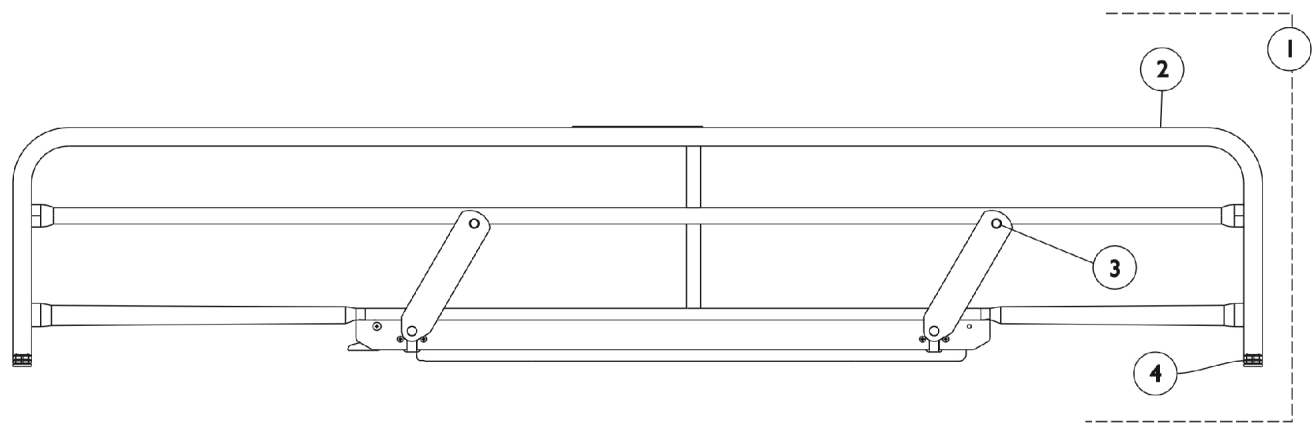

Model IH6801A - Full Length Epoxy

Item Number	Note	Part Number	Description	Quantity Per					
				Package	Assembly				
1	A	1143713	Full Length Side Rail Assembly	1	1				
2		1143693	Side Rail, Full Length - Right	1	1				
2		1143694	Side Rail, Full Length - Left	1	1				
3		033786	Screw, Socket Head (#10-32 x 3/8")	1	4				
4		1029501	Plug Button, Black (1")	1	4				
	B	020258	Wrench, Allen (5/32")	1	1				
	B	1143214	Instruction Sheet, Bed Rail and ThinkSoft® Positioning Device	1	1				
	B	1143699	Hardware, Rail Mounting Includes: (1) Crossbar Pivot (2) Pivot Bracket (8) Locknut (5/16-18) (8) Screw, Hex Head (5/16-18 x 1")	1	1				
NOTE: A - Includes: Items 2-4, all Attaching Hardware, and Instruction Sheet. B - Not Shown.									


Model IH6808A - 3/4 Length Epoxy Rail

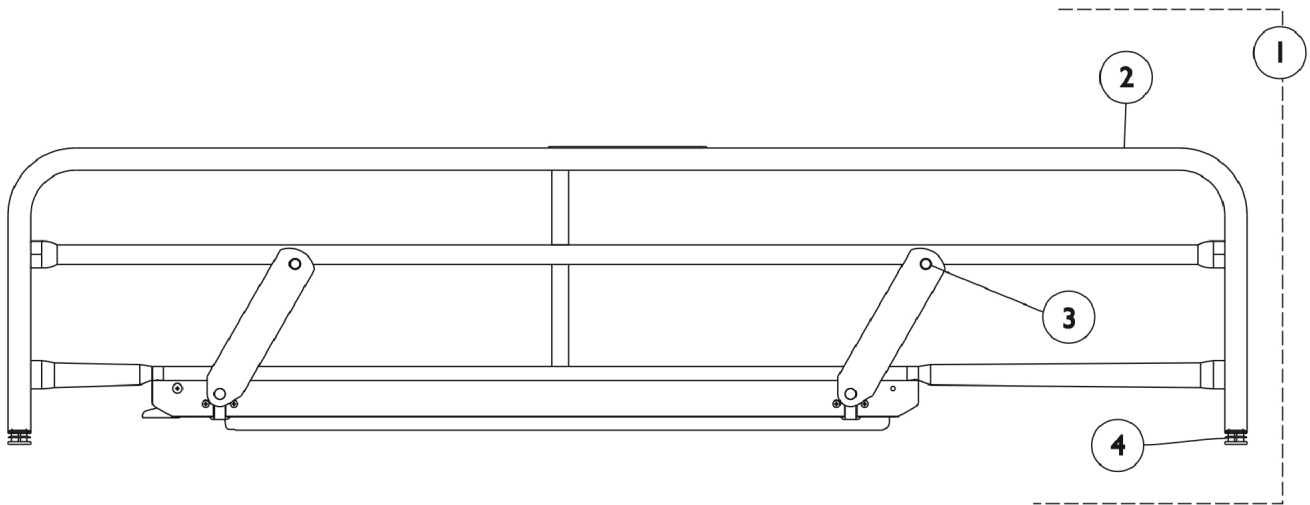

Model IH6808A - 3/4 Length Epoxy Rail

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1143712	Side Rail Assembly, 3/4 Length	1	1
2		1143387	Side Rail, 3/4 Length - Right	1	1
2		1143388	Side Rail, 3/4 Length - Left	1	1
3		033786	Screw, Socket Head (#10-32 x 3/8")	1	4
4	B	1029501	Plug Button, Black (1")	1	4
		020258	Wrench, Allen (5/32")	1	1
		1143214	Instruction Sheet, Bed Rail and ThinkSoft® Positioning Device	1	1
		1143699	Hardware, Rail Mounting	1	1
			Includes: (1) Crossbar Pivot (2) Pivot Bracket (8) Locknut (5/16-18) (8) Screw, Hex Head (5/16-18 x 1")		
NOTE: A - Includes: Items 2, Instruction Sheet, and Attaching Hardware. B - Not Shown.					


Model IH6815A - 1/2 Length Epoxy Rail, Head End

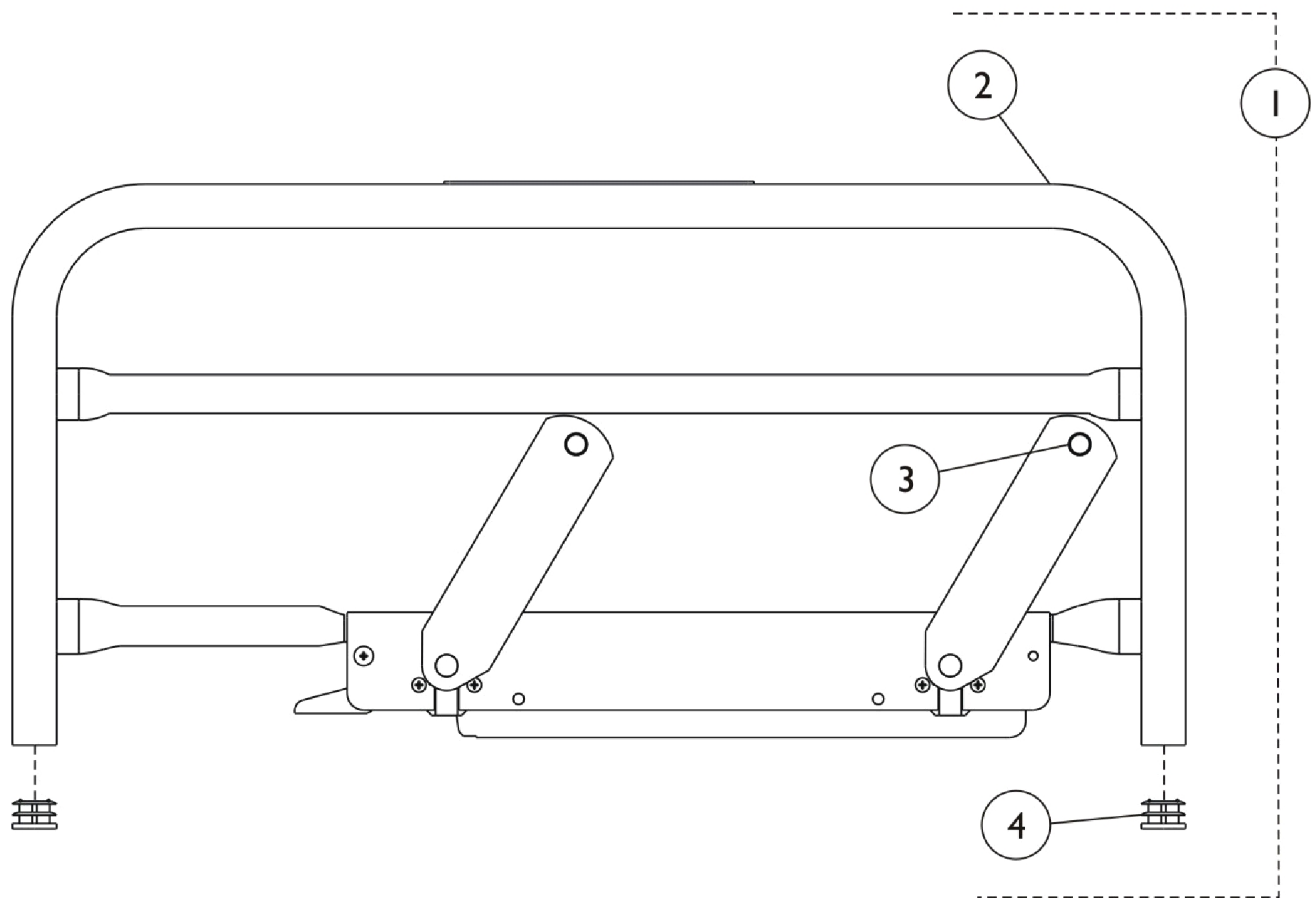

Model IH6815A - 1/2 Length Epoxy Rail, Head End

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1143708	Side Rail Assembly, 1/2 Length	1	1
2		1143362	Side Rail, 1/2 Length - Right	1	1
2		1143363	Side Rail, 1/2 Length - Left	1	1
3		033786	Screw, Socket Head (#10-32 x 3/8")	1	4
4	B	1029501	Plug Button, Black (1")	1	4
		020258	Wrench, Allen (5/32")	1	1
		1143214	Instruction Sheet, Bed Rail and ThinkSoft® Positioning Device	1	1
		1143698	Hardware, Rail Mounting Includes: (1) Crossbar (2) Pivot Bracket (4) Locknut (1/4-20) (4) Screw, Hex Head (1/4-20 x 5/8")	1	1
NOTE: A - Includes: Items 2, Instruction Sheet, and Attaching Hardware. B - Not Shown.					


Model IH6816A - 1/2 Length Epoxy Rail, Foot End

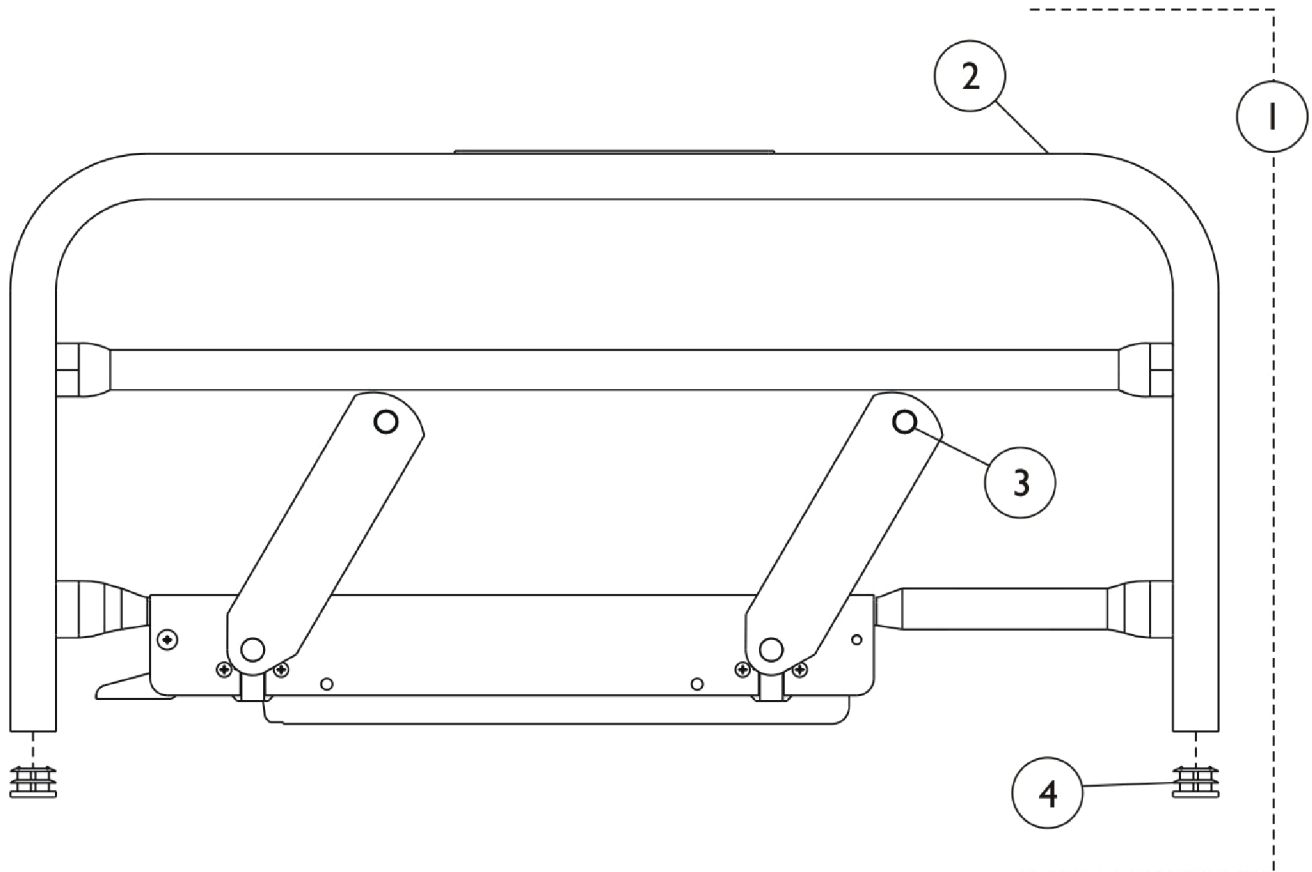

Model IH6816A - 1/2 Length Epoxy Rail, Foot End

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1143710	Side Rail Assembly, 1/2 Length	1	1
2		1143395	Side Rail, 1/2 Length - Right	1	1
2		1143396	Side Rail, 1/2 Length - Left	1	1
3		033786	Screw, Socket Head (#10-32 x 3/8")	1	4
4	B	1029501	Plug Button, Black (1")	1	4
		020258	Wrench, Allen (5/32")	1	1
		1143214	Instruction Sheet, Bed Rail and ThinkSoft® Positioning Device	1	1
		1143700	Hardware, Rail Mounting Includes: (2) Pivot Bracket (4) Locknut (1/4-20) (4) Screw, Hex Head (1/4-20 x 5/8")	1	1
NOTE: A - Includes: Items 2, Instruction Sheet, and Attaching Hardware. B - Not Shown					


Model IH6803A - 1/4 Length Epoxy Rail

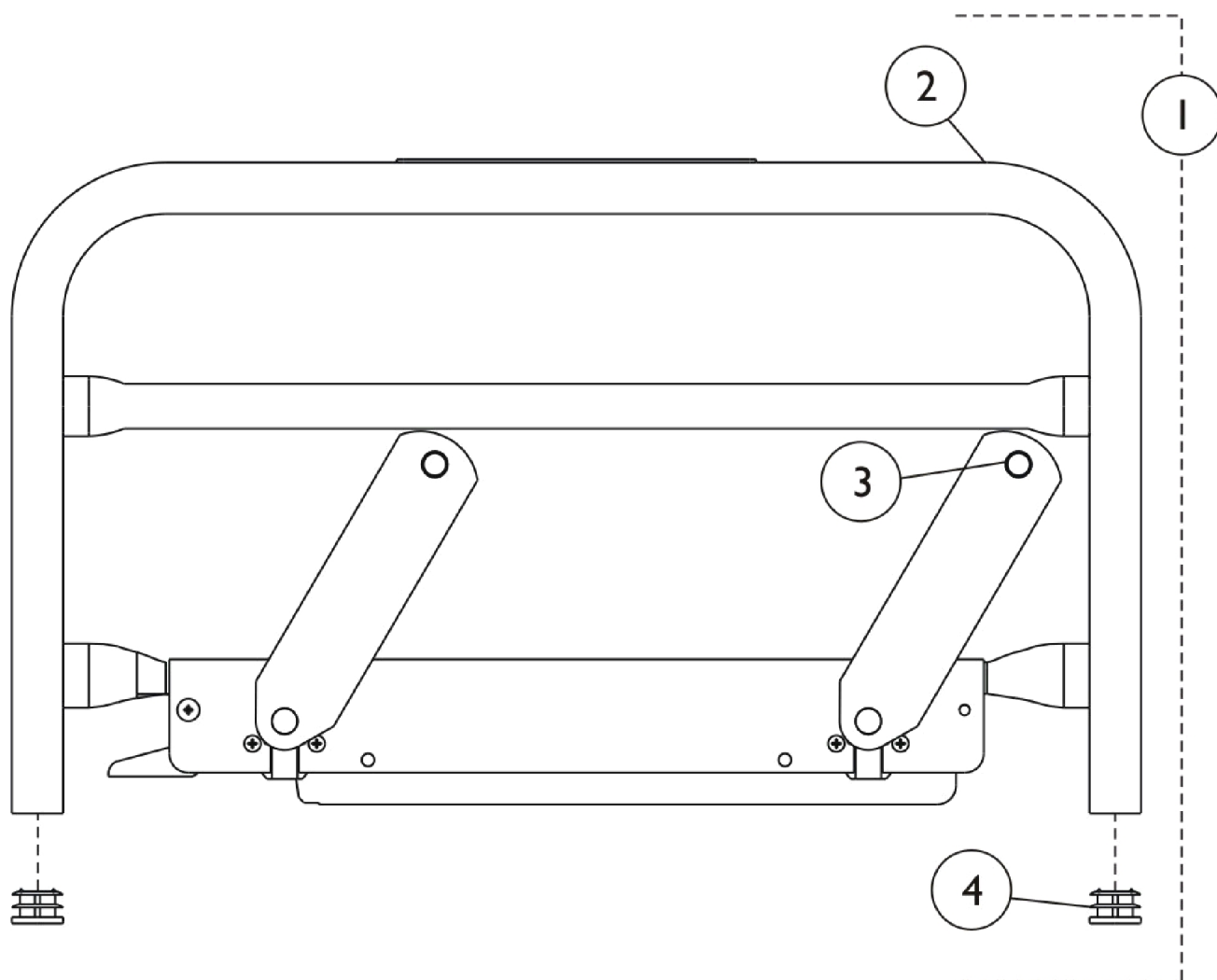

Model IH6803A - 1/4 Length Epoxy Rail

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1143709	Side Rail Assembly, 1/4 Length	1	1
2		1143360	Side Rail, 1/4 Length - Right	1	1
2		1143361	Side Rail, 1/4 Length - Left	1	1
3		033786	Screw, Socket Head (#10-32 x 3/8")	1	4
4	B	1029501	Plug Button, Black (1")	1	4
		020258	Wrench, Allen (5/32")	1	1
		1143214	Instruction Sheet, Bed Rail and ThinkSoft® Positioning Device	1	1
		1143698	Hardware, Rail Mounting	1	1
			Includes: (1) Crossbar (2) Pivot Bracket (4) Locknut (1/4-20) (4) Screw, Hex Head (1/4-20 x 5/8")		
NOTE: A - Includes: Items 2, Instruction Sheet, and Attaching Hardware. B - Not Shown.					

Bed Rails - IHRAILTS-DLX ThinkSoft® Positioning Device

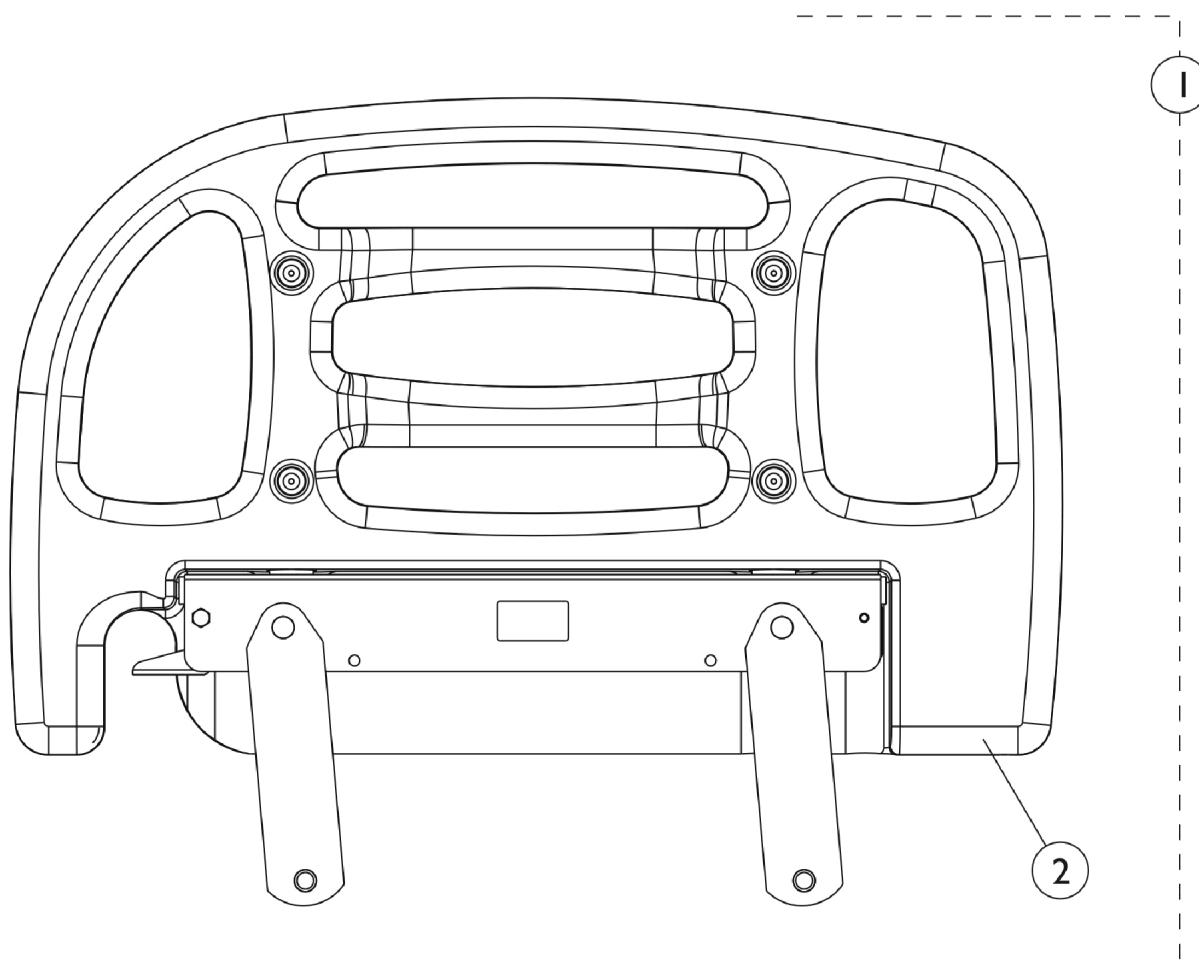


Bed Rails - IHRAILTS-DLX ThinkSoft® Positioning Device

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1159105	ThinkSoft® Assembly Packaging	1	1
2		1157357	ThinkSoft® Positioning Device - Right	1	1
2		1157358	ThinkSoft® Positioning Device - Left	1	1
3		1157350	Kit, Pin Retaining DLX	1	1
			Bag, Poly (5" x 7")	1	1
			Pin, Retaining	4	1
			Washer, Nylon (1/2 x 7/8 x 1/16")	4	1
	B	1143214	Instruction Sheet, Bed Rail and ThinkSoft® Positioning Device	1	1
	B	1143698	Hardware, Rail Mounting Includes: (1) Crossbar (2) Pivot Bracket (4) Locknut (1/4-20) (4) Screw, Hex Head (1/4-20 x 5/8")	1	1
NOTE: A - Includes Item 2 and 3, also comes with Instruction Sheet and Rail Mounting Hardware. B - Not Shown.					

Assist Rails

1

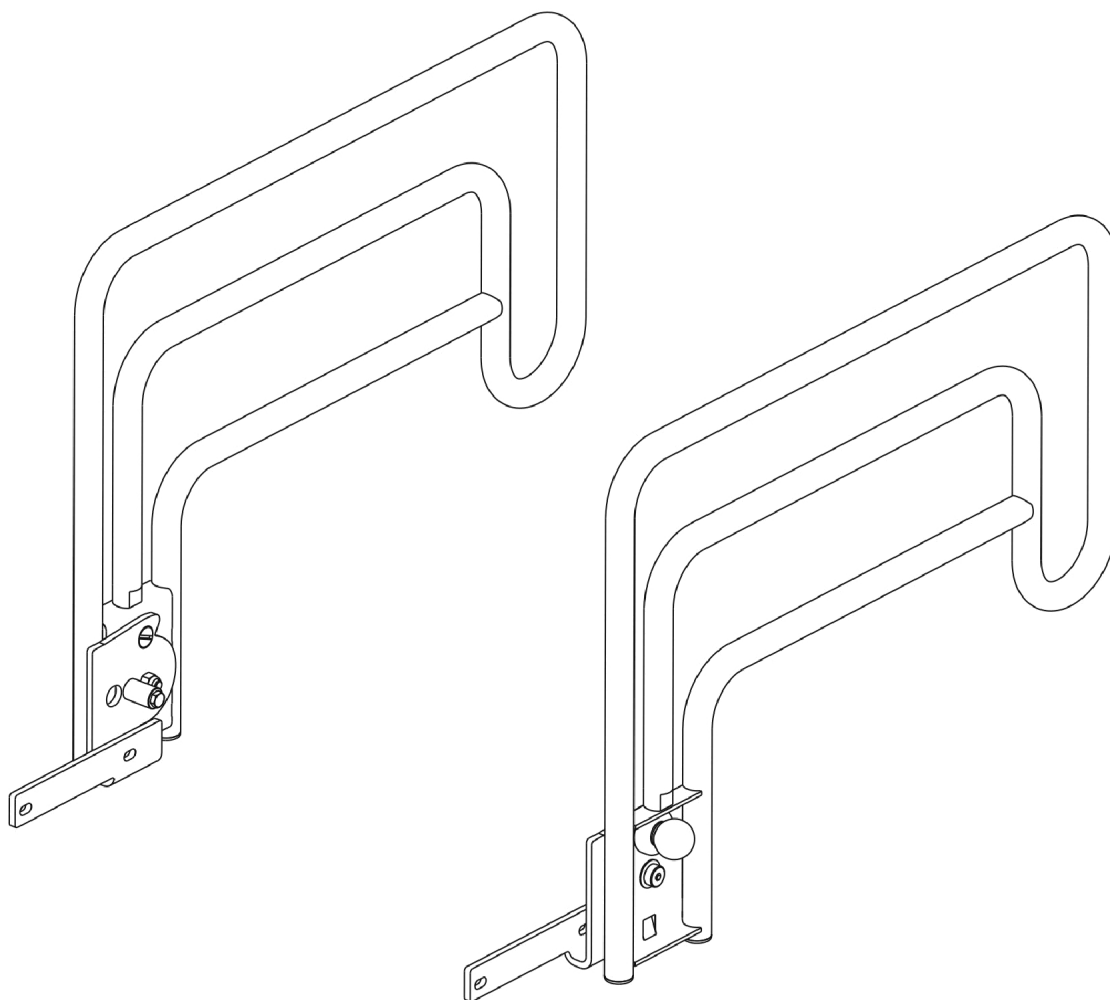

Assist Rails

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	IHRAILAE-DLX-QS	Rail, Assist with Hardware	1	2
		IHRAILAB-DLX-QS	Bar, Assist	1	2
NOTE: A - Not Shown.					

Miscellaneous Accessories

Bumper and I.V. Holder

Part Number	Description	Package Quantity	Assembly Quantity
003790	Holder, I.V. Rod and Bumper	1	2
000834	Screw, Pan Head (#10 x 3/4")	1	4

I.V. Bracket

Part Number	Description	Package Quantity	Assembly Quantity
026394	Bracket, I.V. Bolt On - Left	1	1
026395	Bracket, I.V. Bolt On - Right	1	1
001743	Screw, Slotted Pan Head (1/4-20 x 3/4")	1	8
003005	Nut, Hex (1/4-20)	1	8
004033	Washer, Split Lock (1/4")	1	8
1070795	Instruction Sheet, I.V. Bracket	1	1

Bed Extender and Mattress

Part Number	Description	Package Quantity	Assembly Quantity
032906	Bed Extender (Dark Brown)	1	1
1133065	Bed Extender (Taupe)	1	1
032981	Mattress (13-3/8 x 32 x 2")	1	1
000194	Grommet, Pan Deck	1	8
1070793	Instruction Sheet, Bed Extender and Mattress	1	1

Miscellaneous Accessories

Corner Bracket Kit

Usage Date	Kit Part Number	Kit Includes:
Before 1/1/2009	1124029-NLA NLA - Kit, Corner Bracket, Taupe	(2) Left Corner Brackets (2) Right Corner Brackets (8) Screws (8) Nuts
After 1/1/2009	1161904 Kit, Corner Bracket, White	(2) Left Corner Brackets (2) Right Corner Brackets (8) Screws (8) Nuts



One Invacare Way
Elyria, Oh USA
44035
800-333-6900

Technical Support
800-668-2337
Customer Service
800-668-2337

Service Availability
Monday - Friday
Hours of Availability
8 AM - 5:30 PM

© 2018 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. (Revised 3/27/18)