



Panacea®

3-POSITION RECLINER

Standard and Extra-Wide

Owner's Manual

Please keep and refer to this Owner's Manual.

Thank you for purchasing a Panacea® 3-Position Recliner from Direct Supply Equipment & Furnishings®. Please read this entire guide carefully and keep it for future reference. This guide will provide you with instructions, warnings, warranty information and other important information about your recliner. Share this information with your housekeeping, nursing and maintenance staff to help ensure the recliner is cared for properly.

Assembly & Operating Instructions

Assembly Instructions

NOTE: Unpack the recliner in an area with sufficient room to work. Do not allow residents near the recliner until it has been completely set up and the work area has been cleared of all debris.

Caster Installation

1. Place the lower frame assembly (Figure 2. B) on its side or upside down to install the casters.
2. Insert the caster stem into the bottom square tubes of the lower frame assembly. Align the holes in the caster stem and tubing.

NOTE: Insert the swivel locking casters on the rear frame tubes.

3. Secure the caster in place with a bolt, washer, and nut using the provided tools (Figure 1).



Figure 1

Back Installation

1. Slide the back assembly (C) into the square tubes of the lower frame assembly (B) so the holes in the backrest line up with the holes located on the horizontal brace of the lower frame.
2. Raise recliner Lock Bar (F), adjust position of Back Assembly (C) until Lock Release Button (G) positively engages in the middle slot of the lock bar when chair is in storage position.
3. Insert screws through holes of the horizontal brace (B) and tighten.

NOTE: Make sure that the top screws on both sides of the back assembly are secure before using recliner.

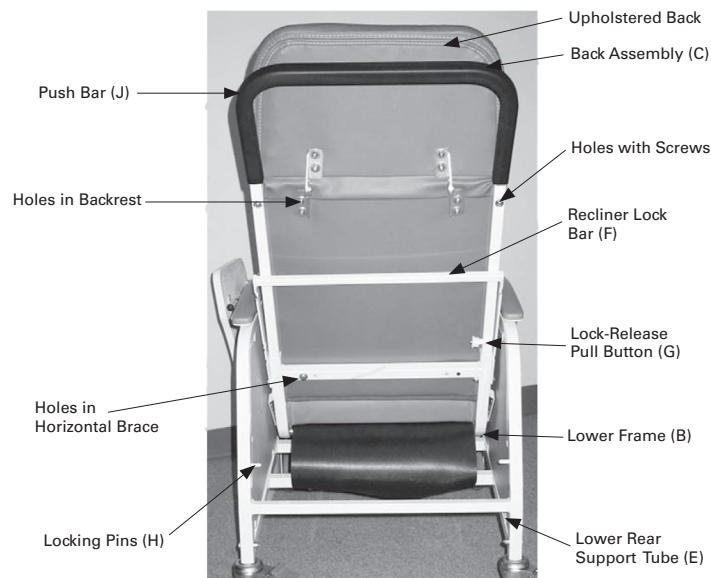


Figure 2

Operating Instructions

To Place Chair in Reclining Position with Recliner Lock Bar in Storage Position

By Occupant:

Place hands on armrests and push against seat back. Legrest will automatically adjust to each recline position.

NOTE: The lap tray should only be used in upright sitting position.

By Caregiver:

1. Ensure that the recliner Lock Bar is in the storage position (F). See figure 1.
2. Stand at either side of the recliner.
3. Grasp armrest with one hand, and PVC push bar (J) with the other hand.
4. Push down on push bar (J) to recline the back to the first recline position. To achieve second recliner position, push on push bar again.
5. To raise the backrest to upright, pull forward on push bar.

Reclining Lock Operation

To Lock Recliner Position

1. Place recliner into the desired position.
2. Pull out Lock-Release Pull Button (G) to release Lock Bar (F) from the storage position.
3. Rotate recliner Lock Bar (F) down until notches are fully engaged over Locking Pins (H).

To Unlock Recliner:

1. Lift recliner Lock Bar (F) up to disengage from Locking Pins (H).
2. While holding Lock-Release Pull Button (G) out, rotate recline Lock Bar (F) all the way up until completely against seat back.
3. Release Lock-Release Pull Button (G) to hold recliner Lock Bar in the storage position (F).

NOTE: Do not attempt to change position of chair with recliner Lock Bar engaged on the Locking Pins. Doing so may damage to the mechanism.

Operating Instructions *(cont.)*

Maintenance & Cleaning

Operating Instructions

Articulating headrest may be adjusted to desired position by pulling forward or pushing back.

Operating Instructions for Lap Tray


NOTE: Lap tray included with Standard Recliner ONLY.

To Operate

1. Underneath each arm, pull the tray pins down (Figure 3 A & B) and twist 90° so the pins are in the open position.
2. Hold the lap tray by the circular support bars and slide into receiving tubes in front of each arm.
3. Slide the table in to the desired position. Align the holes on the lap tray support bars with the Tray Pins (A & B).
4. Twist the left and right tray pins 90° to release up. The tray is now locked in position.

Storage

The lap tray hangs on the dual hooks under the right-side armrest for storage.

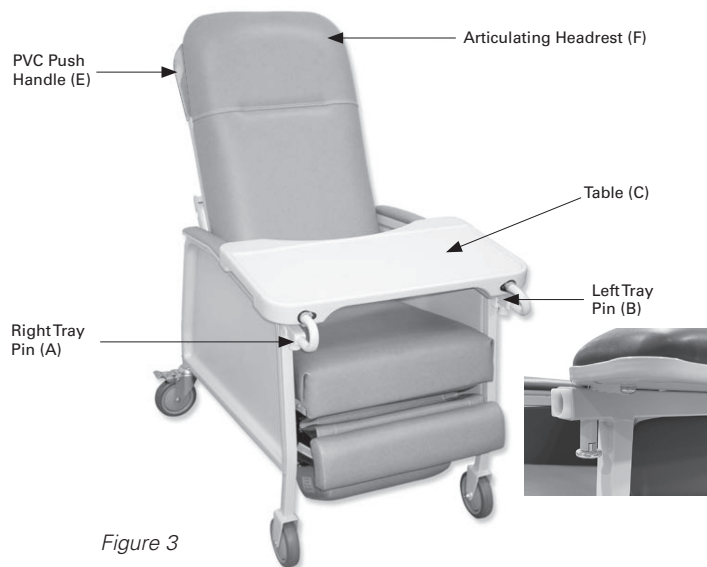

CAUTION:

Do not operate recliner by applying force or pressure to the legrest. Avoid placing hands or legs in creases near recliner mechanism.

Maintenance & Cleaning

Always ensure the product is functioning properly prior to each use.

Every Three Months:

- Check and Clean Casters
- Tighten Nuts
- Check Brakes on Casters
- Check and Clean Lock Recline Mechanism

For upholstery maintenance and stain removal, the following hints will be helpful:

Ordinary Dirt: Wash with warm water and a mild soap or detergent.

Stubborn Dirt: Using a soft bristle brush, follow direction above.

Ground-In Dirt: Scrub with a mild cleaning solution until clean.

Chewing Gum: Scrape carefully; remove with kerosene or naphtha.

Paint, Shoe Polish, Heel Marks, Etc: Remove immediately! Use a white cloth dampened in kerosene, naphtha or turpentine. Do not use paint remover or liquid brush cleaners. When using kerosene or naphtha, do so in a well ventilated area and keep away from fire and flame.

Waxing and Refinishing: Improper wearability. Use any quality paste wax.

A diluted solution of 10% (or less) bleach or Clorox may be used when cleaning this chair.

Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality recliner, and enjoy outrageous customer service. If you have any questions about the recliner you have purchased or would like to request warranty service, please contact your distributor.

Limited Warranty

We offer you, as the original purchaser of the product, a limited warranty for the structure and cushion of the product. Our warranty applies for the limited warranty period stated below. We will repair or replace any product or product part listed below if defective in material or workmanship. Please note that the decision to repair or replace a product or product part will be at our discretion.

Our warranty applies only if the product is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the product or damage caused by abuse, improper use, failure to maintain, use which exceeds the published product limitations or the combination of any product with another product. In addition, our warranty does not cover fading, characteristics or natural variations in wood grain or fabric, texture, colorfastness, stains, spills or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts, and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary, from state to state. Please note that our limited warranty period begins when we ship the product to you. The limited warranty period and our obligations under the warranty end once you transfer the product to someone else, or at the end of the applicable limited warranty period identified below, whichever is earlier.

Anticipated Usable Product Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the product as described in this guide, as the product may need to be replaced sooner than anticipated in particular situations.

	Warranty Period (parts)	Weight Capacity
Direct Supply Standard 3-Position Recliner	1 Year	250 lbs.
Direct Supply Extra Wide 3-Position Recliner	1 Year	500 lbs.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE DEVICE OR YOUR USE OF OR INABILITY TO USE THE DEVICE EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE DEVICE OR SERVICE GIVING RISE TO YOUR CLAIM.

 **WARNING:** Cancer and Reproductive Harm - www.P65Warnings.ca.gov



Distributed by Direct Supply Manufacturing, Inc. 6635 W Champions Way, Milwaukee, WI 53223.

© 2024 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply, Direct Supply Equipment & Furnishings and all associated logos are the registered trademarks of Direct Supply, Inc., used under license.