

Direct Supply® Panacea® Abductor Pillow



Thank you for purchasing a Direct Supply® Panacea® Abductor Pillow from Direct Supply Equipment & Furnishings®, a division of Direct Supply, Inc. Please read this entire manual carefully and keep it for future reference. This manual will provide you with instructions, warnings, warranty information and other important information about your product. Share this information with your housekeeping, nursing and maintenance staff to help ensure the product is cared for properly. **Please keep and refer to this Owner's Manual.**

Definitions

Direct Supply Panacea Abductor Pillow Brace ("Product") is distributed by Direct Supply Manufacturing, Inc. ("We," "Us" or "Our"). "You" are the facility, community or other entity that has purchased the device.

Safety Notices – Warnings and Cautions

Read and follow all directions and warnings before using this product. Do not use this product without first reading and understanding this user manual. Damage, injury or even death may result from improper use of this product or not following directions and warnings. This product is intended for use by or under the supervision of trained and licensed healthcare or senior care providers in normal, indoor conditions.

⚠ WARNING: Immediately discontinue use and contact appropriate healthcare personnel for further instructions if the user experiences any of the following while using the product:

- Swelling
- Pain
- Skin Redness

⚠ WARNING: Inspect the product for damage before each use and do not use if product appears to be damaged.

⚠ CAUTION: Do not place this product in an industrial washing machine or dryer.

Product Description

The Direct Supply® Panacea® Abductor Pillow (Model # DS-MSC-AP) attaches to one leg for a secure placement between the knees. It is covered in a fabric that is comfortable and may wick away perspiration. The cover can easily be removed and laundered in mild detergent and warm water. The cover should be air dried before replacing it on the pillow.



Width: 8"

Fits circumference: 15"-23"

Directions for Use

To apply the product: Wrap the straps around one of the resident's legs at the desired position and connect the hook-and-loop closures. Adjust the pillow so it rests between the resident's legs. Ensure that the strap is snug but not tight and that the resident is comfortable.

To launder: The cover can be removed and laundered in a mild detergent and warm water. The cover should be air dried before replacing it on the pillow.

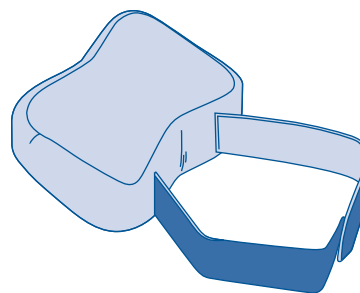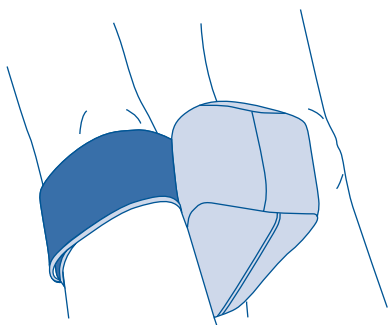

RESIDENT'S NAME: _____

ROOM #: _____

DIRECT SUPPLY PANACEA ABDUCTOR PILLOW SHOULD BE WORN AT TIMES SPECIFIED:

7 - 3 shift	3 - 11 shift	11 - 7 shift
_____	_____	_____
_____	_____	_____
_____	_____	_____

Limited Warranty

We, Direct Supply Manufacturing, Inc. offer to you, as the original purchaser, a warranty for the Direct Supply Panacea Abductor Pillow. Our warranty applies for the limited warranty period stated below. If any product or product part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a product or product part will be at our discretion.

Our warranty applies only if the product is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the product, or damage caused by abuse, improper use, failure to maintain, use which exceeds the published product limitations, or the combination of any device with another device. In addition, our warranty does not cover fading, colorfastness, stains, spills, or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts, and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary, from state to state. Please note that our limited warranty period begins when we ship the device to you. The limited warranty period and our obligations under the warranty end once you transfer the product to someone else, or at the end of the applicable limited warranty period identified below, whichever is earlier.

Product	Warranty Period (Parts)	Anticipated Usable Product Life
Direct Supply® Panacea® Abductor Pillow	90 days	90 days

Anticipated Usable Product Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the product as described in this guide, as the product may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE PRODUCT OR YOUR USE OF OR INABILITY TO USE THE PRODUCT EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE PRODUCT OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.

Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality Direct Supply Panacea Abductor Pillow and enjoy outrageous customer service. If you have any questions about the Abductor Pillow you have purchased or would like to request warranty service, please contact: **Direct Supply Equipment & Furnishings** at 1-800-634-7328, 6767 North Industrial Road, Milwaukee, WI 53223, DearDirect@DirectSupply.net.