



Panacea®

EXERCISE SUPPLIES

Owner's Manual

Thank you for purchasing Panacea Exercise Supplies from Direct Supply Equipment & Furnishings®, distributed by Direct Supply Manufacturing, Inc. Please read this entire guide carefully and keep it for future reference. This guide will provide you with instructions, warnings, warranty information and other important information about your Panacea Exercise Supplies. Share this information with your housekeeping, nursing, therapy and maintenance staff to help ensure the products are cared for properly. Should you have any questions, please contact your Direct Supply Equipment & Furnishings account manager.

Definitions

"We," "Us" or "Our": Direct Supply Manufacturing, Inc.

"You": The facility, community or other entity that has purchased the product.

Product: Panacea Exercise Supplies

⚠ WARNING: a potential danger requires correct procedures or practices in order to prevent personal injury or death.

⚠ CAUTION: correct operating or maintenance procedures must be followed in order to prevent damage to or destruction of the product or other property.

Safety Notices – Warnings

⚠ WARNINGS

Read and follow all directions and warnings before using this product. Do not use this product without first reading and understanding this user manual. Damage, injury or even death may result from improper use of this product or not following directions and warnings.

⚠ WARNING: Inspect the products for damage before each use, and do not use if products appear to be damaged.

⚠ WARNING: Exercise Supplies are not for use with individuals with reduced cognitive abilities.

⚠ WARNING: Hand Putty should not be ingested. May cause choking hazard.

⚠ WARNING: A caregiver should determine if the product may be properly used based on each resident's current health condition.

⚠ WARNING: Consult with your physical therapist, coach, trainer or physician before starting any rehabilitation or exercise program.

⚠ CAUTION: Perform exercises in a slow and controlled motion, in an open area.


⚠ CAUTION: Maintain proper posture throughout exercise session.

⚠ CAUTION: 500 lb. weight capacity—wrist weight rack.

Wrist Weight Assembly Instructions:

2-Sided Mobile Wrist Weight Storage Cart

1. Insert 4 casters (1) into the two rack legs (2), then assemble the rectangular metal basket (3) with the legs.
2. Insert peg board (5) into bottom metal frame slot (4); press it evenly to ensure all is flush to the frame slot (4).
3. Assemble the horizontal separation bar (7) on top of bottom peg board (5).
4. Insert top peg board (5) into top metal frame slot (8); press it evenly to ensure all is flush to the top frame slot (8).
5. Assemble them (5+8) with bottom metal frame (4) by inserting the two sides with slight tension.
6. Fasten two screws (6) on each side of the frame by hex key (10).
7. Hook pegs onto peg board to your desired position.

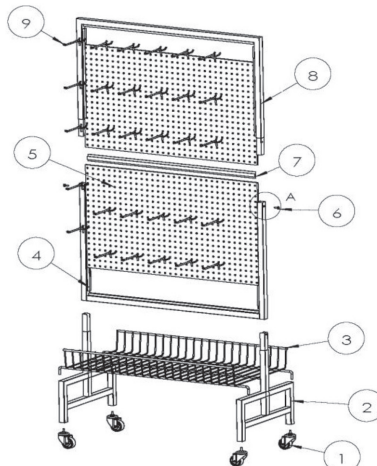

No.	Item.	Quantity
1.	Caster (Lock + No Lock)	2+2
2.	Metal Rack Leg	2
3.	Rectangular Metal Basket	1
4.	Bottom Metal Frame	1
5.	Peg Board	2
6.	5m*10 Round-Head Screw	2
7.	Horizontal Separation Bar	1
8.	Top Metal Frame	1
9.	Pegs	25
10.	M3 Hex Key	1

Upon receipt of the product, inspect for damage. If the product is damaged or the wrong product was accidentally sent, **DO NOT USE IT**, and immediately contact Direct Supply Equipment & Furnishings for further instruction. Before each use, verify the product is undamaged and properly functioning. Do not use the product if you suspect it may be functioning incorrectly. The entire product can be wiped clean with a damp cloth and soapy water.

Limited Warranty

We, Direct Supply Manufacturing, Inc., offer to you, as the original purchaser, a warranty for the Panacea Exercise Supplies you purchase. Our warranty applies for the limited warranty period stated below. If any product or product part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a product or product part will be at our discretion. Our warranty applies only if the product is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the product or damage caused by abuse, improper use, failure to maintain, use which exceeds the published product limitations or the combination of any product with another product. In addition, our warranty does not cover characteristics or natural variations in fabric, texture, colorfastness, stains, spills or exposure to excessive heat. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts and ask you to perform a repair or adjustment or replace a defective part. Our warranty gives you specific legal rights, and you may also have other rights which vary from state to state. Please note that our limited warranty period begins when we ship the product to you. The limited warranty period and our obligations under the warranty end once you transfer the product to someone else or at the end of the applicable limited warranty period identified below, whichever is earlier.

Product	Warranty Period (Parts)	Anticipated Usable Product Life
Wrist Weights	12 months	12 months
Dumbbell Weight	12 months	12 months
Exercise Band	12 months	12 months
Hand Putty	12 months	12 months
Wrist Weight Rack	12 months	12 months

Anticipated Usable Product Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the product as described in this guide, as the product may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE PRODUCT OR YOUR USE OR INABILITY TO USE THE PRODUCT, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE PRODUCT OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.

Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality product and enjoy outrageous customer service. If you have any questions about the product you have purchased or would like to request warranty service, please contact us at: **Direct Supply Equipment & Furnishings**, 1-800-634-7328, 6767 N. Industrial Road, Milwaukee, WI 53223, DearDirect@DirectSupply.net.



DirectSupply.com

Aptura / Equipment & Furnishings / Capital Projects / Technology Solutions / DSSI / TELS/Local Services

Distributed by Direct Supply Manufacturing, Inc. 6699 N. Industrial Road, Milwaukee, WI 53223.

© 2014 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply, Panacea, Direct Supply Equipment & Furnishings and all associated logos are the registered trademarks of Direct Supply, Inc., used under license.