



Panacea® OVERBED TABLE

Assembly & Operating Instructions
Ensamblado de Mesa de Cama e Instrucciones de Uso

Important Warnings


To avoid serious injury and/or death to a patient or resident, follow ALL instructions and these guidelines:

- This product is designed ONLY as patient/resident convenience as a surface for reading, eating, or keeping small personal items such as glasses within reach.
- The weight limit for this product is 50 lbs. (22 kg.); the weight should be evenly distributed over the top of the product's table.
- Overloading this product could result in the product breaking as well as damage to property and/or serious injury.
- This product should NOT be used: as a walker, to assist a person in sitting down or standing up, to prevent or brace a person's fall, or to transport persons. Use ONLY as directed.
- This product is designed for indoor use ONLY within close proximity of skilled caregivers. It is not designed to hold electrical items (such as televisions) and should be kept away from open flames and excessive heat.
- Inspect the product for damage before each use. DO NOT use the product if it appears to be damaged or unstable and DO NOT alter the product in any way.
- DO NOT overtighten the screws or use power tools when assembling this product.



Components & Parts


Componentes y Piezas

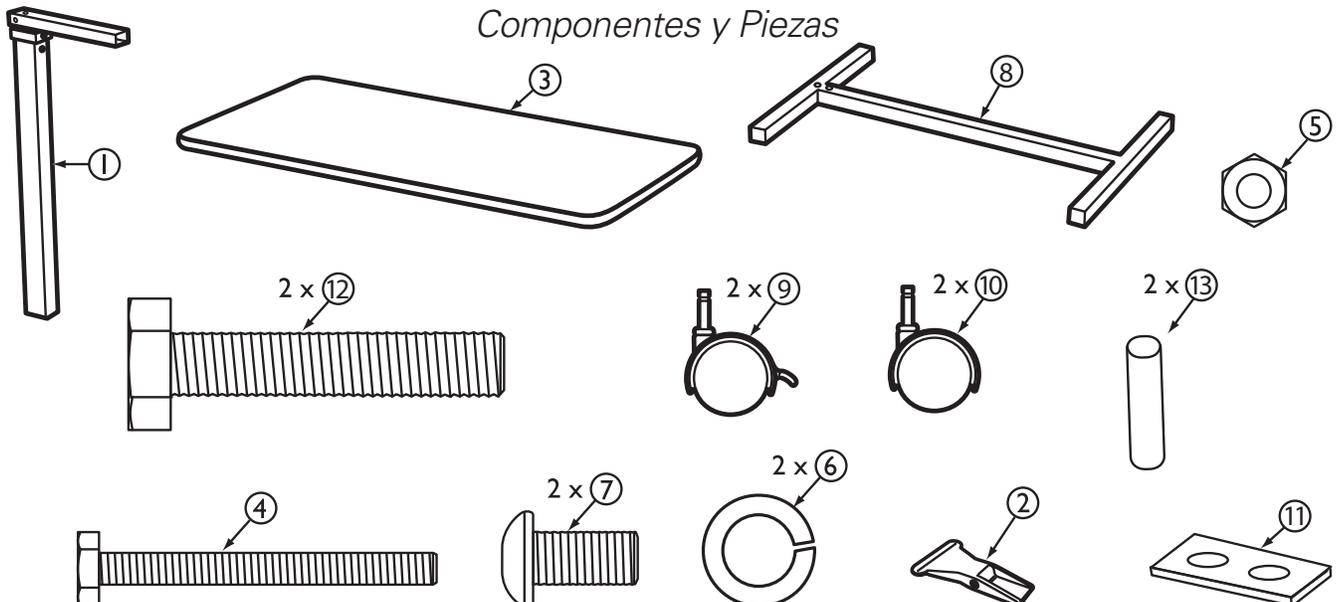


1






NOTE: For U-shaped bases, use the outer 8 holes. For H-shaped bases, use the centered 8 holes.

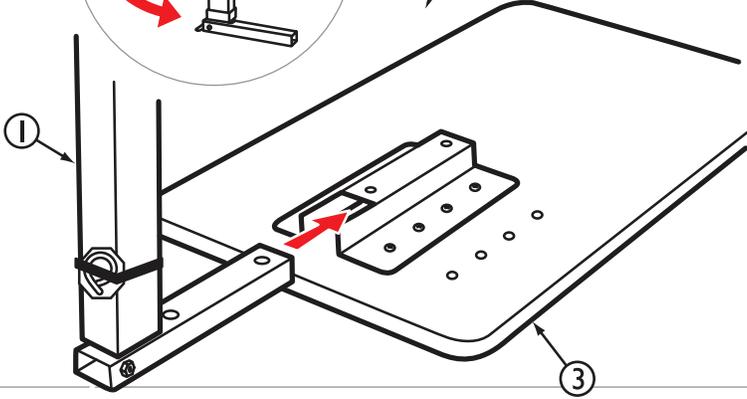



2

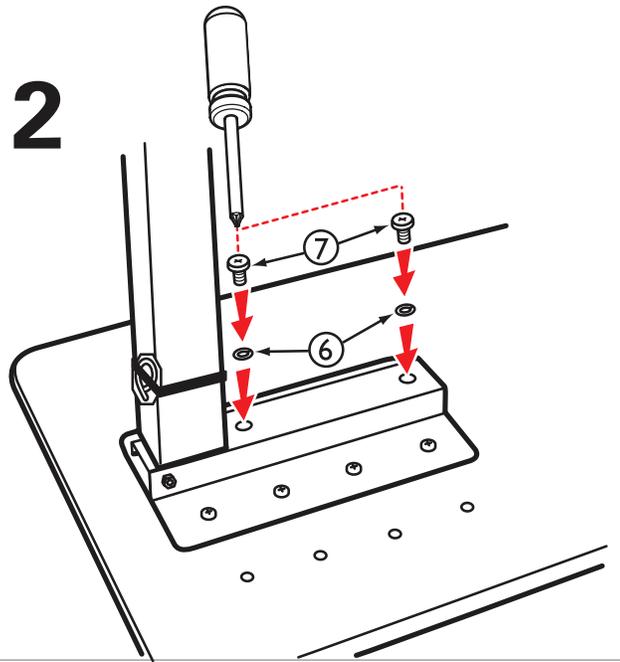




3

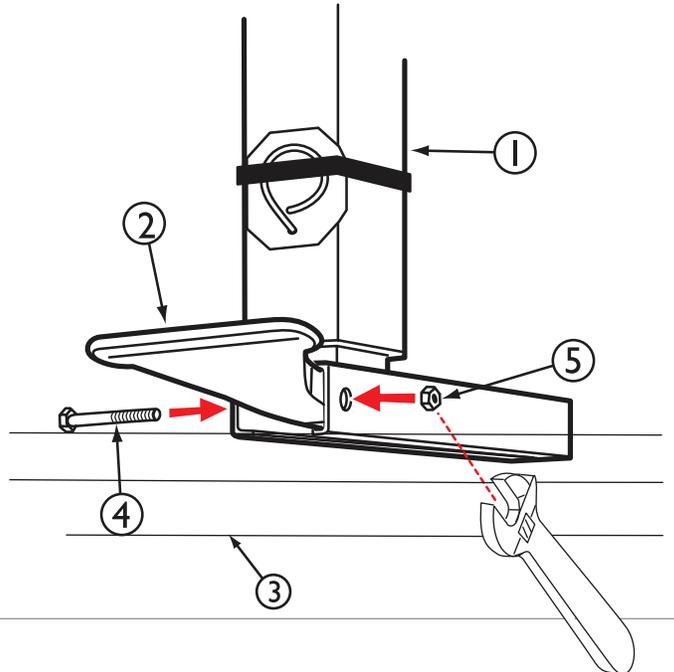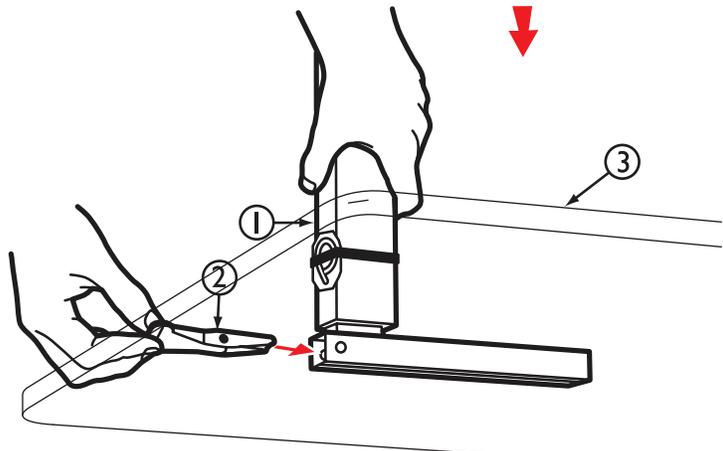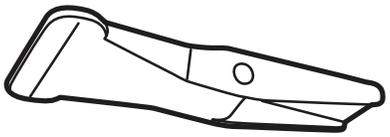



4

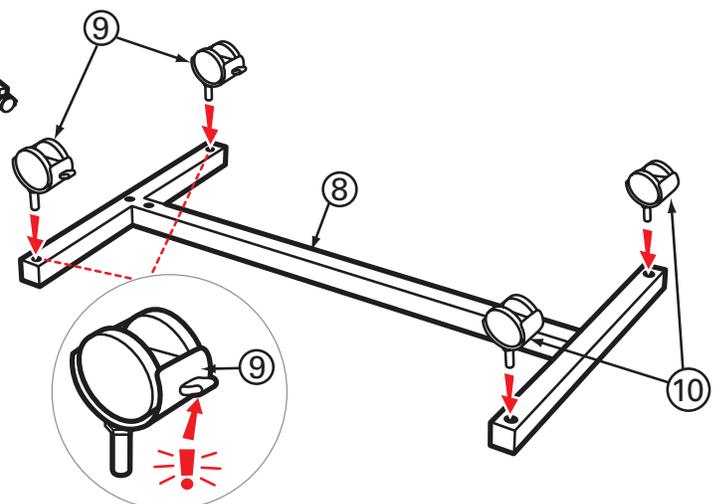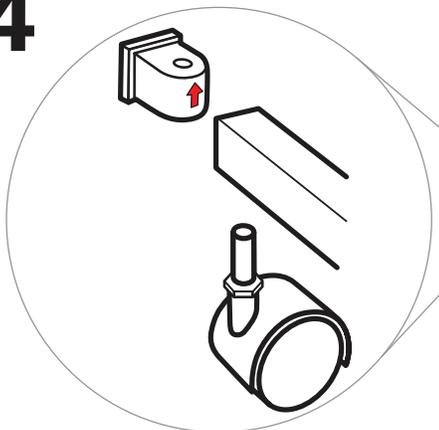


5

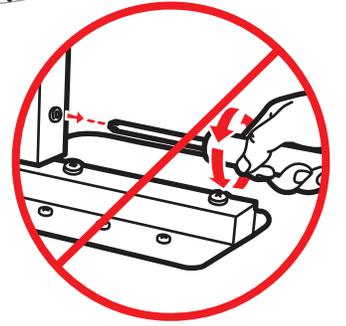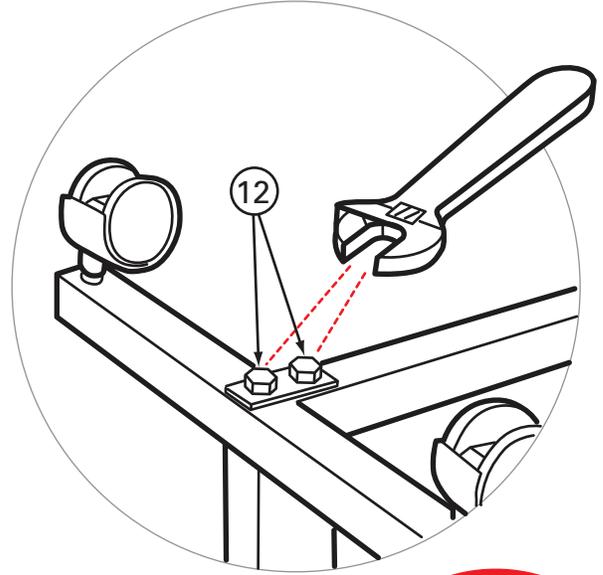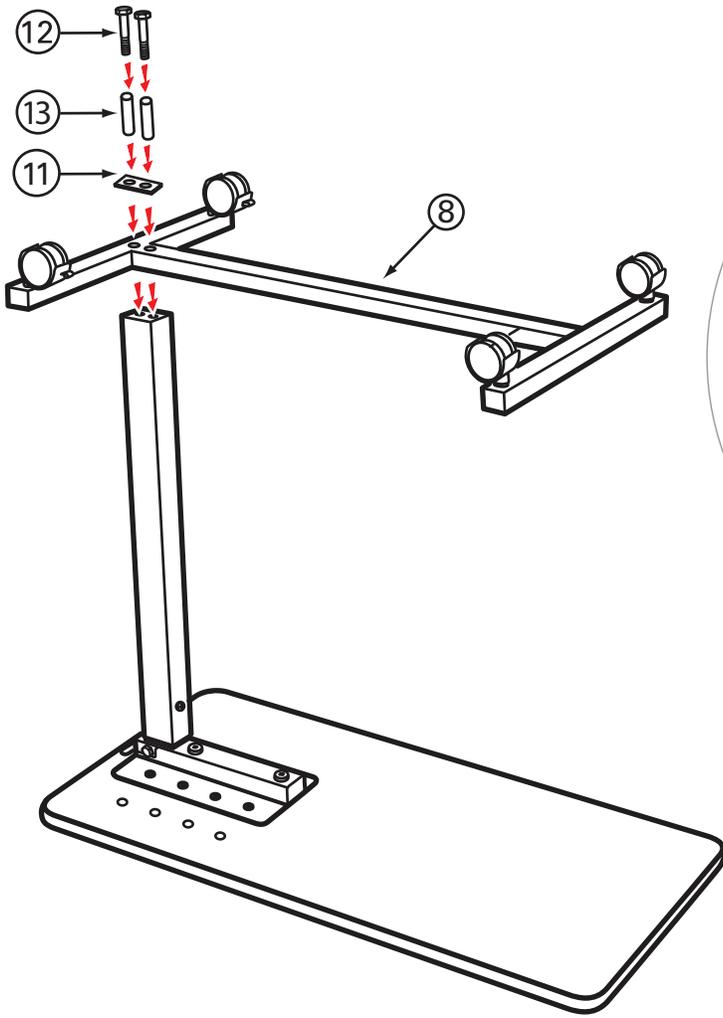



6

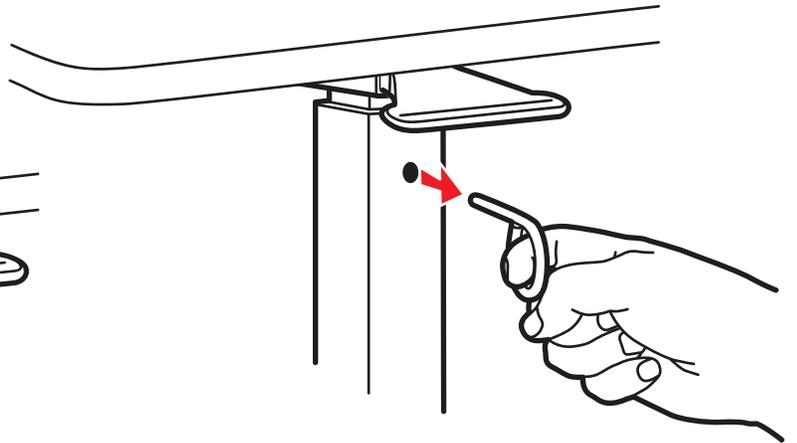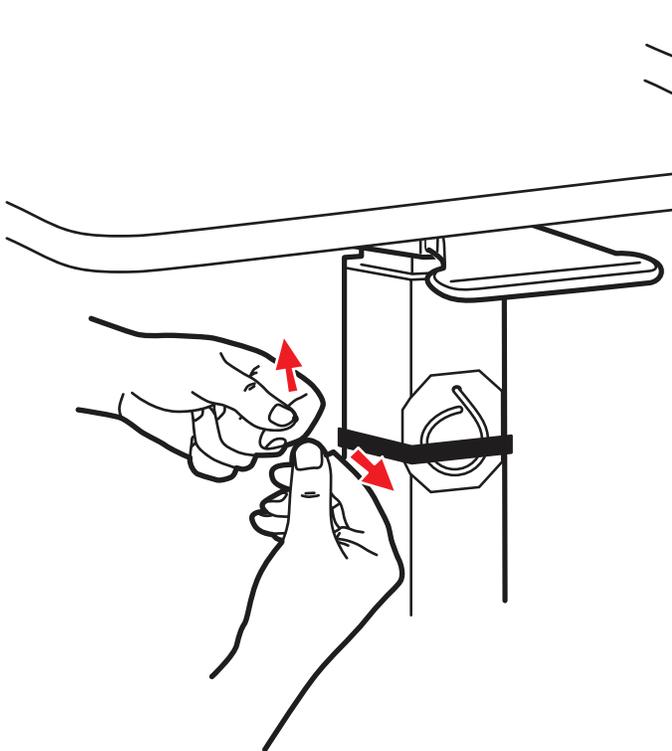

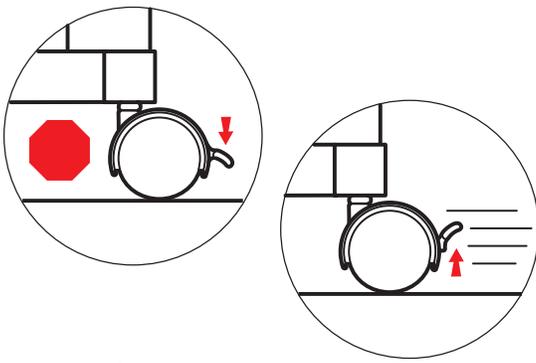

CUSTOMER SERVICE

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality overbed table, and enjoy outrageous customer service. If you have any questions about the overbed table you have purchased or would like to request warranty service, please contact the distributor from whom you purchased the overbed table. For a complete list of replacement parts for your overbed table, please visit store.directsupply.com or please contact your distributor.

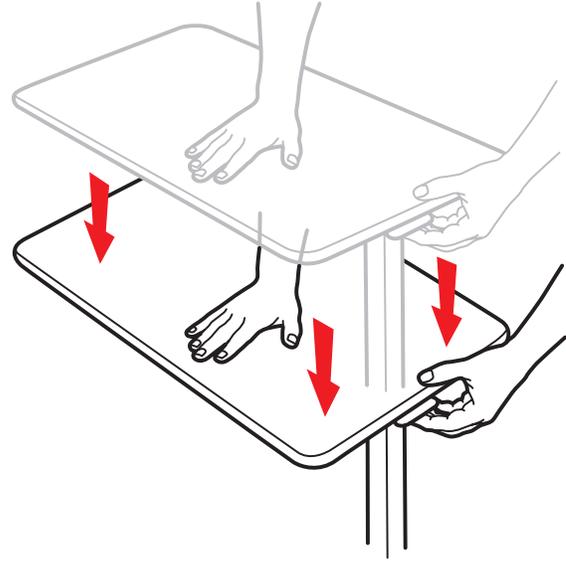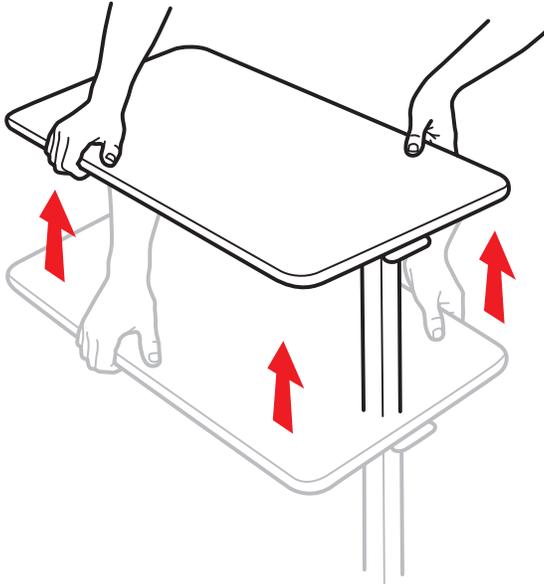

We offer to you, as the original purchaser, a warranty for the Direct Supply® Panacea® Overbed Table. Our warranty applies for the limited warranty period stated below. If any device or device part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a device or device part will be at our discretion. Our warranty applies only if the device is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the device, or damage caused by abuse, improper use, failure to maintain, use which exceeds the published device limitations, or the combination of any device with another product. In addition, our warranty does not cover fading, characteristics or natural variations in fabric, texture, colorfastness, stains, spills, or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts, and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary, from state to state. Please note that our limited warranty period begins when we ship the device to you. The limited warranty period and our obligations under the warranty end once you transfer the device to someone else, or at the end of the applicable limited warranty period identified below, whichever is earlier.

Product/Part	Warranty Period (Parts and Labor)	Anticipated Usable Device Life
Direct Supply® Panacea® Overbed Table	1 year	1 year

Anticipated Usable Device Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the device as described in this guide, as the device may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR CONSEQUENTIAL OR INCIDENTAL DAMAGES THAT MAY RESULT FROM THE DEVICE OR YOUR USE OF OR INABILITY TO USE THE DEVICE. TO THE EXTENT ALLOWED BY LAW, OUR TOTAL LIABILITY TO YOU,

IF ANY, IS LIMITED TO THE PRICE OF THE DEVICE OR SERVICE GIVING RISE TO YOUR CLAIM.

Some states do not allow an exclusion or limitation of incidental or consequential damages or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.



DirectSupply.com

APTURA / EQUIPMENT & FURNISHINGS / DSSI / TEL

Distributed by Direct Supply® Manufacturing™, Inc. 6635 W Champions Way, Milwaukee WI 53223.

© 2023 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply Panacea and all associated logos are the registered trademarks of Direct Supply, Inc., used under license. All other trademarks are the property of their respective owners.

Date of Printing: January 2024 15506