

# Attendant®

## 5-LITER OXYGEN CONCENTRATOR

# QUICK START GUIDE

### **Unpacking:**

1. Check for any obvious damage to the carton or its contents
2. If damage to the concentrator is evident, notify Direct Supply
3. Remove all loose packing from the carton
4. Carefully remove all the components from the carton

### **Inspection:**

1. Examine exterior of the oxygen concentrator for nicks, dents, scratches or other damages
2. Inspect all accessories for damage

### **Initial setup without humidifier bottle:**

1. Remove the cannula from the plastic packaging
2. Connect the cannula to the oxygen outlet on the concentrator
3. Plug the concentrator into a 120V AC power source
4. Turn the concentrator on by pressing the power switch  
("I" indicates the power is on, "O" indicates the power is off)

### **Optional initial setup with humidifier bottle:**

1. Remove humidifier bottle from plastic packaging
2. Unscrew the cover of the humidifier bottle
3. Add purified or distilled water to the bottle. Ensure that the water levels fall between the MINIMUM and MAXIMUM indicators on the side of the bottle
4. Screw the cover onto the humidifier bottle
5. Insert the humidifier bottle into the elastic strap on the front of the concentrator
6. Remove the cannula from the plastic packaging
7. Connect the cannula to the humidifier bottle outlet
8. Connect the oxygen hose between the oxygen outlet on the concentrator and the humidifier adapter on the humidifier bottle
9. Plug the concentrator into a 120V AC power source
10. Turn the concentrator on by pressing the power switch  
("I" indicates the power is on, "O" indicates the power is off)