



# Direct Supply®

# MULTIPURPOSE MOBILE

# WEIGHT RACK WITH MIRROR

## Owner's Manual

Thank you for purchasing a Direct Supply® Multipurpose Mobile Weight Rack from Direct Supply Equipment & Furnishings®, a division of Direct Supply, Inc. Please read this entire manual carefully and keep it for future reference. This manual will provide you with instructions, warnings, warranty information and other important information about your device. Share this information with your housekeeping, nursing and maintenance staff to help ensure the device is cared for properly.

## Definitions and Symbols

**NOTE:** Indicates a tip.

**CAUTION:** Indicates correct operating or maintenance procedures in order to prevent damage to or destruction of the equipment or other property.

**WARNING:** Calls attention to a potential danger that requires correct procedures or practices in order to prevent personal injury.

**DEVICE:** Your Multipurpose Mobile Weight Rack.

**YOU and YOUR:** The facility, community or other entity that has purchased the device.

**WE, US, and OUR:** Direct Supply Manufacturing, Inc.

⚠ Attention. Read the instructions.

## Warnings and Cautions

⚠ **WARNING:** Do not exceed the 300 lb. weight capacity on the Weight Rack.

⚠ **CAUTION:** Engage locking casters when cart is not in use.

⚠ **CAUTION:** Do not use if any loose fasteners are found.

⚠ **CAUTION:** Discontinue use of any product that is NOT in proper working order until situation is resolved. Failure to do so may result in serious injury.

⚠ **CAUTION:** Do not clean with harsh/abrasive chemicals as these may cause permanent damage to the device.

**NOTE:** Careful steps are taken to ensure you receive a quality product. As a best practice, thoroughly inspect the device after assembly for any imperfections that could appear from shipping or assembly, such as scratches, sharp edges or missing protective end caps.

## Directions for Use

Prior to assembly, ensure that you have received all parts specified below, that the items do not have any shipping damage, and that all holes are drilled and tested for alignment. If any part appears to be damaged or incorrect, do not continue with assembly and immediately contact Direct Supply.

This product is intended to be used as a storage unit for dumbbell weights, cuff weights, exercise bands, and other miscellaneous exercise supplies. Prior to each use, verify that the device is properly assembled, sturdy and clean.

## Cleaning Instructions

The wood surface may be cleaned with an antimicrobial wood cleaner and wiped off with a cloth. Be sure the surfaces are thoroughly dried after cleaning. Make sure the cleaning product is safe to use on wood. Use of harsh chemicals, or allowing liquids to sit over long periods of time, may damage the wood.

## Assembly Instructions

**TOOLS REQUIRED:** Corded/Cordless Drill with #2 Phillips Bit (OR) #2 Phillips Head Screwdriver

**NOTE:** Two people are required for assembly.

### PARTS CHECKLIST:

Part	Qty.	Description
<b>A</b>	1	Base
<b>B</b>	1	Body
<b>C</b>	2	Locking Caster
<b>D</b>	2	Standard Caster
<b>E</b>	2	Metal Dumbbell Rail
<b>F</b>	12	Peg Board Cuff Weight Hangers

### HARDWARE CHECKLIST:





Part	Qty.	Description
<b>G</b>	22	#12 x 3/4" Phillips Pan Screw
<b>H</b>	8	#8 x 1 1/4" Phillips Round Washer Wood Screw

### INSPECTION CHECKLIST:

- ☐ All holes are drilled and tested for alignment
- ☐ Base, Body, Casters, Rails, and Hangers are finished and present
- ☐ Hardware enclosed
- ☐ All components properly packaged

### PRODUCT SPECIFICATIONS:


Dimensions	71"H x 34"W x 19"D
Weight Capacity	300 lbs.

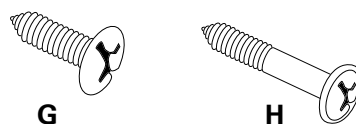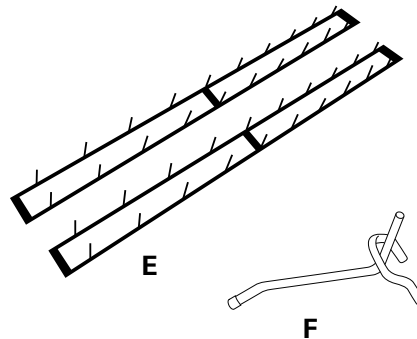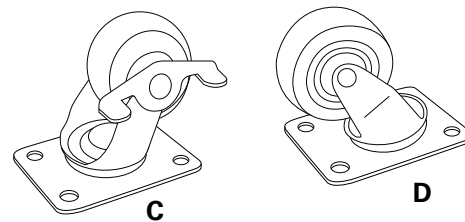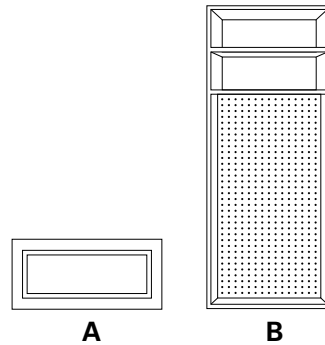

### STEP 1: Connect Base to Body

- Place the weight rack body (*Part B*) on a table with mirror face up and bottom of body closest to you (For orientation: top of body contains 2 shelves).
- Place the weight rack base (*Part A*) onto weight rack body, as shown in Figure 1.
- Use a drill with #2 Phillips bit and (8) #8 x 1¼" Wood Screws (*Part H*) to secure the base to the body (*Figure 2*). Pre-drilled hole locations for the screws are marked in Figure 1.

**NOTE:** This step is best accomplished with two people, one person to stabilize the base and one person to tighten the screws.

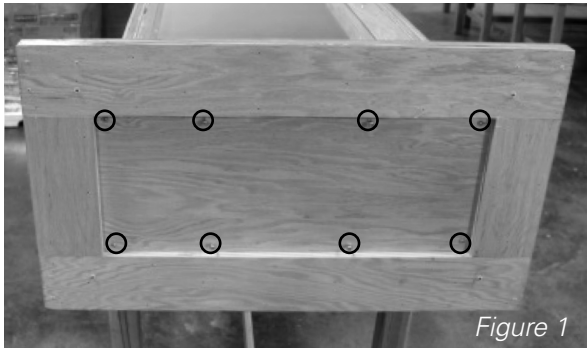

### STEP 2: Install Casters on Base

- Use a drill with #2 Phillips bit and (4) #12 x ¾" Pan Screws (*Part G*) to secure one of the locking casters (*Part C*) to the base on the pegboard side of the rack. See Figure 3.

**NOTE:** Pre-drilled holes on the base indicate the caster location.

- Secure the second locking caster to the base on the pegboard side of the rack using a drill with #2 Phillips bit and (4) #12 x ¾" Pan Screws.
- Secure the standard casters to the base on the mirror side of the rack using a drill with #2 Phillips bit and (4) #12 x ¾" Pan Screws for each caster.

**NOTE:** Pre-drilled holes on the base indicate the location of each caster.

- Use two people to lift the weight rack from the table and place it on its casters on the ground.
- Ensure the casters roll smoothly and caster locks function properly.




### STEP 3: Install Dumbbell Racks

- On one side of the weight rack, use a drill with #2 Phillips bit and (3) #12 x ¾" Pan Screws (*Part G*) to secure one metal dumbbell rack to the body. The dumbbell rack should be oriented so the shelves are extending at a slightly upward angle.

**NOTE:** Pre-drilled holes on the side of the weight rack body indicate the location of the dumbbell rack.

- Repeat for the dumbbell rack on the other side of the weight rack body.

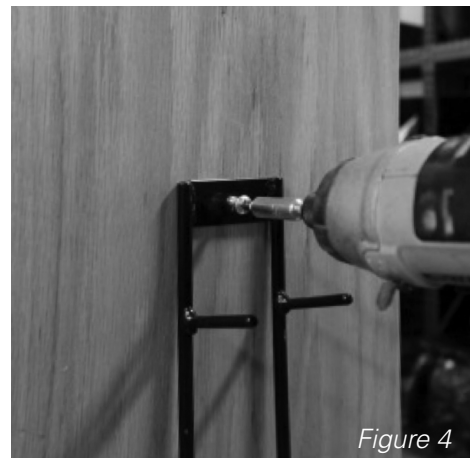

## Limited Warranty

We, Direct Supply Manufacturing, Inc. offer to you, as the original purchaser, a warranty for the Direct Supply® Multipurpose Mobile Weight Rack. Our warranty applies for the limited warranty period stated below. If any product or product part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a product or product part will be at our discretion.

Our warranty applies only if the product is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the product, or damage caused by abuse, improper use, failure to maintain, use which exceeds the published product limitations, or the combination of any product with another product. In addition, our warranty does not cover fading, colorfastness, stains, spills, or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts, and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary, from state to state. Please note that our limited warranty period begins when we ship the product to you. The limited warranty period and our obligations under the warranty end once you transfer the product to someone else, or at the end of the applicable limited warranty period identified below, whichever is earlier.

<b>Device</b>	<b>Warranty Period (Parts)</b>	<b>Anticipated Usable Product Life</b>
Direct Supply® Multipurpose Mobile Weight Rack	3 years	3 years

Anticipated Usable Product Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the product as described in this guide, as the product may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE PRODUCT OR YOUR USE OF OR INABILITY TO USE THE PRODUCT EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE PRODUCT OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.

## Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality Direct Supply® Multipurpose Weight Rack and enjoy outrageous customer service. If you have any questions about the weight rack you have purchased or would like to request warranty service, please contact: **Direct Supply Equipment & Furnishings**, 1-800-634-7328, 6767 N. Industrial Road, Milwaukee, WI 53223, SalesSupport@DirectSupply.com.



*DirectSupply.com*

*APTURA / EQUIPMENT & FURNISHINGS / DSSI / TELS*

Distributed by Direct Supply Manufacturing, Inc. 6635 N Industrial Road, Milwaukee, WI 53223.

© 2017 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply, Direct Supply Equipment & Furnishings and all associated logos are the registered trademarks of Direct Supply, Inc., used under license.