

Limited Warranty

The seat of your Direct Choice™ Raised Toilet Seat is warranted by Direct Supply Manufacturing, Inc. ("we" and "our") to be free from material defects in workmanship and materials for the lifetime of the device when properly maintained and used for its intended purpose. The lock and armrests of the device are warranted by us to be free from material and defects in workmanship and materials for one (1) year after shipment again when properly maintained and used for its intended purpose.

If the device is defective in material or workmanship during the applicable limited warranty period, we will, at our cost and our option, repair or provide a replacement unit or parts so that you may adjust or replace a defective part or unit as your sole and exclusive remedy. Our warranty gives you specific legal rights, and you may also have other rights, which vary from state to state. The limited warranty period and our obligations under the warranty end once you transfer the device to someone else, or at the end of the applicable limited warranty period identified, whichever is earlier.

This warranty and its anticipated usable device life of three (3) years is based on normal, intended use and does not cover wear and tear, modification, accidents, misuse, use outside published limitations, circumstances beyond our control, devices with removed or defaced serial numbers, combination of the device with another product, fading, spills or exposure to unusual chemicals, odors, heat or light. You should monitor the device as it may need to be replaced sooner than the anticipated usable device life in certain situations.

WE DISCLAIM ALL WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE WHETHER EXPRESS OR IMPLIED. TO THE FULLEST EXTENT ALLOWED BY LAW, WE WILL NOT BE LIABLE FOR ANY INCIDENTAL, INDIRECT, CONSEQUENTIAL OR PUNITIVE DAMAGES THAT MAY RESULT FROM THE DEVICE OR YOUR USE OR INABILITY TO USE THE DEVICE EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. ANY LIABILITY TO YOU IS LIMITED TO THE PRICE OF THE PRODUCT GIVING RISE TO YOUR CLAIM.

Some states do not allow an exclusion or limitation of incidental or consequential damages or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, we limit any and all implied warranties to the applicable warranty period identified above to the extent permitted by law.

Direct Choice™ products are available to purchase exclusively through Direct Supply® Equipment & Furnishings™, a division of Direct Supply®, Inc. Distributed by Direct Supply Manufacturing, Inc. 6699 N Industrial Road, Milwaukee WI 53223.

© 2010 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply®, Direct Choice®, and all associated logos are the registered trademarks of Direct Supply Inc., used under license.

Deluxe Raised Toilet Seat with Arms & Safety Legs

DIRECT
SUPPLY®
Direct Choice™



Assembly Instructions

SET-UP

The Direct Choice locking raised toilet seat with arms & safety legs has been designed to accommodate a wide variety of toilet bowl types. However, certain bowl types will require the removal of the existing seat and/or lid to ensure proper fit when installed.

1. Lift up toilet seat and lid. You may also need to remove the entire seat and lid from the toilet (varies depending on the size of the toilet).
2. Make sure the toilet seat is clean and sanitized.
3. Turn the toilet seat upside-down and insert the safety legs into the holes in the raised toilet seat.
4. Turn the toilet seat onto its back and insert the safety arms into the safety legs until the locking button clicks into place.
5. Turn the unit upright and place the raised toilet seat onto the toilet. Make sure the rear anchor of the raised toilet seat sits beneath the back of the rim.
6. Adjust the height of the safety legs using the locking buttons until the feet touch the floor and the toilet seat is sitting directly on top of the toilet bowl.
7. Make sure the square clamp is set against the inside of the toilet's front rim.
8. Turn the knob clockwise until the raised toilet seat is securely locked on the toilet (do not over-tighten the knob).
9. Pull on the raised toilet seat to make sure it is securely locked on the toilet. When the raised toilet seat is properly installed, the unit will sit flat on the top of the bowl and should fit securely against the inner rim of the bowl.

CARE AND MAINTENANCE

1. Check regularly to make sure the raised toilet seat is securely locked onto the toilet.
2. Clean regularly with a toilet sanitizer or with regular household cleaning products. **DO NOT** use abrasive powders or scouring pads.

Safety Precautions

DO NOT ATTEMPT TO ADJUST OR OPERATE TOILET SEAT WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Always be sure that the seat is correctly and securely locked in place before using
- Be sure that the adjustment knob in front of the seat is tight at all times, thus securing the seat to the toilet bowl.
- This product type is designed for use with the most common types of commodes. It will not fit all commode types. Be sure that the unit has been installed correctly, fits properly and is stable before use.
- The arms are for the purpose of providing assistance when getting on and off the seat. **DO NOT** try to use the arms to support full body weight during transfer, as this may result in failure of the unit.
- Direct Choice assumes no responsibility for any damage or injury caused by improper **installation, assembly or use of this product.**

Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality Deluxe Raised Toilet Seat with Arms & Safety Legs, and enjoy outrageous customer service. If you have any questions about the raised toilet seat you have purchased or would like to request warranty service, please contact: Direct Supply® Equipment & Furnishings™ at 1-800-634-7328, 6767 North Industrial Road, Milwaukee, WI 53223, deardirect@directsupply.net.