


OXFORD / HOYER JOURNEY PARTS MANUAL

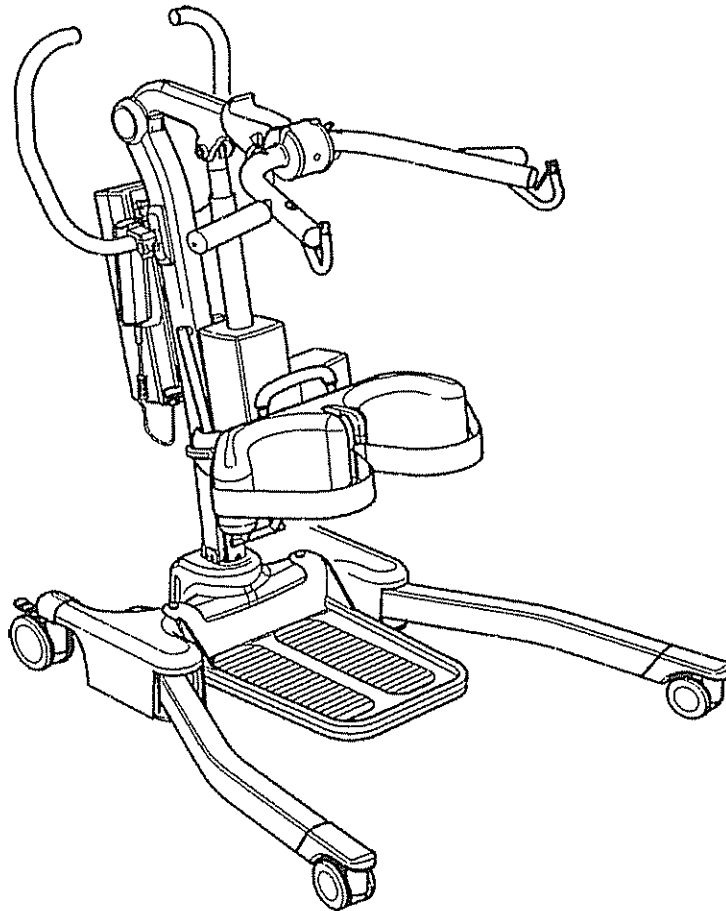

Joerns Healthcare Limited
High Street, Wollaston, West Midlands, DY8 4PS
Tel: +44 (0)1384 44 66 22
Fax: +44 (0)1384 44 66 01
info@joerns.co.uk
www.joerns.co.uk


Joerns Healthcare Inc
5001 Joerns Drive, Stevens Point
Wisconsin, 54481-5040
Tel: (715) 341 3600
(800) 826 0270 (Toll Free)
www.joerns.com

Rev. A

Oxford/Hoyer Journey

294000-10054

OXFORD / HOYER JOURNEY

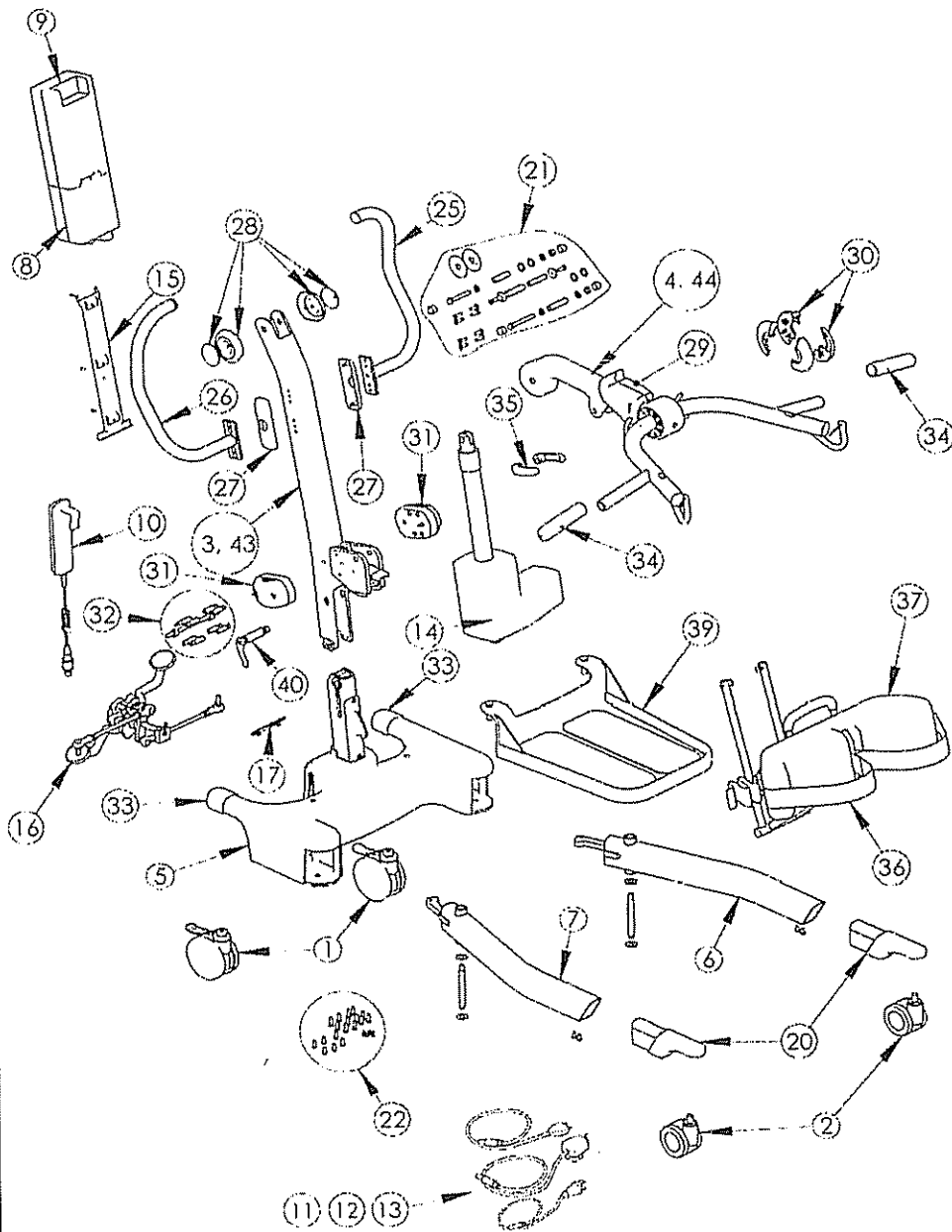

Rev. A

OXFORD / HOYER JOURNEY			
Nº	PART NUMBER	DESCRIPTION	QTY
1	0Y0066	100mm rear castor (braked) (Pack of 2)	2
2	0Y0109	75mm front castor (non-braked) (Pack of 2)	2
3	0Y0680	Mast Assembly (including latch mechanism) Oxford	1
4	0Y0684	Cow Horn and Boom Assembly - Oxford	1
5	0Y0660	Base assembly (inc. base casting, bottom plate, welded post & labels. not internal parts)	1
6	0Y0675	Left leg extrusion (including fixings)	1
7	0Y0676	Right leg extrusion (including fixings)	1
8	0Y0678	Linak controller	1
9	0Y0067	Linak battery	1
10	0Y0111	Linak hand control	1
11	0Y0071	Linak charge lead (UK)	1
12	0Y0068	Linak charge lead (US) 0Y0114	1
13	0Y0072	Linak charge lead (EU)	1
14	0Y0677	Linak actuator - 210mm stroke (Inc. location pins, sleeves, cover, grub screw, hex key and labels)	1
15	0Y0069	Linak battery and controller bracket (including fixings)	1
16	0Y0669	Foot pedal mechanism complete	1
17	0Y0042	Foot push pad assembly complete (Pack of 2)	2
18	0Y0668	Decal set (Oxford) not shown - no serial number labels	1
19	0Y0667	Decal set (Hoyer) not shown - no serial number labels	1
20	0Y0045	Front castor leg inserts including fixings (plastic) (Pack of 2)	2
21	0Y0659	Upper Fixings Kit	1
22	0Y0661	Base Fixings kit (all bolts, screws and fixings)	1
23	0Y0070	Handset clip - not shown (Pack of 2)	2
24	0Y0053	Linak charger and stand assembly (not shown)	1
25	0Y0684	Push Handle Left Hand	1
26	0Y0685	Push Handle Right Hand	1
27	0Y0683	Push Handle Covers (Pack of 2)	2
28	0Y0682	Pivot Joint Plastic Cover (includes inner and outer piece - Pack of 2 sets)	2
29	0Y0662	Cover Cow Horn Adjustment Slide (left and Right part)	1
30	0Y0666	Cow Horn Indicator Cover (Left and Right Hand)	1
31	0Y0674	Kneepad rod adjustment clamp	1
32	0Y0671	Kneepad adjustment fixings	1
33	0Y0686	Rear Base Cast Cover (Pack of 2)	2
34	0Y0665	Cow Horn Handle Grips (Pack of 2)	2
35	0Y0672	Kneepad Adjustment Handle	1
36	0Y0500	Leg Strap	1
37	0Y0499	Knee pad assembly (including fixing bolts)	1
38	0Y0673	Kneepad assembly (Not including Kneepad or leg strap) (not shown)	1
39	0Y0670	Foot Tray Assembly	1
40	0Y0681	Mast Lock Pin and Handle	1
41	0Y0687	Sling Retaining Clip (Pack of 10) (not shown)	10
42	0Y0128	Castor Tightening Wrench - not shown	1
43	0Y0679	Mast Assembly (including latch mechanism) - Hoyer	1
44	0Y0663	Cow Horn and Boom Assembly - Hoyer	1