



Panacea®

WHEELCHAIR REAR ANTI-TIPPER

Owner's Manual

Thank you for purchasing the Panacea® Wheelchair Anti-Tipper. Please read this entire manual carefully and keep it for future reference. This manual will provide you with instructions, warnings, warranty information and other important information about your device.

⚠ WARNING

- Do not remove or disable the anti-tipper device. Doing so may result in the wheelchair tipping over, causing injury to the user.
- Always ensure that the anti-tipper arms are securely attached to the wheelchair frame. Loose or damaged anti-tippers may compromise stability.
- Avoid sudden movements or sharp turns when using the wheelchair. Abrupt changes in direction may cause the anti-tipper to engage unexpectedly.

⚠ CAUTION: Proper Adjustment Required

- Before using the wheelchair, verify that the anti-tipper arms are adjusted correctly. Improper adjustment may reduce effectiveness.
- Regularly inspect the anti-tipper components for wear, damage, or rust. Replace any worn or damaged parts promptly.
- When transferring the user in and out of the wheelchair, ensure that the anti-tipper does not interfere with the transfer process.
- The anti-tippers are compatible with many standard and light weight wheelchairs. Proper fit and installation must be verified and the device should not be used if it does not fit properly. Never modify the anti-tipper during installation.

Indications for Use

The Panacea wheelchair anti-tipper device is designed to reduce tipping or backward falls when the wheelchair encounters obstacles, ramps, or uneven surfaces. It increases stability and safety during both indoor and outdoor use. The anti-tipper should be installed and adjusted to make contact with the floor when the wheelchair tilts backward, effectively limiting the wheelchair from tipping. Verify that the anti-tipper device is appropriately installed and functioning correctly prior to each use.

Directions for Use

1. Remove the plastic caps from the tubes on the back of the wheelchair. They may need to be cut to remove them.
2. Insert anti-tipper device into the tubes on the back of the wheelchair and secure properly. The squeeze buttons should click into the holes in the wheelchair frame.
3. Adjust the height of the anti-tipper by squeezing the locking buttons and pulling the wheels out until the buttons click into a location when the wheels are close to but not touching the ground. Some clearance is needed to allow the wheelchair to turn without the anti-tipper scraping the floor.

⚠ CAUTION: Improper installation without some clearance between the anti-tipper and the ground can cause damage to the device when turning the wheelchair.

⚠ WARNING: Check for proper performance of the anti-tippers by pushing down on the handles to confirm the wheelchair cannot tip backwards before using with residents or patients.

Cleaning Instructions

Use mild non-abrasive soap to clean the device, rinse and then dry to prevent corrosion.

Limited Warranty

We, Direct Supply Manufacturing, Inc. offer to you, as the original purchaser, a warranty for the Panacea Wheelchair Anti-Tipper. Our warranty applies for the limited warranty period stated below. If any device or device part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a device or device part will be at our discretion.

Our warranty applies only if the device is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the device, or damage caused by abuse, improper use, failure to maintain, use which exceeds the published device limitations, or the combination of any device with another device. In addition, our warranty does not cover fading, colorfastness, stains, spills or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts, and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary from state to state. Please note that our limited warranty period begins when we ship the device to you. The limited warranty period and our obligations under the warranty end once you transfer the device to someone else or at the end of the applicable limited warranty period identified below, whichever is earlier.

Device	Warranty Period (Parts)	Anticipated Usable Device Life
Panacea® Wheelchair Anti-Tipper	3 years	3 years

Anticipated Usable Device Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the device as described in this guide, as the device may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE DEVICE OR YOUR USE OF OR INABILITY TO USE THE DEVICE EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE DEVICE OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages, or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.

Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality Panacea Wheelchair Anti-Tipper and enjoy outrageous customer service. If you have any questions about the Wheelchair Anti-Tipper you have purchased or would like to request warranty service, please contact the distributor from whom you purchased the device.



Distributed by Direct Supply® Manufacturing™, Inc. 6635 W Champions Way, Milwaukee WI 53223.

© 2024 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply and all associated logos are the registered trademarks of Direct Supply, Inc., used under license. All other trademarks are the property of their respective owners.