

Limited Warranty

We, Direct Supply Manufacturing, Inc., offer to you, as the original purchaser, a warranty for the Direct Supply® Drop-Arm Commode. Our warranty applies for the limited warranty period stated below. If any product or product part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a product or product part will be at our discretion.

Our warranty applies only if the product is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the product, or damage caused by abuse, improper use, failure to maintain, use which exceeds the published product limitations or the combination of any product with another product. In addition, our warranty does not cover fading, colorfastness, stains, spills or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts, and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary from state to state. Please note that our limited warranty period begins when we ship the product to you. The limited warranty period and our obligations under the warranty end once you transfer the product to someone else or at the end of the applicable limited warranty period identified below, whichever is earlier.

Device	Warranty Period (Parts)
Drop Arm	Limited Lifetime (frame and hardware)
Commode	1 year (plastic parts)

Anticipated Usable Product Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the product as described, as the product may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE PRODUCT OR YOUR USE OF OR INABILITY TO USE THE PRODUCT EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE PRODUCT OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages, or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.




Direct Supply® DROP-ARM COMMmode

Owner's Manual

Please keep and refer to this Owner's Manual.

Thank you for purchasing a Direct Supply® Drop-Arm Commode. Please read this entire guide carefully and keep it for future reference. This guide will provide you with instructions, warnings, warranty information, and other important information about your Drop-Arm Commode. Share this information with those individuals who will be assembling, using, servicing, and/or cleaning the product to help ensure it is cared for properly.

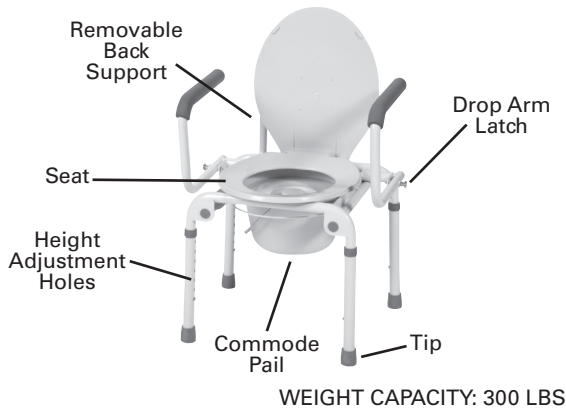

Assembly Instructions

SET-UP

1. Install the removable back support bar by inserting it into the holes in the back of the commode frame until the buttons snap into place.
2. Allow plastic parts to reach room temperature before assembly to prevent cracking.
3. Align the toilet seat with the back of the frame rail. Push down on seat firmly until the clamps snap into place.
4. Repeat step #2 for the toilet seat lid.
5. Place the splash guard onto the commode bucket frame. Slide commode pail onto the support rails from rear under seat.

ADJUST COMMODE HEIGHT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
2. Repeat for other three legs.

⚠ CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

USE AS A BEDSIDE COMMODE

1. Set commode to desired height
2. Slide commode pail onto pail supports from rear below seat. Ensure that pail is properly positioned below commode opening. Commode is now ready to use.
3. When user is finished using commode, place commode pail lid over commode pail to contain odors.
4. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

DROP ARM USAGE

1. To lower drop arms, push in latch and move arm into down position.
2. To raise drop arms, pull up arms until they are fully seated in latches.

MAINTENANCE

1. Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that commode is well rinsed before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of commode.
3. Check fit and tightness of all nuts, bolts and knobs weekly to ensure that the commode is safe to use.

Safety Precautions

⚠ WARNING: DO NOT ATTEMPT TO ADJUST OR OPERATE COMMODE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- This product should not be used without instruction from a healthcare professional.
- Toilet seat must be in down position before use. Failure to do so may result in injury.
- Make sure that the height-adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure the leg extensions are locked in position and an even height is achieved.
- All four leg extensions with rubber tips must touch the floor at all times.
- Users with limited physical strength should be supervised or assisted while using commode.
- Do not exceed the maximum weight capacity intended for this product. This product's weight capacity is 300 lbs.
- Direct Supply assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality Drop-Arm Commode, and enjoy outrageous customer service. If you have any questions about the commode you have purchased or would like to request warranty service, please contact the distributor from whom you purchased the product.