



Sara® Plus

Sara Plus is a powered standing and raising aid designed for active transfers, balance, stepping and gait training.

ARJO MOBILITY GALLERY



C



D

C. Able to partially bear weight on at least one leg. Uses a wheelchair for moving a distance.
D. Unable to weight bear through feet. Is able to sit if well supported.
Please contact Arjo for further information.

Visit the webpage



KEY FEATURES

SCALE (OPTION)

The scale can be used to weigh patients, and monitor their weight-bearing capacity when Sara Plus is used without the footplate.

DUAL CONTROLS

Dual controls on the mast and the hand control. Using the hand control gives the caregiver the opportunity to move and stay close to the patient and maintain good eye contact throughout the process.

ARC REST

The arc rest supports the patient's arms and upper body and can help them support their weight through their shoulders in addition to their feet.

PRO-ACTIVE PAD

The pro-active knee pad provides support and stability, while also allowing natural forward movement of the knees and flexion of the ankles during standing and sitting actions.

DETACHABLE FOOTPLATE

Detachable footplate enables stepping and gait training with the patient's feet directly on the floor.



PRODUCT INFORMATION

GENERAL

| | |
|--|---------------------|
| Maximum safe working load | 190 kg (420 lbs) |
| Lift weight (incl battery) | 78.7 kg (173.5 lbs) |
| Minimum total storage height | 1195 mm (47") |
| Minimum lifting height | 906 mm (35 5/8") |
| Maximum lifting height | 1497 mm (58 7/8") |
| External width legs closed | 646 mm (25 3/8") |
| External width legs open | 1107 mm (44") |
| Internal width leg closed | 534 mm (21") |
| Internal width leg open | 1002 mm (39 3/8") |
| Leg clearance (floor to top of legs) | 116 mm (4 1/5") |
| Chassis clearance (floor to bottom of chassis) | 30 mm (1 1/5") |
| Lift total length | 1033 mm (40 5/8") |
| Turning radius | 1235 mm (48 5/8") |
| Lift protection class | IPX4 |
| Hand control protection class | IPX7 |
| Battery | 24V, 6 Ah |
| Battery charge indicator | |
| Service meter - displays the total time in use (hours) | |
| Emergency stop and system failure override | |
| Automatic safety cut out if lowered onto an obstacle | |
| Low-friction castors, the rear two with brakes | |

APPROVALS

The product is in compliance with current legislation and applicable product standards.

SCALE

| | |
|-----------------------------|---|
| Safe working load (SWL) | 190 kg (420 lbs) |
| Accuracy - Standard scale | 2-120 kg +/- 100 g (4-240 lb +/- 3.5 oz) 120-190 kg +/- 1000 g (240-418 lb +/- 7 oz) |
| Power supply | Main lift battery |
| Operating temperature range | +5°C to +35 °C (+41 °F to +95 °F) |
| Weight - scale unit | Approx 10.6 kg (18 lbs) |
| Protection | IP 24 |
| In accordance with | EN ISO 10535 IEC 60601-1 |

PRODUCT NOMENCLATURE

| | | | | | | | |
|-----------|---|-------|---|---|------------------|---|---|
| HEP | X | 0 | 0 | 1 | - | X | X |
| Sara Plus | | Scale | | | Country / Region | | |

SARA PLUS:



SCALE:

| | |
|---|-----------------|
| 0 | None |
| 1 | Standard scale |
| 2 | Class III scale |

COUNTRY/REGION:

| | |
|-------|---|
| Empty | European plug of main lead (Type C) |
| UK | British plug of main lead (Type G) |
| US | American plug of main lead (Type A) |
| AU | Australia plug of main lead (Type I) |
| CN | Chinese plug of main lead (Type A with CCC mark) |
| BR | Brazil plug of main lead (Type C with inmetro approval) |
| IL | Israel plug of main lead (Type H) |
| KR | Korean plug of main lead (Type F with KSC8305 approval) |
| ZA | South african plug of main lead (Type M) |
| JP | Japanese plug of main lead (Type A with PSE approval) |

General Note: Product availability may vary depending on your region.

MOST COMMON SPARE PARTS AND ACCESSORIES

| ARTICLE # | DESCRIPTION |
|------------|--|
| 1 HMX977 | 4 functions handset |
| 2 SPL3021 | Battery pack |
| 3 KKS5050 | Commode package (commode frame & seat) |
| 4 KKA5180 | Footboard narrow |
| 4 KKA5190 | Footboard standard |
| 4 KKA5200 | Footboard extension 100 mm (3") |
| 5 KKA5210 | Kneepad standard |
| 5 KKA5220 | Kneepad wide |
| 6 KKS5004 | Arc rest with handgrips |
| 7 KKS5006 | Grab handle assembly |
| 8 SPL3002 | Rear braked castor 100 mm (3") |
| 9 SPL3001 | Front castor 100 mm (3") |
| KKX53400.0 | Kneepad fleece cover |
| KKA5110 | Kneepad strap |
| SPL3024S | Wall-mounted charger |
| KKS5240 | Sara Plus look & lock assy |
| KKA5170 | Arm straps |

This list represents the most frequently sold spare parts and/or accessories. For a complete list, refer to the product parts list.






SLINGS

| ARTICLE # | NAME | SIZE |
|-----------|---------------------------------|--|
| KKA5120 | Sara Plus Standing Sling | S M L XL |
| KKA5130M | Sara Plus Walking Sling | S M L XL |
| KKA5370 | Sara Plus EPS Sling (BOS Sling) | XS S M L XL |
| KKA6000 | Wipeable Standing C-Hook sling | S M L XL |
| MFA4000 | SARA Plus Flites | One size |

SPACE REQUIREMENTS

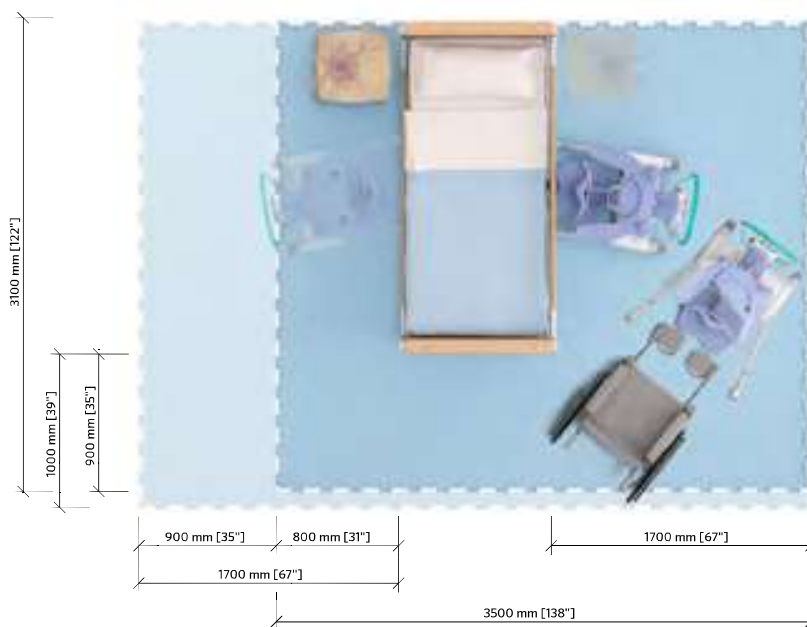

For more information about Space requirement, consult the Arjo Guide for Architects and Planners website.



- Blue area shows the minimum working area required for the staff to be able to use the floor lift in an ergonomic way from one side.
- Light blue area shows required extension of working area to facilitate activities from either side to provide adequate access for the patient, floor lift and assisting carer.