

When to Take Measurements

First thing in the morning or at the end of treatment so the edema is at its most reduced state.


How to Take Length Measurements

Length measurements should be taken at each circumference. It is recommended that you mark the arm with a non-permanent, nontoxic marker.

Arm sleeve B, C, D, E*

Gauntlet B & C

Glove A, B & C

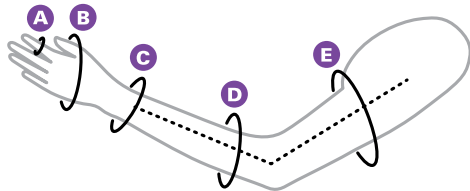

	S	M	L
A	1 ³ / ₄ " - 2 ³ / ₈ " (4.4 - 6 cm)	2 ³ / ₁₆ " - 3" (5.5 - 7.5 cm)	2 ⁵ / ₈ " - 3 ⁹ / ₁₆ " (6.6 - 9 cm)
B	5 ⁷ / ₈ " - 7 ³ / ₈ " (15 - 18.7 cm)	7 ¹ / ₂ " - 8 ⁵ / ₈ " (19 - 21.9 cm)	8 ³ / ₄ " - 10 ¹ / ₄ " (22 - 26 cm)
C	6" - 7 ¹ / ₂ " (15 - 19 cm)	6 ¹ / ₂ " - 8 ¹ / ₂ " (16.5 - 21.6 cm)	7 ¹ / ₂ " - 9 ¹ / ₄ " (19 - 23.5 cm)
D	7" - 9 ¹ / ₈ " (17.8 - 23.2 cm)	9 ¹ / ₄ " - 11 ¹ / ₄ " (23.3 - 28.7 cm)	11 ³ / ₈ " - 13 ¹ / ₂ " (28.8 - 34.3 cm)
E	8 ¹ / ₂ " - 15 ¹ / ₂ " (21.5 - 39.5 cm)	10 ⁵ / ₈ " - 17 ³ / ₄ " (27 - 45 cm)	12 ³ / ₄ " - 19 ⁷ / ₈ " (32.5 - 50.5 cm)

	Length B - E
Long Length*	16 ¹ / ₂ " - 22" (43 - 56 cm)
Short Length*	14 ¹ / ₂ " - 17" (37 - 44 cm)

*Both short and long arm sleeve extend entire length of arm; the difference in size relates to the length of the person's arm