



# Attendant® MEDICAL CART

## Owner's Manual

*Please keep and refer to this Owner's Manual.*

Thank you for purchasing a Direct Supply® Attendant Medical Cart. Please read this entire guide carefully and keep it for future reference. This guide will provide you with instructions, warnings, warranty information, and other important information about your Cart. Share this information with those individuals who will be assembling, using, servicing, and/or cleaning the product to help ensure it is cared for properly.

# Table of Contents

## Table of Contents

<b>Introduction .....</b>	<b>3</b>
<b>Safety Precautions.....</b>	<b>5</b>
<b>Specifications.....</b>	<b>10</b>
<b>Optional Side Accessories .....</b>	<b>12</b>
Tool & Accessory Package Standard .....	12
Push Handle .....	13
Removal of Drawer and Base.....	14
Accessory Rail.....	15
Oxygen Tank Side Rail .....	16
Catheter Holder.....	17
Glove Dispenser Holder .....	17
Multi Bin Storage Basket .....	17
Waste Container and Lid.....	17
Sharps Container Holder .....	18
<b>Optional Accessories Back &amp; Top.....</b>	<b>19</b>
CPR Cardiac Board & Hooks .....	19
Defibrillator Shelf .....	20
IV Pole .....	21
5 & 12 Bin Organizer with Accessory Bridge .....	22
Drawer Divider System .....	24
<b>Limited Warranty .....</b>	<b>25</b>
<b>Customer Service .....</b>	<b>26</b>

# Introduction

This manual will provide you with information regarding specifications, maintenance, cleaning and warranty information. Should you have more questions, please call 1-800-634-7328, Monday through Friday from 8 am to 5 pm Central Standard Time.

Throughout the manual, the term “product” refers to the Attendant Medical Cart. “You” and “your” refer to the facility, community or other entity that has purchased the product. “We,” “us” and “our” refer to Direct Supply®, Inc.

## Recommended Use

Attendant’s medical carts come with many useful options that allow doctors, nurses and care takers to quietly, carefully and expediently transport medical supplies and instruments around your healthcare facility. These medical carts are color coded to help healthcare professionals visually designate a certain cart to a department.

## Definitions and Symbols

**NOTE:** Indicates a tip.

**⚠ CAUTION:** Means correct operating or maintenance procedures which must be followed in order to prevent damage to or destruction of the product or other property.

**⚠ WARNING:** Calls attention to a potential danger that requires correct procedures or practices in order to prevent personal injury.

**Attention.** Read the instructions.

**PRODUCT OR DEVICE:** Your Attendant Medical Cart, which is distributed by Direct Supply Manufacturing, Inc. (“We,” “Us” or “Our”).

**“You”:** Means the User, Caregiver and any other individuals who will be using or servicing the Product.

**“Distributor”:** Means the reseller or other entity from whom you purchased this product.

**Caregiver:** The individual(s) responsible for the care and needs of the User, such as a spouse, adult family member or home health aide. The Caregiver must be available at all times to provide the User with physical and cognitive assistance and provide supervision as needed.

**User:** The individual who uses the device.

# Introduction *(cont.)*

## Attention

1. Please read this User Guide before using product.
2. The correct ways to safely use this product are provided by this User Guide.
3. This User Guide lists attention points when you are using this product. If you miss the attention points and use incorrectly, it is possible to get hurt or damage this product.

**DO NOT** sit, lean on or tilt the product. Tilting the product with the drawers open allows the weight of the cart to rest on the drawers, causing permanent damage to the drawers and hinging.

**DO NOT** place heavy items on top of the product.

**DO NOT** store the product near an open flame.

When loading the product, ensure all casters are locked. Before moving the product, be sure to unlock all casters.

**FAILURE TO COMPLY WITH ALL INSTRUCTIONS, WARNINGS AND PRECAUTIONS, OR USING THE PRODUCT FOR A PURPOSE OTHER THAN THE RECOMMENDED USE, COULD RESULT IN BODILY INJURY.**

## Precautions

**⚠ WARNING:** Incorrect use of this cart can result in serious injury to operator and/or damage to the cart.

**! ATTENTION**

**X PROHIBITED**

# Safety Precautions

## ⚠ WARNING

Do not open two or more drawers at same time.

X




Carefully move cart and do not allow it to fall.

X




Pay attention to direction of cart when moving it.

!

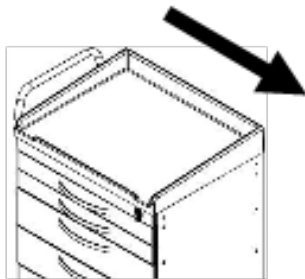

Pay attention to the front of cart when moving forward.

!

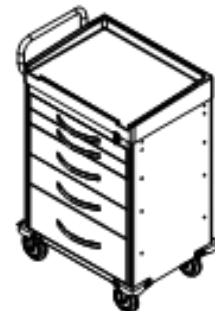

Do not use cart on stairs.

X



Do not force cart to turn when caster brake is locked.

X




## Safety Precautions *(cont.)*

### **⚠ WARNING**

To prevent falling, do not push on the back of cart.

**X**

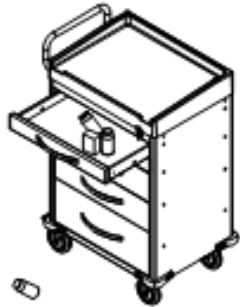

Do not move cart with an open drawer.

**X**




Do not move cart by pushing or pulling on accessory parts.

**X**




Do not move cart without unlocking caster brakes.

**X**

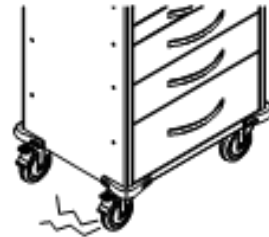


Attention!! Remember to lock caster brakes when you stop or park cart.

**!**

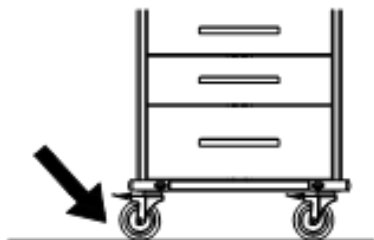


**⚠ WARNING**

**Attention!! Remember to tighten screws when you install parts and accessories.**

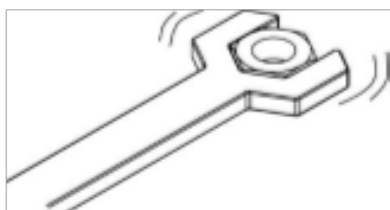

**!**

**Do not overload top of cart with 2,200 lbs or 1,000 kg.**

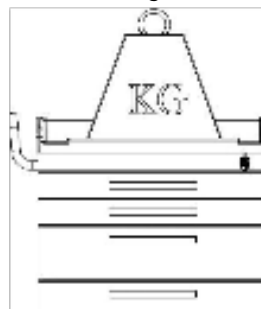


**X**

**Do not overload drawers with 45 lbs or 20 kg.**




**X**

**Attention!! Always close and lock drawers.**

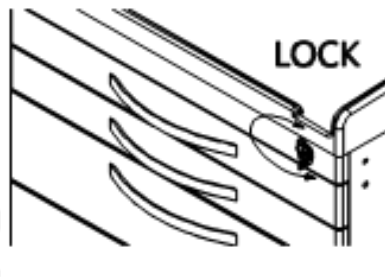


**!**

**Do not park cart on an inclined or un-level floor.**

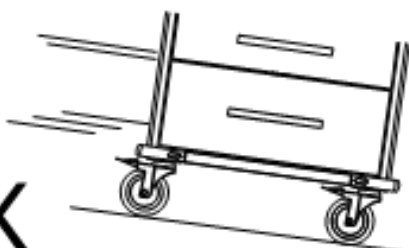

**X**

**Attention!! Park cart on flat level floor.**

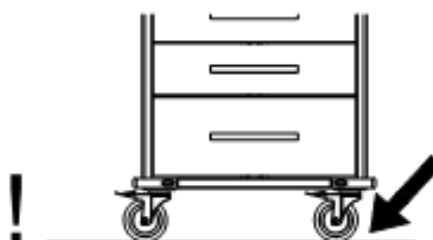


**!**

## Safety Precautions *(cont.)*

### **⚠ WARNING**


Do not lean against cart with open drawer.

**X**

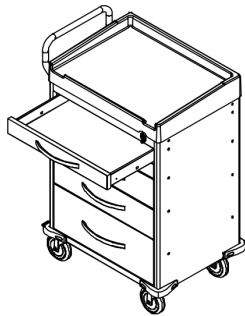


Do not move cart with an open drawer.

**X**

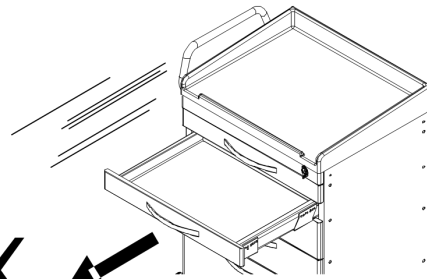


Do not strike cart with objects (ex: Hammer).

**X**

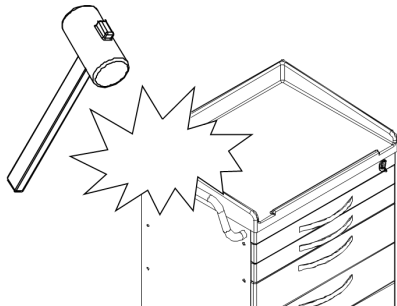


Do not pour hot or cleaning liquids into cart drawer.

**X**

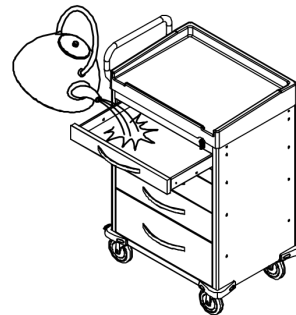



Do not put high temperature products on cart.

**X**

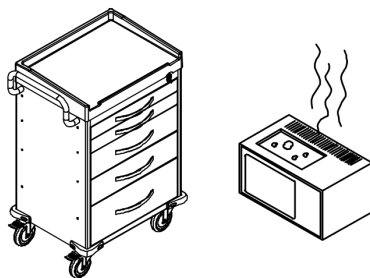



**⚠ WARNING**

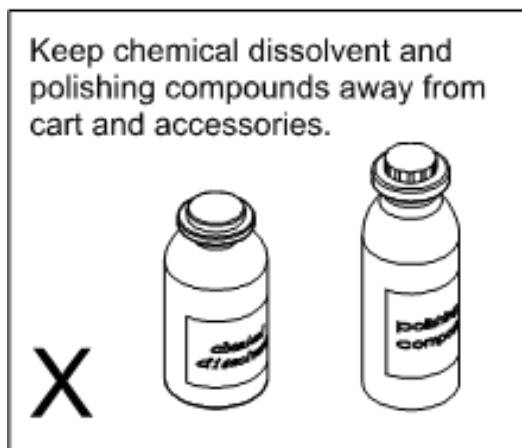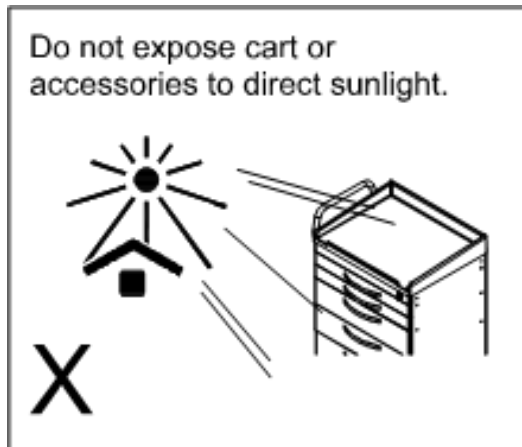

**! ATTENTION:** It is recommended to use 75% neutral detergent to clean cart.

# Specifications

## Construction


- Frame material: steel dimensions (with push handle): 29.5"W x 21"D x 43"H
- Casters: 2 parking lock casters, 1 steering lock caster, and 1 swivel caster
- Top material: ABS
- Finish: powder coat

## Standard Features:

Attendant's medical carts come in multiple color options (red, blue, yellow, and white) with five or six drawers and 360-degree rotation for tight corners. Attendant's carts are manufactured from quality steel cart body construction that is powder-coat painted for durability with ABS countertops that are easy to clean. The keyed or EMG breakaway central lock secures all drawers at the same time for added convenience. The carts' 5-inch-diameter wheels feature two parking locks and one steering lock. ABS bumpers protect all four corners of the medical carts. The push handle offers a user-friendly design. Multiple optional accessories are available to customize your cart or you may purchase a preloaded cart.

## Models


Product Number	Manual Description
0-37GMC	Anesthesiology Blue - 5 Drawer
0-37GMD	Anesthesiology Blue with Accessories - 5 Drawer
0-37GMF	Emergency Red - 5 Drawer
0-37GMG	Emergency Red with Accessories - 5 Drawer
0-37GMH	General Purpose White - 5 Drawer
0-37GMJ	Isolation Yellow - 5 Drawer
0-37GMK	Anesthesiology Blue - 6 Drawer
0-37GMM	Anesthesiology Blue with Accessories - 6 Drawer
0-37GMP	Emergency Red - 6 Drawer
0-37GMQ	Emergency Red with Accessories - 6 Drawer
0-37GMR	General Purpose White - 6 Drawer
0-37GMT	Isolation Yellow - 6 Drawer

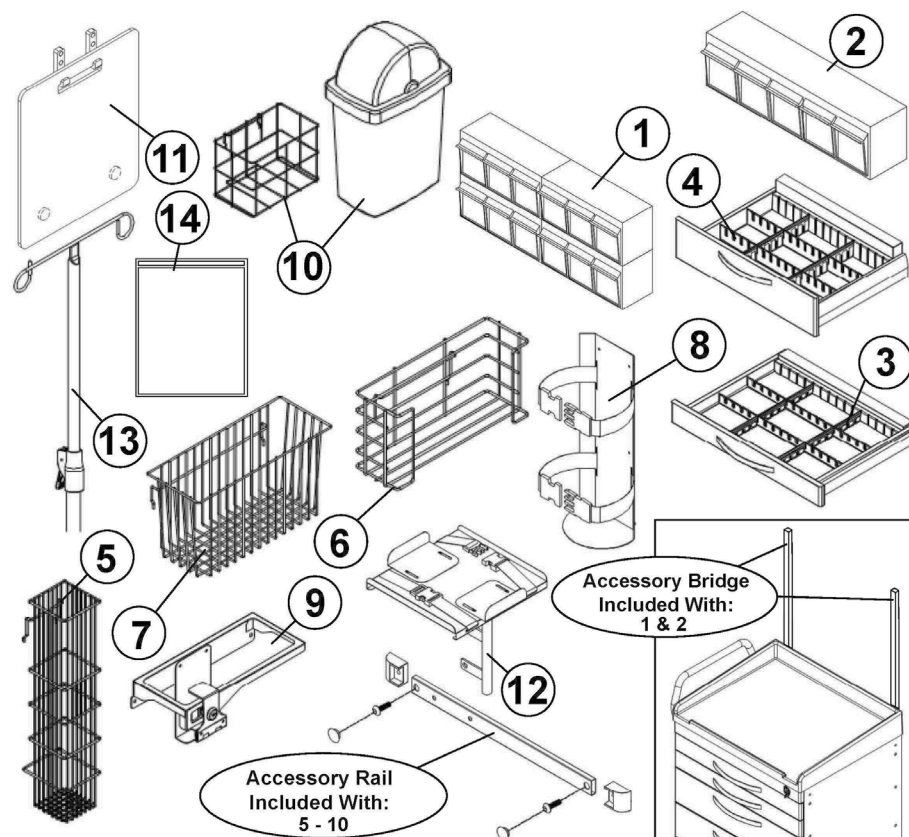


## Accessories and Replacement Parts

Diagram Number	Product Number	Description	Included in pre-loaded cart
1	37GMV	12 Bin Organizer with Accessory Bridge	Anesthesiology Blue
2	37GMX	5 Bin Organizer with Accessory Bridge	
3	37GMW	3" Drawer Dividers	
4	37GMY	6" Drawer Dividers	
5	37GP3	Catheter Holder with Accessory Rail	Anesthesiology Blue
6	37GP6	Glove Dispenser Holder with Accessory Rail	
7	37GP8	Multi-storage Bin with Accessory Rail	
8	37GP9	Oxygen Tank Holder with Accessory Rail	Emergency Red
9	37GPC	Sharps Container Holder with Accessory Rail	Emergency Red and Anesthesiology Blue
10	37GPD	Waste Container and Lid with Accessory Rail	Anesthesiology Blue
11	37GP2	CPR Cardiac Board and Mounting Hooks	Emergency Red
12	37GP4	Defibrillator Shelf	Emergency Red
13	37GP7	IV Pole	Emergency Red
14	37GPB	50 Red Plastic Seals, individually numbered	


# Optional Side Accessories

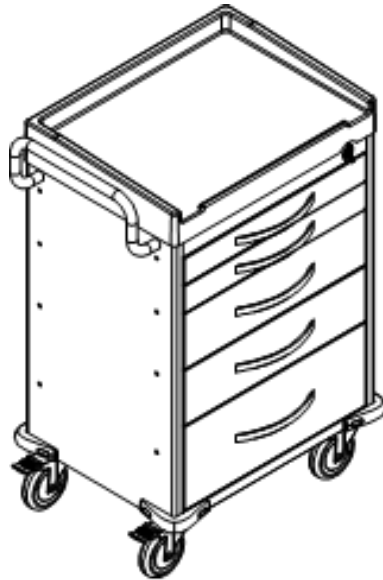


## Tool and Accessory Package (Standard)

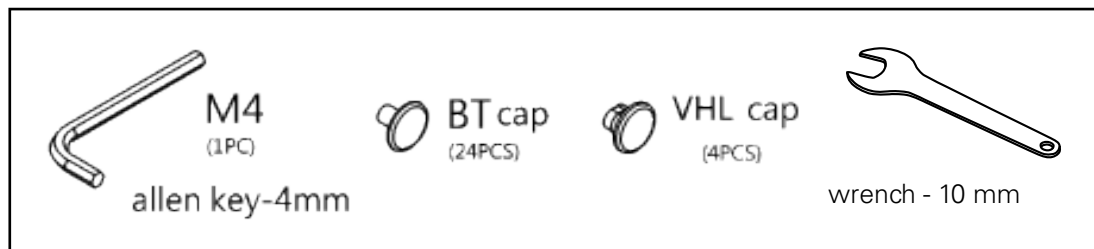

## Installation of caps

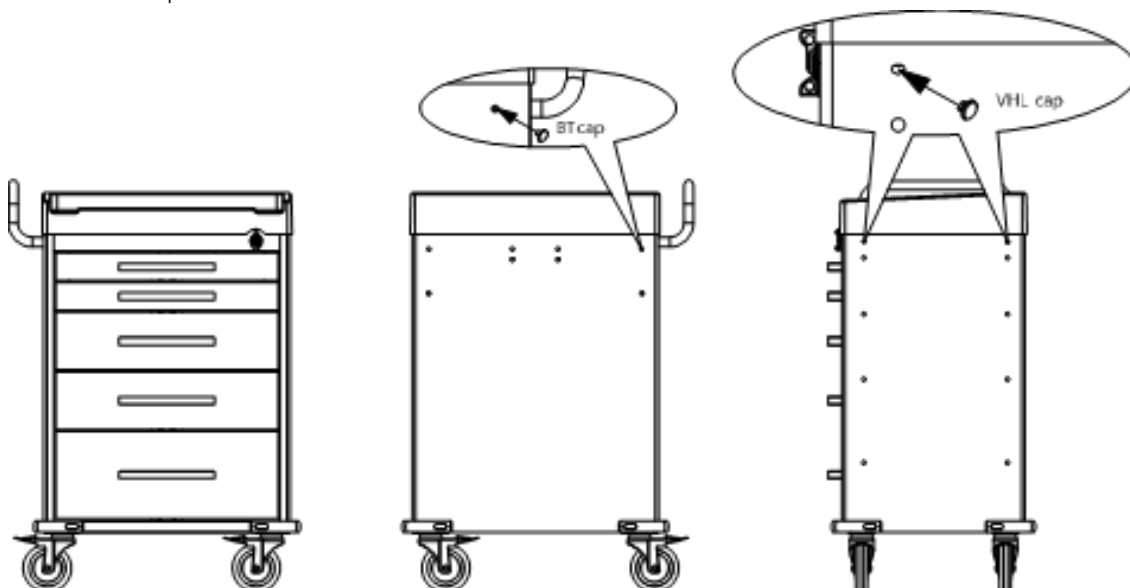


## Push Handle

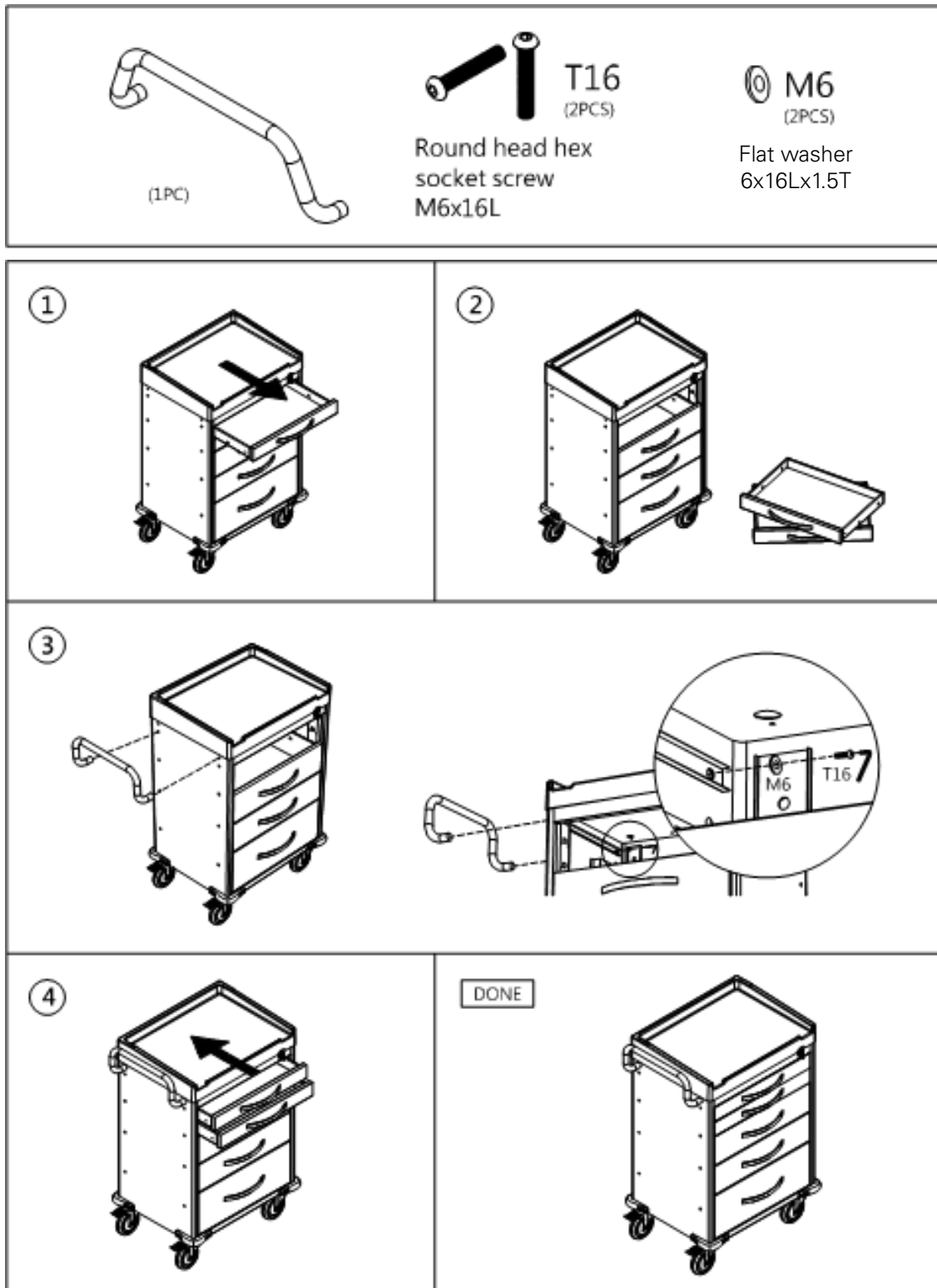


# Optional Side Accessories *(cont.)*

## Removal of Drawer



1. Pull out a drawer

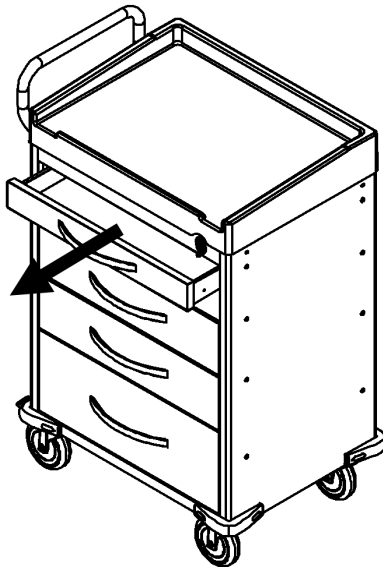

2. Push release levers in middle of both slides, one up and one down.

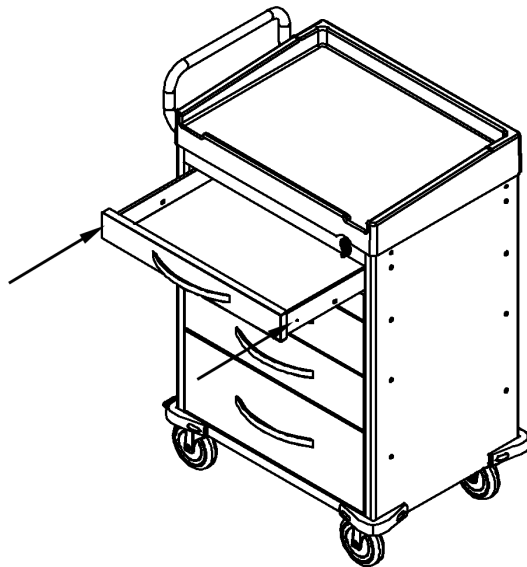


## Accessory Rail Instalation

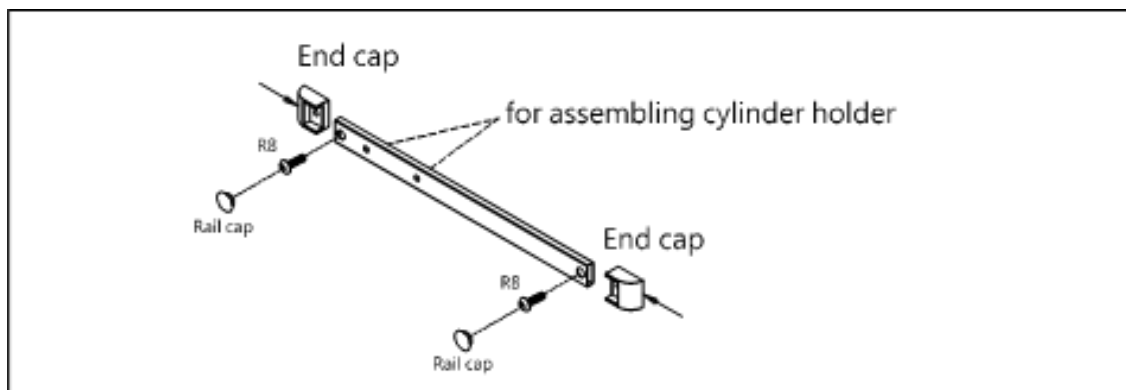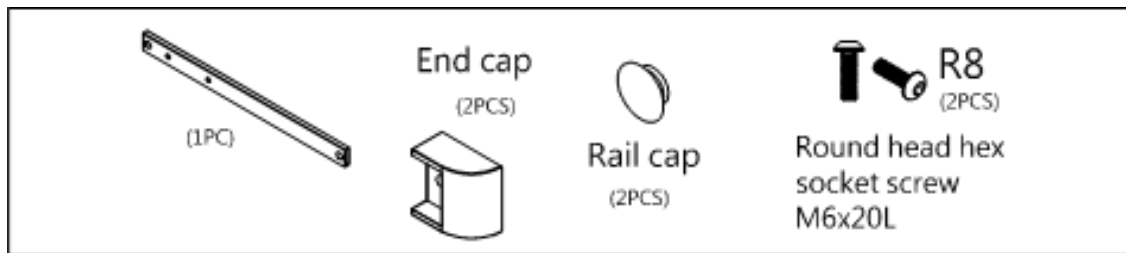


## Optional Side Accessories (cont.)

### Oxygen Tank Holder with Accessory Rail - 37GP9

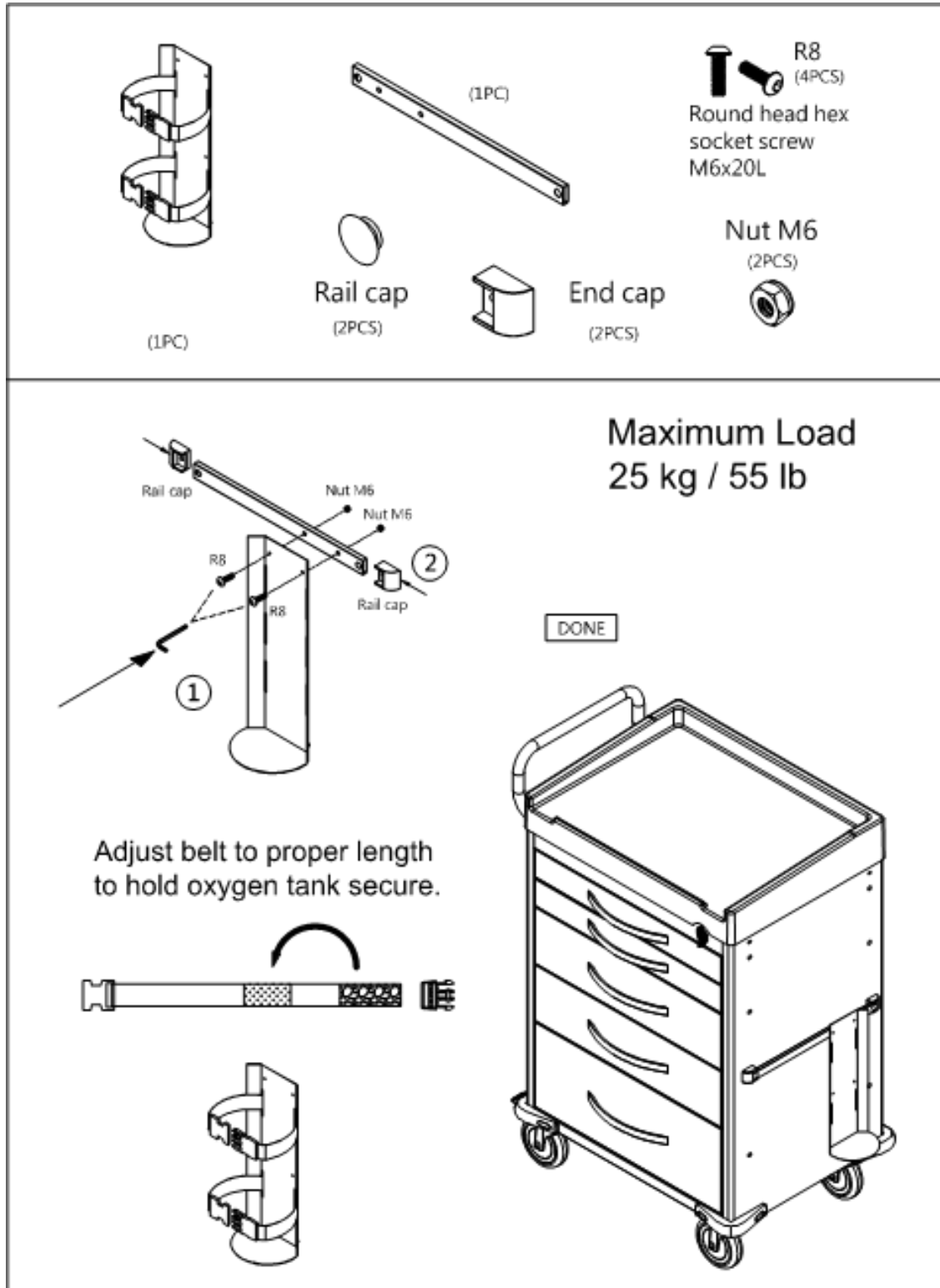





### Catheter Holder - 37GP3

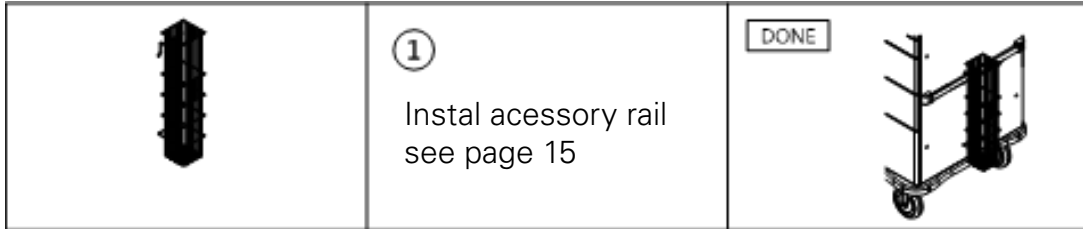


### Glove Dispenser Holder - 37GP6

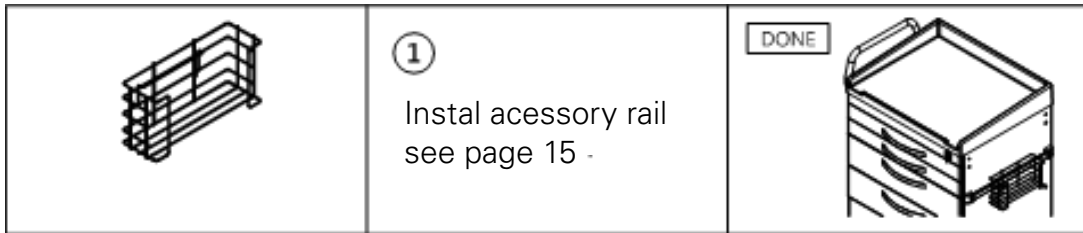

### Multi Bin Storage Basket - 37GP8

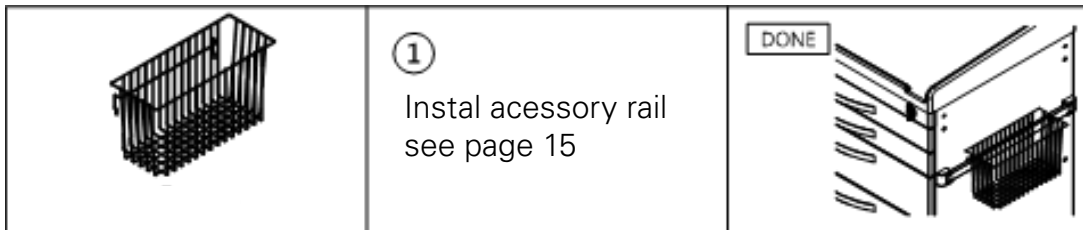


### Waste Container and Lid - 37GPD

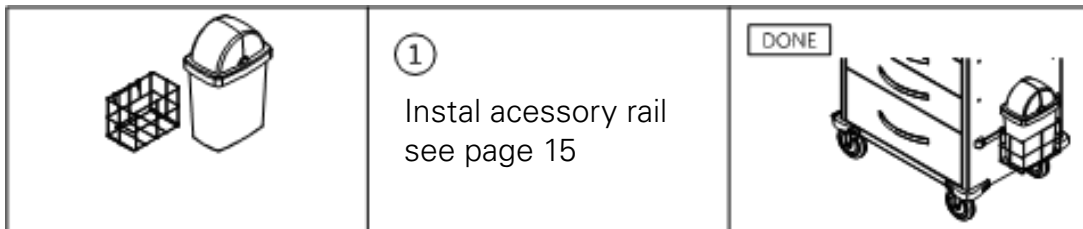


## Optional Side Accessories *(cont.)*

### Sharps Container Holder - 37GPC

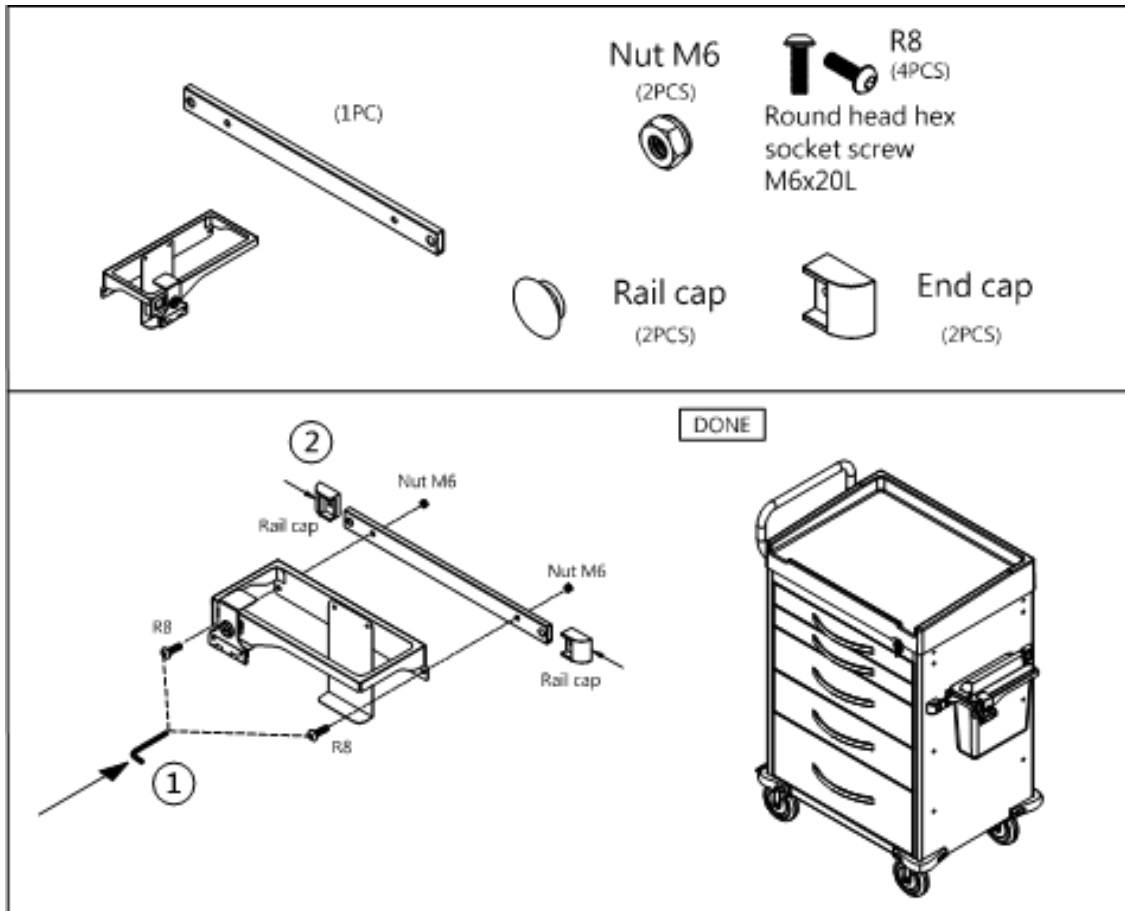


# Optional Accessories - Back & Top

## CPR Cardiac Board with Mounting Hooks - 37GP2

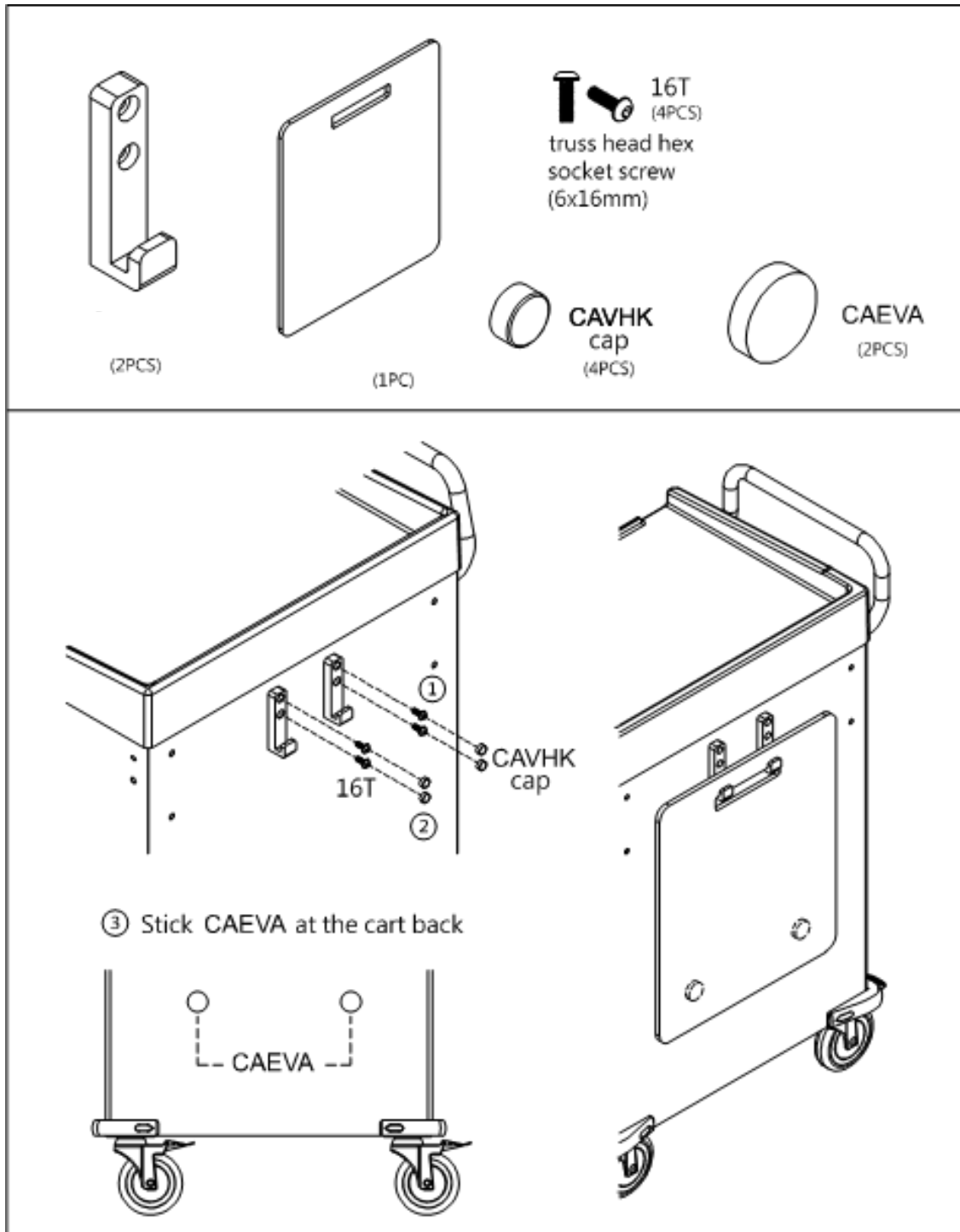


## Optional Accessories - Back & Top (cont.)

### Defibrillator Shelf - 37GP4


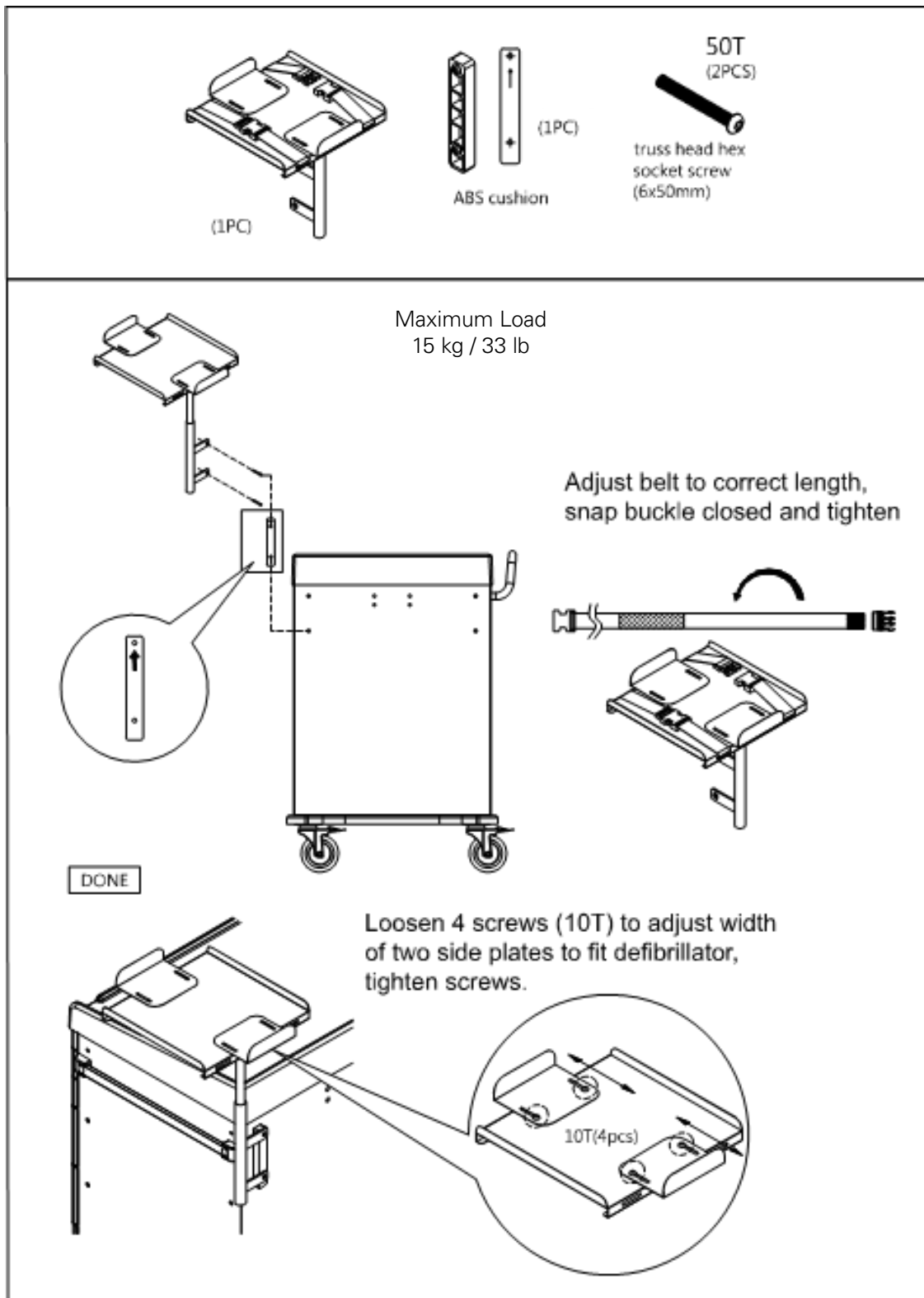


## IV Pole - 37GP7


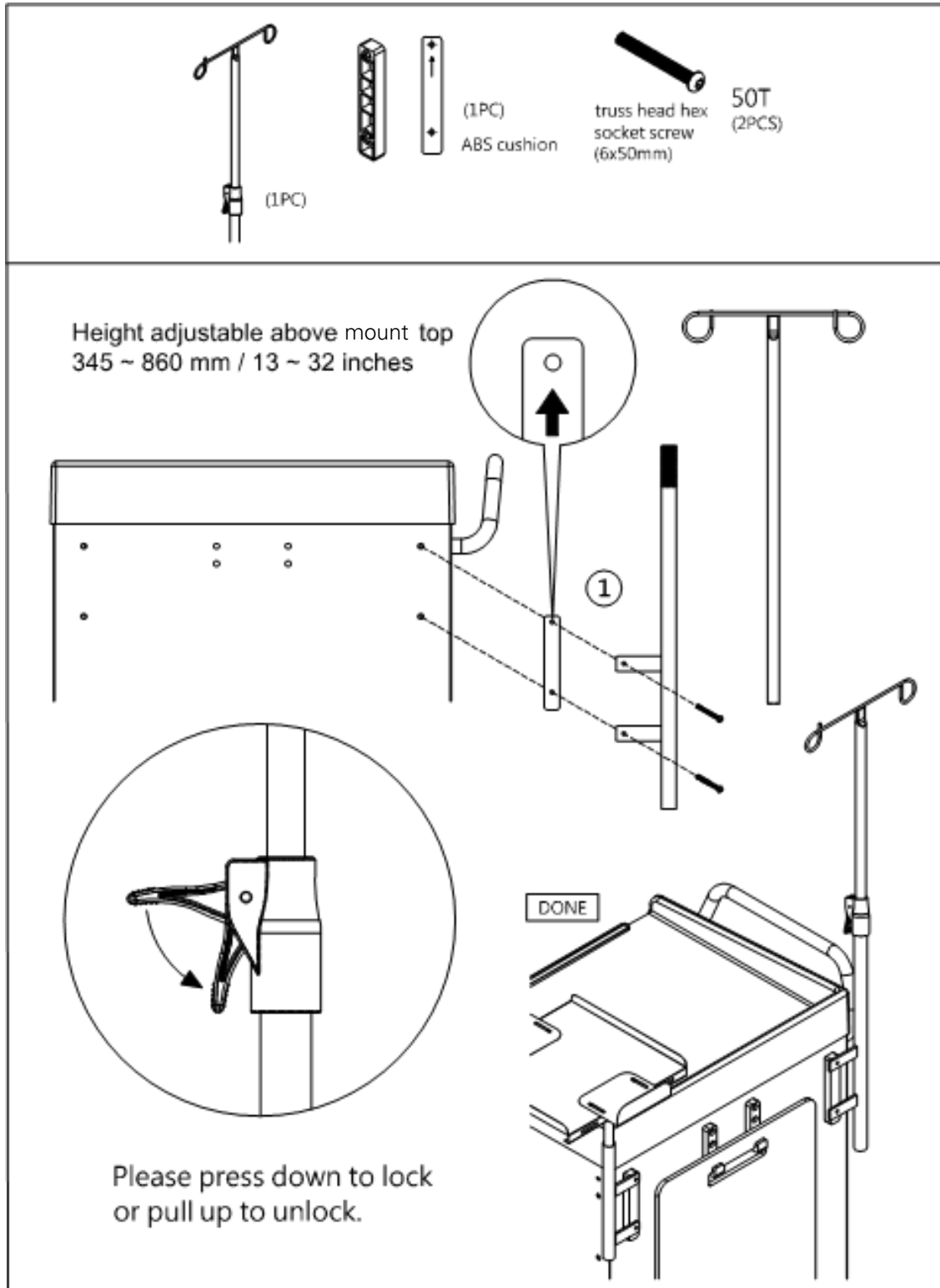


## Optional Accessories - Back & Top (cont.)




### 5 and 12 Bin Organizer with Accessory Bridge - 37GMX and 37GMV

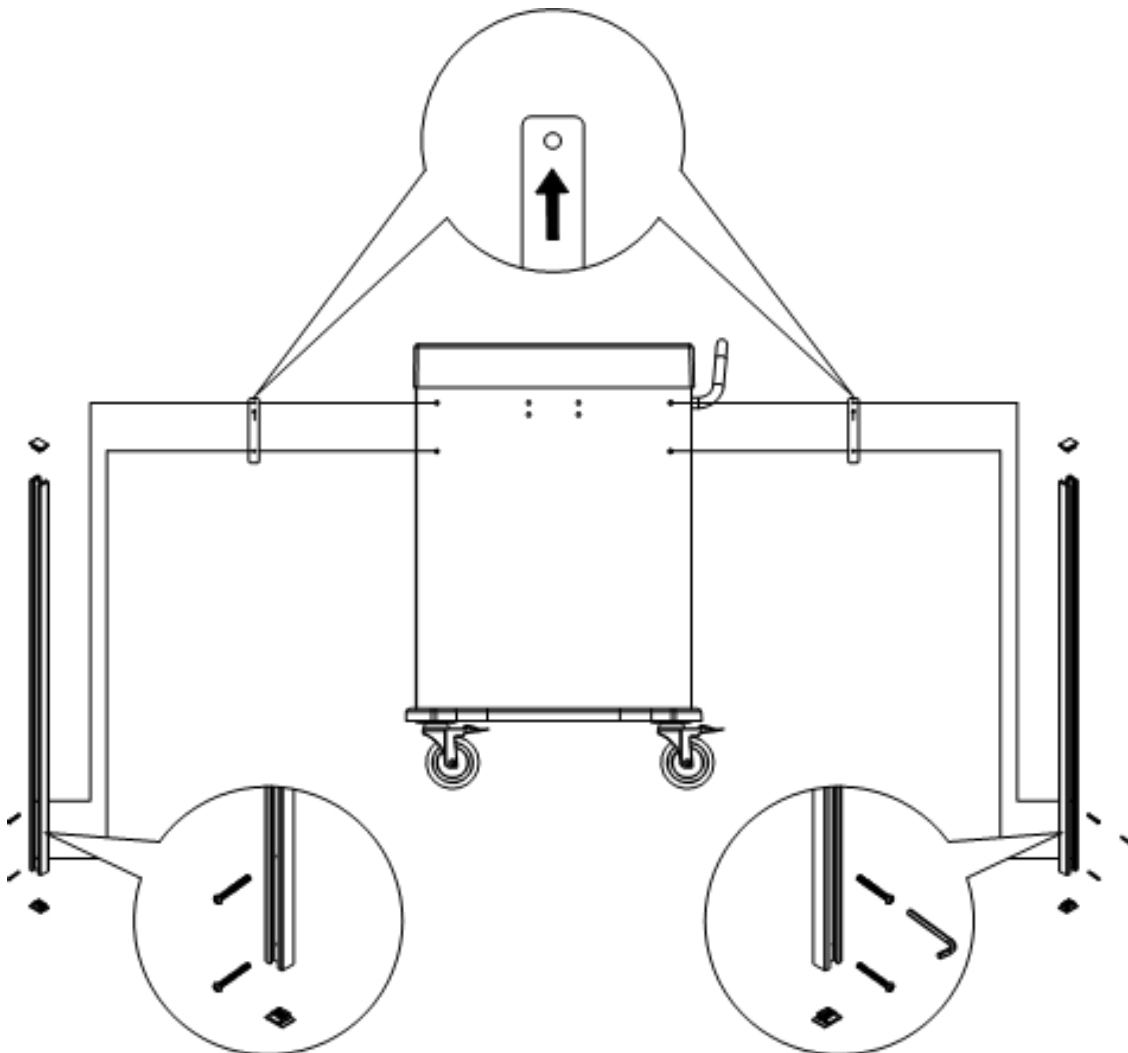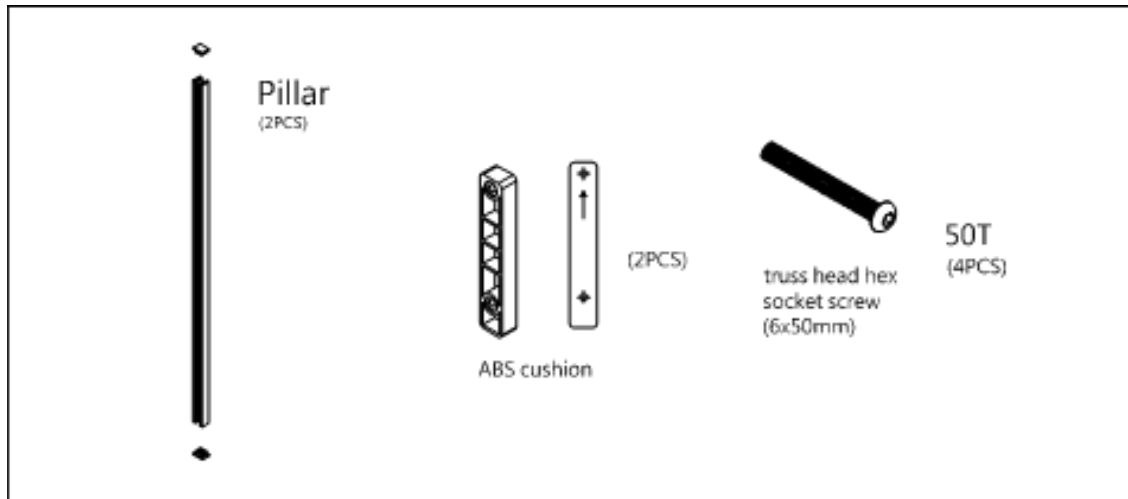



## 5 and 12 Bin Organizers with Accessory Bridge - 37GMX and 37GMV (cont.)

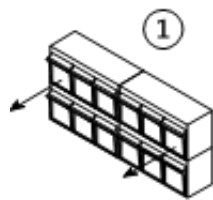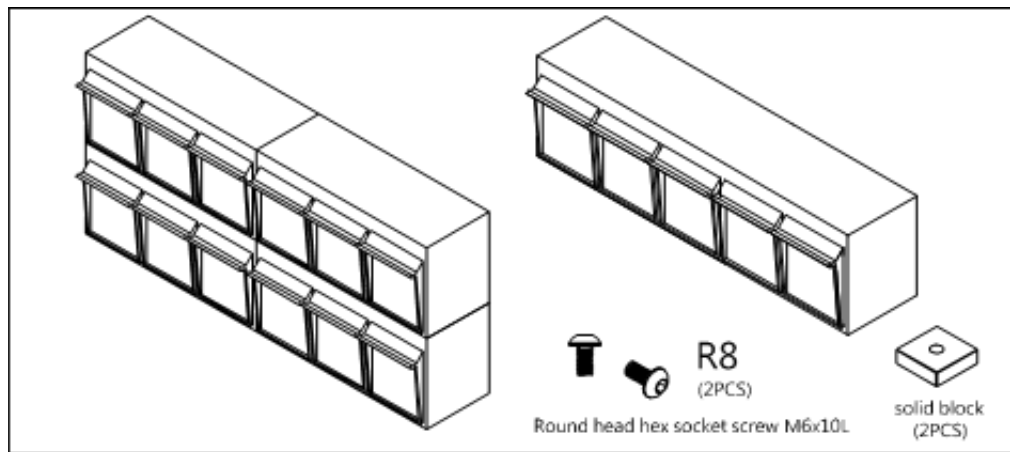

12 Bin Organizer -  
Remove  
drawers (#1 & #6)

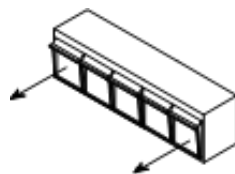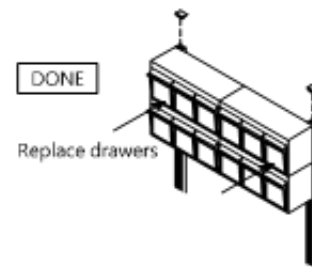
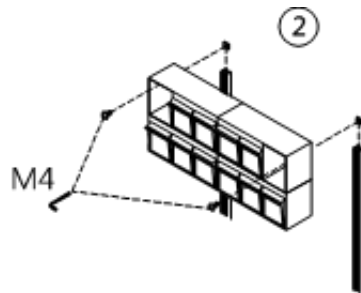


5 Bin Organizer -  
Remove  
drawers (#1 & #5)

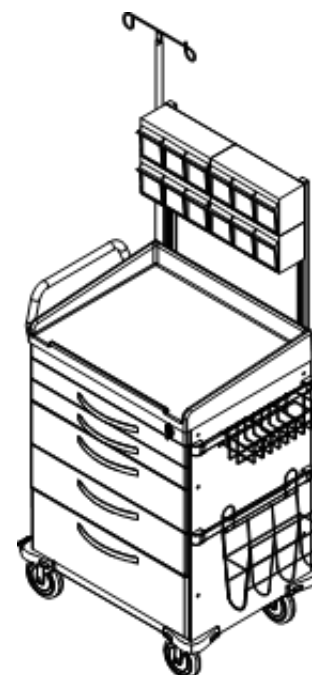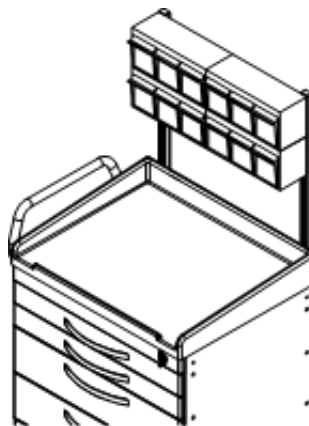

## Optional Accessories - Back & Top *(cont.)*

### Drawer Dividers System

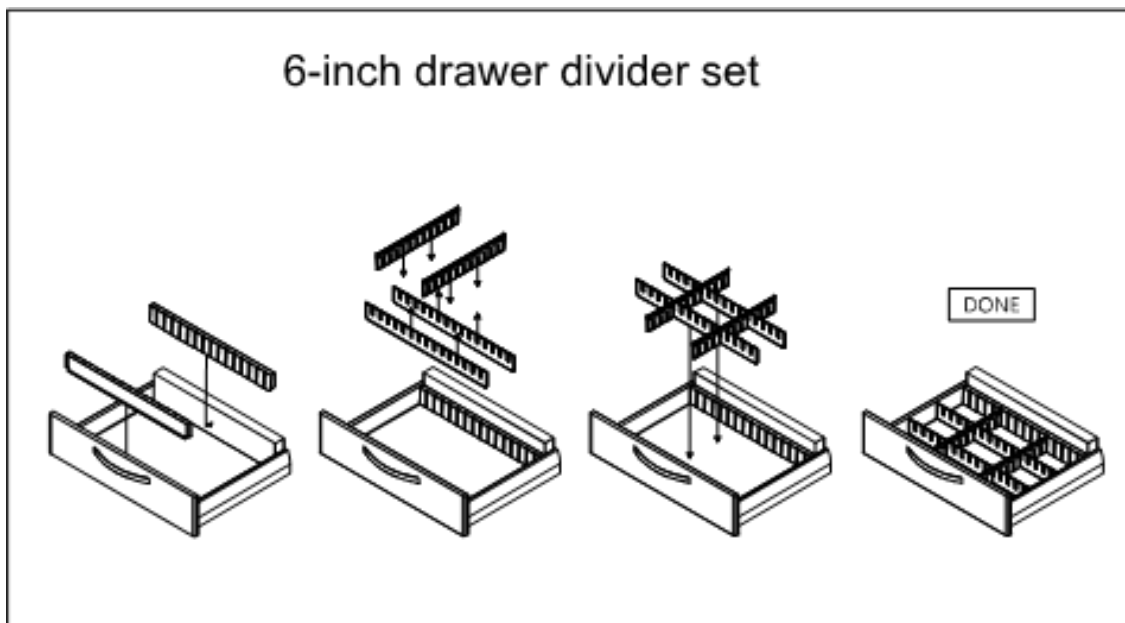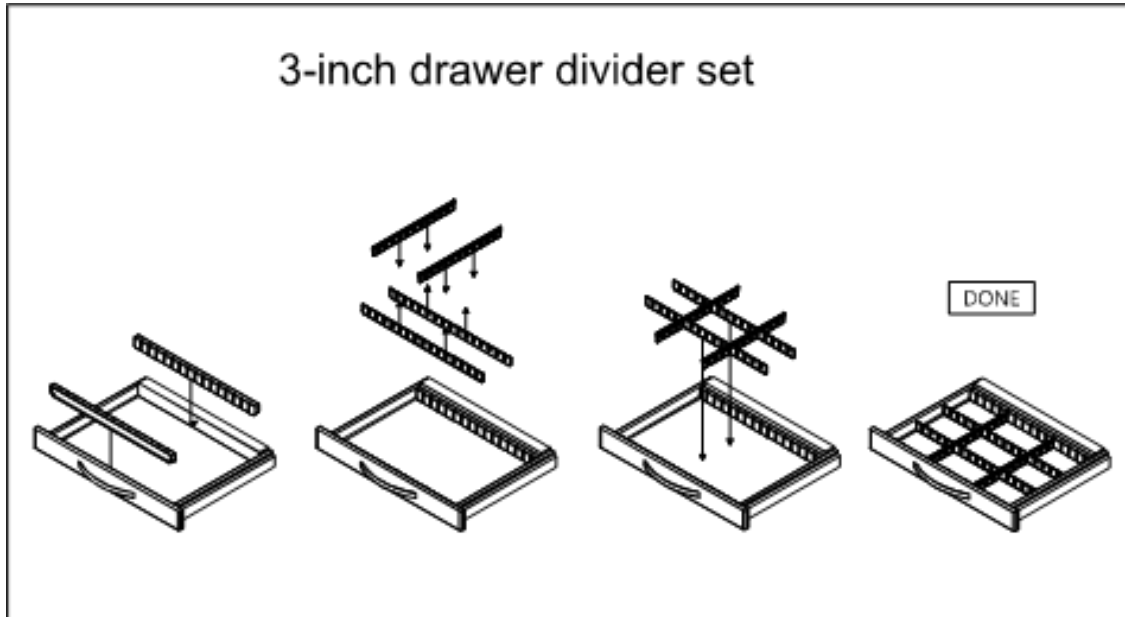

**⚠ WARNING:** Maximum load is 10 kg or 22 lbs



# Limited Warranty

We offer to you, as the original purchaser, a warranty for the Direct Supply® Attendant Medical Cart. Our warranty applies for the limited warranty period stated below. If any device or device part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a device or device part will be at our discretion. Our warranty applies only if the device is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the device, or damage caused by abuse, improper use, failure to maintain, use which exceeds the published device limitations, or the combination of any device with another product. In addition, our warranty does not cover fading, characteristics or natural variations in fabric, texture, colorfastness, stains, spills, or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts, and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary, from state to state. Please note that our limited warranty period begins when we ship the device to you. The limited warranty period and our obligations under the warranty end once you transfer the device to someone else, or at the end of the applicable limited warranty period identified below, whichever is earlier.

<b>Product/Part</b>	<b>Warranty Period (Parts and Labor)</b>	<b>Warranty Period Accessories</b>	<b>Anticipated Usable Device Life</b>
Attendant Medical Cart	12 years	2 years	12 years

Anticipated Usable Device Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the device as described in this guide, as the device may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR CONSEQUENTIAL OR INCIDENTAL DAMAGES THAT MAY RESULT FROM THE DEVICE OR YOUR USE OF OR INABILITY TO USE THE DEVICE. TO THE EXTENT ALLOWED BY LAW, OUR TOTAL LIABILITY TO YOU,

IF ANY, IS LIMITED TO THE PRICE OF THE DEVICE OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.

# Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality medical cart and enjoy outrageous customer service. If you have any questions about the medical cart you have purchased or would like to request warranty service, please contact the distributor from whom you purchased this medical cart.

For a complete list of replacement parts for your Direct Supply® Attendant Medical Cart, please visit **store.directsupply.com** or please contact your distributor.



Distributed by Direct Supply® Manufacturing™, Inc. 6635 W Champions Way, Milwaukee WI 53223.

© 2023-2024 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply, Attendant and all associated logos are the registered trademarks of Direct Supply, Inc., used under license. All other trademarks are the property of their respective owners.