



Direct Supply® Attendant® Handheld Pulse Oximeter Rechargeable Battery and Charger Base

Thank you for purchasing a Direct Supply® Attendant® Handheld Pulse Oximeter Rechargeable Battery and Charger Base from Direct Supply® Equipment & Furnishings™. Please read this entire insert carefully and keep it for future reference.

READ THE ENTIRE OWNER'S MANUAL FOR THE ATTENDANT HANDHELD PULSE OXIMETER PRIOR TO USING THE CHARGER BASE AND RECHARGEABLE BATTERY PACK. Refer to your Owner's Manual for additional warnings and important definitions and symbols.

Package Contents:

- Rechargeable Battery Pack
- Charger Base
- Power Cord for Charger Base

⚠ WARNING: The battery pack, charger base and power cord are intended exclusively for use with the Attendant Handheld Pulse Oximeter. Use of these accessories with any other device may damage the device(s), affect the functionality of the device(s), cause injury to staff or residents and void the limited warranty.

⚠ WARNING: The charger base is only designed to recharge the battery pack that it ships with. Do not attempt to charge alkaline batteries with the charger base.

⚠ WARNING: Do not operate charger base with damaged cord or plug or if it has been severely damaged in any way. Disassembly may result in a risk of electric shock or fire.

CAUTION: Indoor use only. Dry location use only. Risk of electric shock – never disassemble or alter any part of the device. Device may become hot during charging and will cool down when charging is complete. Keep away from fire and other heat sources.

This device should not be used by individuals with reduced physical, sensory or mental capabilities.

NOTE: Always charge a new battery pack for a full 24 hours prior to first use.

Instructions for installing rechargeable battery


1. Remove handheld pulse oximeter from protective boot
2. Remove the battery door
3. Remove the AA batteries from the battery compartment
4. Insert the battery pack into the battery compartment, following the orientation shown below
5. Put the battery door back on
6. Return the handheld pulse oximeter back to its protective boot

Using the charger base

1. Plug one end of the power cord into the back of the charger base
2. Plug the other end of the power cord into a 120V AC outlet
3. Insert the handheld pulse oximeter w/protective boot into the charger base.

The charging LED on the charger base will be:

- **Red** when the handheld pulse oximeter is not docked
- **Orange** when the handheld pulse oximeter is docked and charging
- **Green** when the handheld pulse oximeter is docked and fully charged

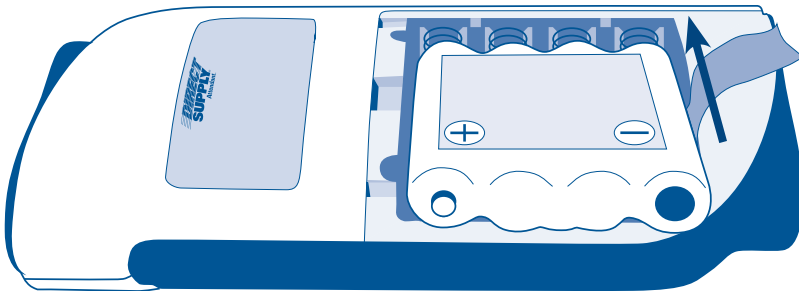

NOTE: Be sure to insert the battery pack in the proper positive/negative orientation or the unit will not operate.

Disposal

Dispose of the battery pack when it no longer holds a charge or when a noticeable degradation of charging capabilities occurs. Always follow state and local ordinances regarding battery disposal.

Limited Warranty

We offer to you, as the original purchaser, a warranty for the Attendant Handheld Pulse Oximeter Rechargeable Battery and Charger Base. Our warranty applies for the limited warranty period stated below. If any device or device part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a device or device part will be at our discretion. Our warranty applies only if the device is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear,

modification of the device or damage caused by abuse, improper use, failure to maintain, use which exceeds the published device limitations or the combination of any device with another product. In addition, our warranty does not cover fading, colorfastness, stains, spills or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary from state to state. Please note that our limited warranty period begins when we ship the device to you. The limited warranty period and our obligations under the warranty end once you transfer the device to someone else or at the end of the applicable limited warranty period identified below, whichever is earlier.

| Product/Part | Warranty Period | Anticipated Useable Device Life |
|---|-----------------|---------------------------------|
| Attendant Handheld Pulse Oximeter Rechargeable Battery and Charger Base | 1 year | 1 year |

Anticipated Usable Device Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the device as described in this guide, as the device may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE DEVICE OR YOUR USE OR INABILITY TO USE THE DEVICE EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE PRODUCT OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.

Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality Attendant Handheld Pulse Oximeter Rechargeable Battery and Charger Base, and enjoy outrageous customer service. If you have any questions about the Attendant Handheld Pulse Oximeter Rechargeable Battery and Charger Base you have purchased or would like to request warranty service, please contact: Direct Supply Equipment & Furnishings at 1-800-634-7328, 6767 North Industrial Road, Milwaukee, WI 53223, deardirect@directsupply.net.



Taking Cost Out of Healthcare®



1-800-634-7328 • directsupply.com

Direct Supply® Attendant® products are available to purchase exclusively through Direct Supply Equipment & Furnishings, a division of Direct Supply®, Inc.

Distributed by Direct Supply Manufacturing, Inc. 6699 N Industrial Road, Milwaukee WI 53223.
©2011 Direct Supply Manufacturing, Inc. All Rights Reserved.

Direct Supply®, Attendant®, and all associated logos are the registered trademarks of Direct Supply, Inc., used under license.