



Panacea®

RESIDENTIAL MATTRESS FOUNDATION

Assembly Instructions

Identify Components to Start Assembly

- Head/Foot Perimeter Base Frame (2)
- Side Perimeter Base Frame (2)
- Corner Pins (4)
- Slat Assembly (2)
- Center Support Bar (1)




Step 1: Perimeter

- Orient the head/foot and side components to form the perimeter of the foundation..
- Align the plastic brackets at each corner.
- Push the corner pin into the hole in the plastic bracket. Press firmly to ensure tight alignment.
- Repeat to secure the brackets in the remaining three corners.



Step 2: Center Support Bar

- Align the center bar over the metal rail on the head and foot perimeter base frame.
- Slide the center bar into the rail on both ends. Press firmly to ensure tight alignment.

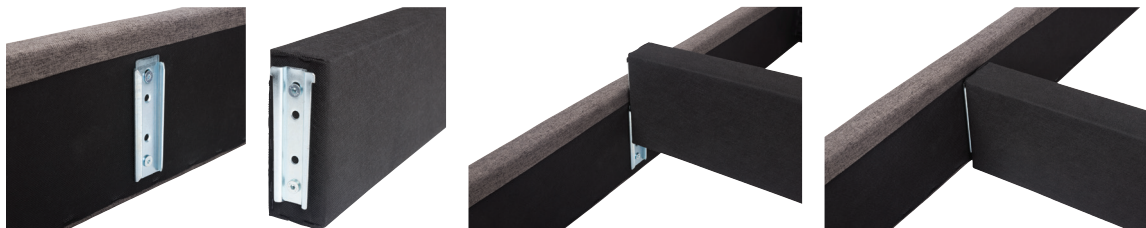

Step 3: Slats

Rest the slat assemblies on top of the side perimeter frame.



Directions for Use

⚠ CAUTION: Do not use razor blades to cut packaging away from the foundation.

Inspect the foundation for shipping damage. If the product is damaged, DO NOT USE and immediately contact Direct Supply Equipment & Furnishings® for further instruction.

The Direct Supply Residential Foundation is intended for use with a mattress.

⚠ WARNING: The weight capacity of the Residential Foundation is 350 lbs. for Twin, Twin XL, and Full-size configurations, 500 lbs. for Queen-size configuration. DO NOT exceed the listed weight capacity.

Limited Warranty

We, Direct Supply Manufacturing, Inc. offer to you, as the original purchaser, a warranty for the Direct Supply® Panacea® Residential Foundation. Our warranty applies for the limited warranty period stated below. If any product or product part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a product or product part will be at our discretion.

Our warranty applies only if the product is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the product or damage caused by abuse, improper use, failure to maintain, use which exceeds the published product limitations or the combination of any product with another product. In addition, our warranty does not cover fading, colorfastness, stains, spills or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary from state to state. Please note that our limited warranty period begins when we ship the product to you. The limited warranty period and our obligations under the warranty end once you transfer the product to someone else or at the end of the applicable limited warranty period identified below, whichever is earlier.

	Warranty Period (Parts)	Anticipated Usable Product Life
Direct Supply® Panacea® Residential Foundation	1 year	1 year

Anticipated Usable Product Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the product as described in this guide, as the product may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE PRODUCT OR YOUR USE OF OR INABILITY TO USE THE PRODUCT EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE PRODUCT OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages, or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.

Customer Service

If you have any questions about the Residential Foundation you have purchased or would like to request warranty service, please contact your distributor.



Distributed by Direct Supply® Manufacturing™, Inc. 6635 W Champions Way, Milwaukee WI 53223.

© 2024 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply, Panacea and all associated logos are the registered trademarks of Direct Supply, Inc., used under license. All other trademarks are the property of their respective owners.