



Direct Supply® Panacea®

Resting Hand Orthosis with Keyhole Strap

Thank you for purchasing a Direct Supply® Panacea® Resting Hand Orthosis with Keyhole Strap from Direct Supply Equipment & Furnishings®, a division of Direct Supply, Inc. Please read this entire manual carefully and keep it for future reference. This manual will provide you with instructions, warnings, warranty information, and other important information about your device. Share this information with your housekeeping, nursing and maintenance staff to help ensure the device is cared for properly. **Please keep and refer to this Owner's Manual.**

Definitions

The Direct Supply Panacea Resting Hand Orthosis with Keyhole Strap ("Device") is distributed by Direct Supply Manufacturing, Inc. , referred to throughout this manual as "We," "Us" or "Our." "You" refers to the facility, community or other entity that has purchased the device.

Safety Notices – Warnings and Cautions

Read and follow all directions and warnings before using this device. Do not use this device without first reading and understanding this user manual. Damage, injury or even death may result from improper use of this device or not following directions and warnings. This device is intended for use by or under the supervision of trained and licensed healthcare or senior care providers in normal, indoor conditions.

⚠ WARNING: Immediately discontinue device use and contact appropriate healthcare personnel for further instructions if the device user experiences any of the following while using the device:

- Swelling
- Pain
- Skin Redness

⚠ WARNING: Device must be used with the soft cover.

⚠ WARNING: Refer to your facility's procedures regarding infection control.

⚠ WARNING: This device should only be applied by trained personnel.

⚠ WARNING: Inspect the device for damage before each use and do not use if device appears to be damaged.

⚠ CAUTION: Do not place this device in an industrial washing machine or dryer.

Device Description

The Direct Supply Panacea Resting Hand Orthosis with Keyhole Strap (Model # DS-HRE-KH) is a lightweight, prefabricated hand orthosis featuring a malleable wrist support integrated with a resting pan that may be molded to aid individual contracted fingers while helping to promote increased range of motion and extension. The frame can support a relaxed hand as well as a more toned, contracted wrist. The soft goods are made of fabric for a soft feel against the skin that may help wick away perspiration. There is a simple keyhole strap that fits over the deviation wings of the resting pan. This device also has a removable and washable cover.

Directions for Use

To apply the device: First, ensure the soft cover is properly attached to the device. Open the hook-and-loop closures. Carefully open the resident's fingers and position the palmar section of the device against the palm of the hand. Gently position the resident's thumb under the pan. Fasten the hook-and-loop closure over the wrist. Check that it is secure but does not interfere with the resident's circulation. Fasten hook-and-loop closures over the back of the fingers between the metacarpal phalangeal (MP) joints and the proximal interphalangeal (IP) joints. Position the strap opening over the lateral/medial wings. Again, check that the straps are secure but do not interfere with the resident's circulation. Check the fit of the device on the resident periodically and more frequently over time. Your healthcare professional may provide a wearing schedule for your resident.

To launder: Separate the soft cover from the hand and wrist supports. On the soft cover, fasten all hook and loop closures, place in the provided laundry bag, and launder with a mild detergent at a low temperature. Afterwards, rinse thoroughly with cool water and let air dry or tumble dry at a low temperature. For the hand and wrist supports, clean with a mild detergent, rinse with clean water, and dry with a soft cloth.

RESIDENT'S NAME: _____

ROOM #: _____ HAND: _____
LEFT RIGHT

DIRECT SUPPLY PANACEA RESTING HAND ORTHOSIS SHOULD BE WORN AT TIMES SPECIFIED:

7 - 3 shift	3 - 11 shift	11 - 7 shift
_____	_____	_____
_____	_____	_____
_____	_____	_____

Limited Warranty

We, Direct Supply Manufacturing, Inc., offer to you as the original purchaser a warranty for the Direct Supply Panacea Resting Hand Orthosis with Keyhole Strap. Our warranty applies for the limited warranty period stated below. If any device or device part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a device or device part will be at our discretion.

Our warranty applies only if the device is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the device, or damage caused by abuse, improper use, failure to maintain, use which exceeds the published device limitations, or the combination of any device with another device. In addition, our warranty does not cover fading, colorfastness, stains, spills, or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts, and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary, from state to state. Please note that our limited warranty period begins when we ship the device to you. The limited warranty period and our obligations under the warranty end once you transfer the device to someone else, or at the end of the applicable limited warranty period identified below, whichever is earlier.

Device	Warranty Period (Parts)	Anticipated Usable Device Life
Direct Supply® Panacea® Resting Hand Orthosis with Keyhole Strap	90 days	90 days

Anticipated Usable Device Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the device as described in this guide, as the device may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE DEVICE OR YOUR USE OF OR INABILITY TO USE THE DEVICE EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE DEVICE OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.

Customer Service

Our promise to you is that you will have a convenient and easy ordering experience, receive a quality Direct Supply Panacea Resting Hand Orthosis with Keyhole Strap, and enjoy outrageous customer service. If you have any questions about the Resting Hand Orthosis you have purchased or would like to request warranty service, please contact:

Direct Supply Equipment & Furnishings at 1-800-634-7328, 6767 N Industrial Road, Milwaukee, WI 53223, deardirect@DirectSupply.net.